Instructions for use

Breast Shells

Bruskansvinning

Vårtskydd

Brugsanvisning

Brystskjold

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Rintasuojat

Bruskansvinning

Brystskjold

Breast Shells

The Medela Breast Shells allow unique weaning comfort. The discreet shape and the flexible silicone membrane, which is gentle on the skin, mean that they are pleasant and invisible to wear.

The ventilation holes in the top of the shells allow air to circulate, thus preventing humidity and promoting healing.

The Breast Shells provide an excellent nursing and weaning (in part as a reason for the soft breast). Milk leakage is smoothly collected. The shells can be worn continuously or between breastfeeding sessions. If necessary, be sure to consult your pediatrician or lactation consultant. Avoid weaning at night to prevent breast spots.

Assembly and Use

Put the silicone membrane on the shell. Be sure it snaps on all the way around. Place the assembled Breast Shells inside your bra with the opening in the silicone membrane centered over your nipple. The ventilation holes in the shell must point upwards. We recommend wearing a nursing bra when using a shell; nursing bras are easier and the shell will fit more comfortably.

Caution: Do not feed the breast milk collected in the shell to the baby. Discard this milk.

Cleaning Instructions

Before first use and subsequently once daily

Place the separated parts in a small pan filled with cold water. Distilled water is recommended — if hot water is used, a teaspoon of citric acid may be added to the water to avoid a build-up of inorganic. Bring the water to boil and allow it to continue boiling for approximately 5 minutes. Drain off the water and place the parts on a clean cloth to dry.

After each use

Separate the parts. Shells and membranes should be washed with a mild cleaning solution, rinsed thoroughly in cold water and dried on a clean cloth.

Note

Shells and membranes can be put in a dishwasher or autoclave.

Material

Shell: Polypropylene / Silicone membrane: Silicone

Disposal

According to local regulations. CE-mark according to 93/42/EEC

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Medela AG