Di Blasi®

Mod. R30

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GENERAL SAFETY WARNINGS

- Before operating the scooter DI BLASI, read carefully this manual so to avoid any personal injury or damage to the scooter resulting from improper use of it. In particular, when using the scooter, keep in mind the limitations arising from its technical characteristics, specified in Sec. 7 of this manual.
- The scooter is intended as an aid for the mobility of persons who are both physically and mentally capable to safely ride it. Therefore before riding this scooter consult your physician to check if your healthy conditions may limit your ability to safely ride it.
- The scooter is built to carry only its rider. The maximum allowed weight of the rider is 100 kg (220 lb).
- Do not modify the scooter: any modification can result in loss of safety for you and voids any warranty.
- Do not drive the scooter in the rain in order to avoid that the electronic components are wet and damaged. Avoid prolonged exposure to excessive heat or cold.
- The maximum range of the scooter, as specified at Sec 7 of this manual, is measured when the scooter is driven by a rider having a mass of 75 kg, on flat and plane road, without wind, with a new battery. The range is reduced if the driver's mass is greater than 75 kg, if the scooter is driven on slopes, rough ground, if it climbs obstacles, if the battery is not a new one or is damaged

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DRIVING SAFETY WARNINGS

- Obey all traffic rules and drive with caution. Pay attention to the motorized vehicles.
- In downhill or bad roads, reduce as much as possible the speed. In downhill never drive at an angle with direction of the road, so to prevent any tipping risk, and never drive backwards. On long descent stop frequently the scooter also to avoid overheating of the brake which could result in loss of its efficiency.
- Do not make any sharp turn: reduce always the speed when turning, especially when riding on sloping roads.
- The scooter can be used only on even paths. Do not ride it on potentially hazardous surfaces, like snow, ice, gravel road, etc.
- Do not make any use of the scooter which may lead to safety hazards for you and undue stresses for the scooter: for instance do not use it on uneven paths, as off road, as a transport vehicle, etc.
- Do not allow that your clothing or any other object drag behind or hang outside the seat of scooter when driving: they could be caught by the wheels and cause injury to you.

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SUGGESTIONS

- Save the packaging for future shipments of the scooter, for instance to service or to repair centre.
- In any communication to the seller concerning the scooter specify the frame number marked as shown in (F.01)

NOTICES

- Photos, drawings and descriptions in this manual are not binding: the manufacturer reserves the right to introduce at any time modifications and updates to the scooter without any obligation to update this manual.

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INDEX

Battery: charging	4.5.2
Battery : cautions	4.5.3
Brake: automatic one	4.3 (g)
Brake: emergency one	4.3 (h)
Charger: description	4.5.1
Charger: cautions	4.5.4
Controls of the scooter	1
Folding the scooter (automatically or manually)	3
Freewheel mode	4.3 (o)
Horn	4.3 (k)
Lighting	4.3 (l)
Maintenance	6
Operating instructions	4
Riding: instructions	4.3
Specifications	7
Standby mode	4.3 (n)
Tyre: pressure	5.1
Unfolding the scooter (automatically or manually)	2
Wiring harness	6.2

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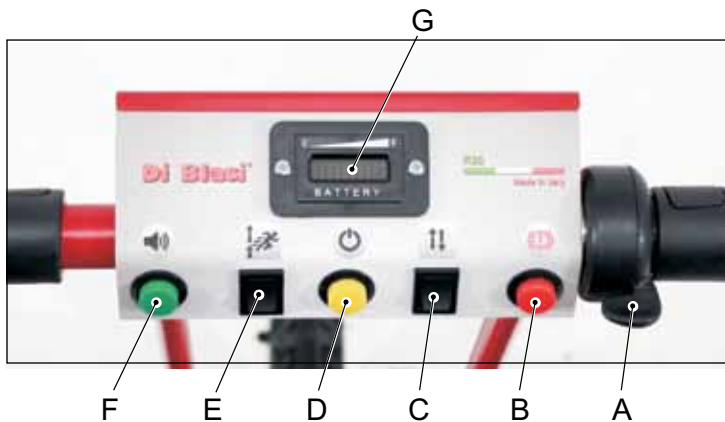
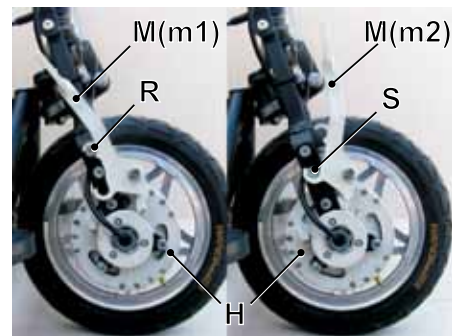
F.22

1. DEVICES FOR CONTROLLING THE SCOOTER

- **Handlebar devices (F.20)**
 - * (A) Speed control lever
 - * (B) Button for activating the emergency electric brake
 - * (C) Rocker device for running forward or rearward
 - * (D) Main switch button for activating or deactivating the scooter.
 - * (E) Rocker device for reducing the maximum speed
 - * (F) Horn button
 - * (G) Battery charge indicator
- **Devices for folding and unfolding the scooter (F.24)**
 - * (R) Red button for folding the scooter
 - * (V) Green button for unfolding the scooter
- **Device for disengaging the automatic brake (F.22)**
 - * (H) Mechanical automatic main brake
 - * (M) Lever for disengaging the automatic brake (H)
- **Device for disconnecting the automatic folding mechanism (F.26)**
 - * (A) Lever for disconnecting the automatic folding mechanism.

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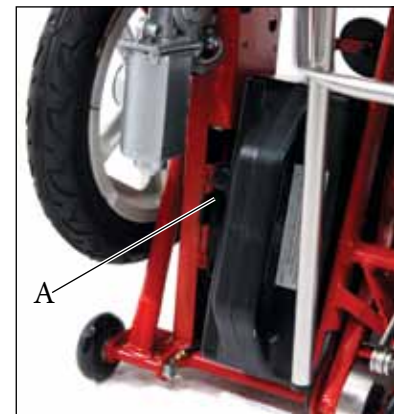
F.03

2. TO UNFOLD THE SCOOTER

- Check that the handle for dragging the scooter is fully lowered (F.02).
- Insert the battery and make sure that it remains blocked by the hook (A) (F.03)
- Press the green button on the left side of the seat until you hear a short tone (beep) indicating that the scooter is fully unfolded (F.05).
WARNING: Press the button keeping the fingers and the other hand away from the surrounding pinch area to prevent them from being pinched in the folding mechanism and from being injured.
- Check that the cable (V) (F.07) which operate the handlebar folding is placed in the centre of the guide (W) (F.07)

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F.26

To unfold manually the scooter

If the automatic folding and unfolding mechanism does not work properly, fold or unfold the scooter manually as follows (F.26):

- Disconnect the automatic mechanism by pushing the lever (A) upwards to the position (a1).
- Fold or unfold manually the scooter by pulling the back rest so to make the seat turning.

To connect again the automatic mechanism, move the lever (A) downwards to the position (a2). If necessary, to make easier this displacement, move a little bit the seat up and down.

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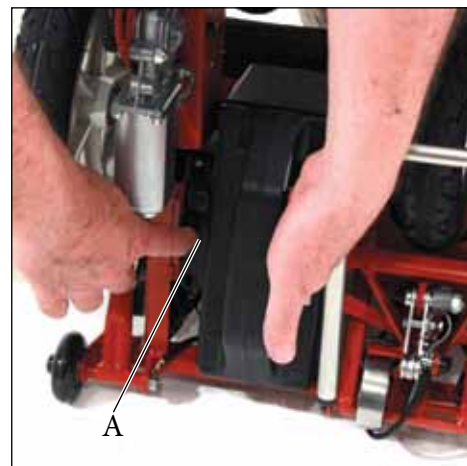
F.12

3. TO FOLD THE SCOOTER

- Check that the handlebar is straight
- Press the red button on the left side of the seat until that you hear a short tone (beep) indicating that the scooter is fully folded (F10).
WARNING: Press the button keeping the fingers and the other hand away from the surrounding pinch area to prevent them from being pinched in the folding mechanism and from being injured
- To remove the battery from the scooter, at first release its box from the hook (A) (F.12).
- To drag the scooter on the small service wheels, raise completely the rear handle (F.14).

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F.16

WARNING

- Once folded, the scooter can be lifted by hand only by a person or persons physically capable to lift its weight. To reduce this weight, remove the battery.
- The weight of the scooter and of the battery are specified at Sec 7 of this manual (“Specifications”)
- Ask to your physician which is to be the correct position of the body when lifting a weight
- Lift the scooter seizing the seat frame (F.16)

Important notices

- Do not press the folding red button is somebody is seated on the scooter.

To fold manually the scooter

- See Sec. 2 – Par. “To unfold manually the scooter”

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4. OPERATING INSTRUCTIONS

4.1 First riding

Before taking your first ride in an area open to pedestrian and motorized traffic, learn the feel of the scooter, become familiar with its handling, etc in an area free of pedestrian and motorized traffic. **In particular learn to use the emergency brake [see Sec 4.3 (h)]**

4.3 Riding the scooter

a) To sit on the scooter:

- Put the hands on the handlebar grips (F.30);
- Put one foot in the central part of the foot board (F.30);
- Sit down and place the feet in the sides of the foot board (F.32).

Important notice

Do not mount putting the foot on the side of the foot board because the scooter could overturn (F.34).

- b) Push the handlebar forward with a moderate force so to check that it is blocked in riding position. However a small play in the running direction is acceptable. Never drive the scooter if the handlebar is not blocked in riding position.

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
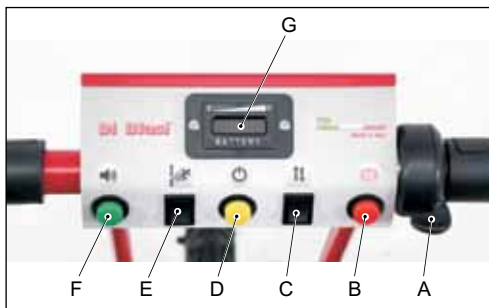
F.20

- c) Check if the rocker device (C) (F.20) is turned forward so that the scooter is set for running forward. In any case, an intermittent sound (beep) advises if the scooter is set for running rearward.
- d) Check that the lever (M) of the brake (H) (F.22) is blocked in the position (m1) by the hook (R).
The scooter can run only if the lever (M) is blocked in the position (m1)
- e) Press the button (D) (F.20) so to activate the scooter. The scooter is activated if any of the lights (LED) of the battery charge indicator (G) is lit.
If the lever (M) of the brake (H) (F.22) is not blocked in the position (m1), the scooter cannot be activated and three short tones (beep) advise that lever (M) is not in the right position.
- f) To start and to regulate the speed, push gradually downward the speed control lever (A) (F.20). To slow down or to stop the scooter just release the lever (A) (F.20).
- g) When pushing downward the speed control lever (A) (F.20), the brake (H) (F.22) is automatically released so that the drive motor can run.
When the speed control lever (A) (F.20) is released, the brake (H) (F.22) automatically engages and slowly stops the scooter.
The brake (H) works also as a parking brake.

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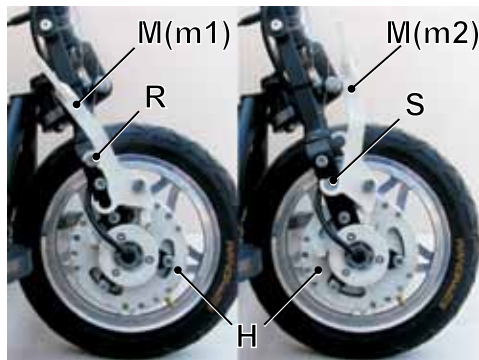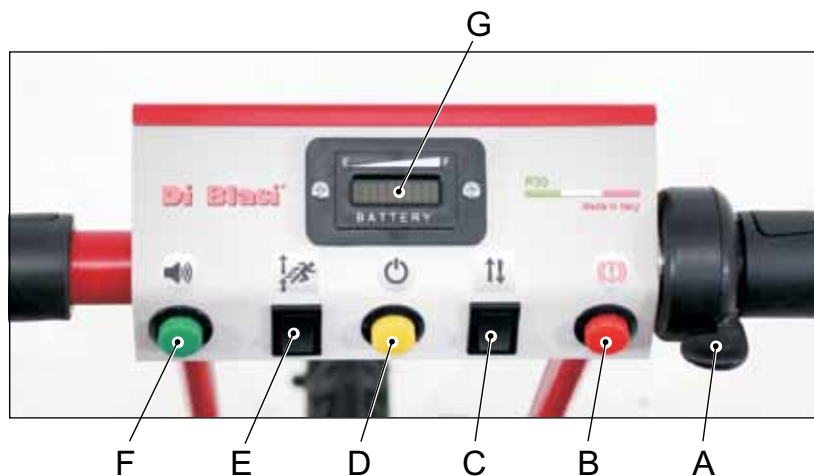
B

A

h) In case of failure of the automatic brake (H) (F.22) press the red button (B) (F.20) which activates an emergency electric brake.

- i) To run rearward, wait that the scooter is stopped and then turn the rocker device (C) (F.20) rearward. Then operate on speed control lever (A) in the same way than when going forward. An intermittent sound (beep) warns that the scooter is going rearward. When going rearward, drive even more prudently, limit the speed as much as possible, drive only for very small distances and never on descent.
- j) To reduce the maximum speed turn rearward the rocker device (E) (F.20).
- k) To activate the horn, push the button (F) (F.20).
- l) The front light automatically switches on when the environment lighting is poor

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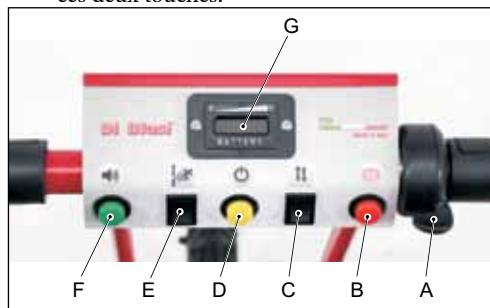
F.20

m) The level of charge of the battery is given by the position of the lit light (LED) in the charge indicator (G) (F.20): if the lit light is the tenth one, the battery is fully charged; if the lit light is the first one, the battery is empty. However this indication is quite approximate because it is based on standard discharging parameters which can be very different from the real discharging way. For example, after driving on slope, the indication of the charge is wrong: to restore a more precise indication press two times the button (D) (F.20). Therefore to estimate the real remaining range it is advisable to verify the instrument indication with the distance (km or miles) already covered since last charging but considering also the characteristics of the road covered (flat, on slope, etc).

n) Standby mode

- In order to save energy when in running configuration, all the functions are automatically deactivated when the scooter, is not activated for more than 2 minutes. In this case press button (D) (F.20) to reactivate the scooter.
- In order to save energy when the scooter is partially or completely folded, the functions are deactivated if neither the folding red button (R) (F.24) nor the unfolding green button (V) (F.24) are pressed for more than 10 seconds. In this case press any of the two above buttons to reactivate the scooter.

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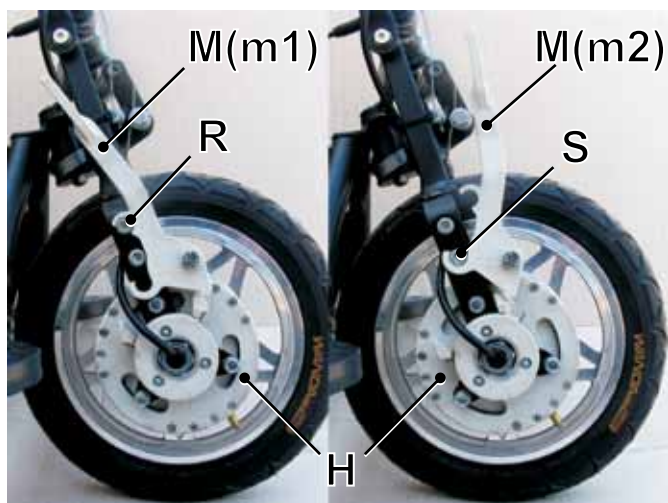
o) Freewheel mode

To push the scooter by hand, it is necessary to disengage the automatic brake (H) (F.22) by pushing forward the lever (M) until it remains locked in the position (m2) by the hook (S). Before driving again by motor, engage again the automatic brake by moving the lever (M) in the position (m1) where it has to remain locked by the hook (R).

WARNING

- If the automatic brake (H) (F.22) is not engaged, it is not possible to activate the drive motor : this is signalled by three short tones (beep).
- Never attempt to drive the scooter by motor if the automatic brake is not engaged.
- To prevent that the scooter can roll on its own uncontrollably, and therefore dangerously, never disconnect the automatic brake (H) (F.22) if the scooter is on an incline or decline.

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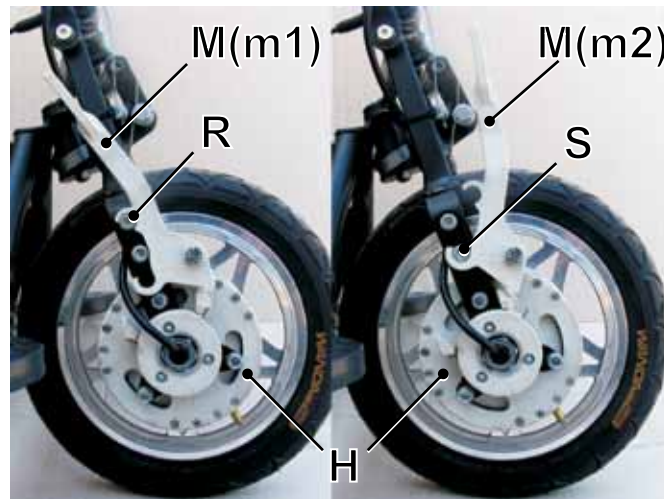
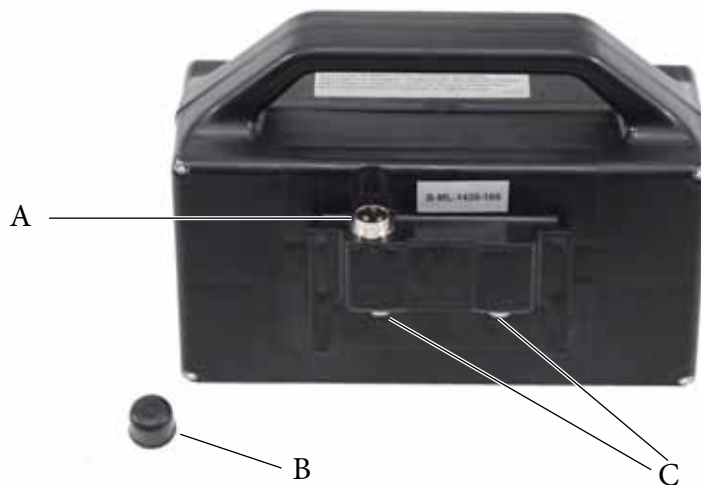
4.5 Battery & Charger

4.5.1 Description

- Battery box (F.40)
 - * (A) Charging plug
 - * (B) Protection cap of the plug
 - * (C) Terminals (+) and (-) of the battery
- Charger (F.42)
 - * (F) Socket for connection to the battery
 - * (G) Plug to the mains power supply
 - * (H) Lighted indicators (LED1 - LED2) showing the charging status

4.5.2 Charging of the battery

- The charger can be connected to a 110V or 220V mains supply without setting any switch.
- Remove the cap (B) from the plug (A) of the battery.
- Connect the charger plug (G) to a mains power socket: LED1 lights up red, LED2 lights up green.
- Connect the charger socket (F) to the plug (A) of the battery: LED2 lights up red.
The battery is fully charged when LED2 lights up green again.

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4.5.3 Cautions concerning the battery

- The battery is stored in the box shown in F.4.5.1 Never open this box and never tamper with the battery.
- Charge the battery only with the charger equipping the scooter.
- Charge or use the battery only if the ambient temperature is between +5°C and +40°C. Do not charge the battery immediately after that it has been used, but wait that it cools a little bit.
- Keep the battery dry: rain and humidity can damage the protection electronic circuits.
- Keep the battery far away from fires or sources of heat.
- Do not use the battery to operate equipments other than the scooter.
- Do not short circuit the battery. Short-circuiting occurs when a metallic object put in direct connection the (+) and (-) terminals (C) of the battery (F.40). A short-circuit can damage the battery and/or the connecting object.
- Never dispose of used batteries with domestic waste nor in a fire or in water. Batteries thrown in the fire can explode very dangerously. Bring used batteries at a authorised disposal point.
- If you foresee to not use the tricycle for long-time:
 - * Remove the battery from the scooter
 - * Store the battery in a cool and dry location
 - * **Charge the battery every two months at least**

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4.5.4 Cautions concerning the use of the charger

- Do not use the charger to charge batteries other than that fitted in the scooter.
- Do not cover the charger while it is charging because it could overheat.
- The charger is active as soon as the plug (G) is connected to the mains supply socket. Use a socket which is easily accessible so that the plug can be immediately disconnected in case of need.
- Disconnect the plug from the mains supply socket when the charger is not in use. Do not pull the plug from the cable because this could damage the plug.
- Do not use the charger if the plug (G) or the socket (F) or the cables are damaged or if it has suffered a violent shock (as consequence of a fall, for example).
- Never disassemble or tamper with the charger
- Dangerous voltages are present within the charger: therefore its cover must never be removed.
- Never put the charger in contact with products which could attack its case.
- Eventual repairs are to be carried out by qualified personnel (contact your retailer).

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• Carga máxima admisible: 100 kg

7. SPECIFICATIONS

- **Dimensions**
 - * Unfolded scooter (F.71)
 - Wheelbase: 55 cm (21,7 in)
 - Maximum width: 61,5 cm (24,2 in)
 - Track: 77,5 cm (30,5 in)
 - * Folded scooter (F.72)
 - Length: 61,5 cm (24,2 in)
 - Width: 38,5 cm (15,2 in)
 - Height: 49,5 cm (19,5 in)
 - * Seat (F.74 – F.75)
 - Width: 47 cm (18,5 in)
 - Depth: 40 cm (15,7 in)
 - Height from the footboard: 47cm (18,5 in)
 - Height from the ground: 61cm (24,0 in)
 - * Seat back rest (F.74 – F.75)
 - Height from the seat: 29 cm (11,4 in)
 - Height from the ground: 90 cm (35,4 in)
 - Distance from the handlebar: 66 cm (26 in)
 - * Ground frame minimum clearance: 12 cm (4,7 in) (F.75)
 - * Turning minimum radius: 71,5 cm (28,1 in) (F.76)
- **Masses**
 - * With battery: 26,5 kg (58,5 lbs)
 - * Without battery 23,6 kg (52,0 lbs)
- **Motor:**
 - * Rated output: 200 W
 - * Voltage: 24V
- **Battery**
 - * Type: LiFePO4
 - * Voltage: 24V
 - * Capacity: 9Ah
 - * Masse: 2,9 kg (6,4 lbs)
- **Wheels (front and rear)**
 - * Rim: 8"
 - * Tyres (pneumatic): 12 ½ x 2 ¼
- **Brakes**
 - * An automatic mechanical brake
 - * A manually operated electric brake
- **Maximum speed** : 6 km/h (3.7 mph)
- **Maximum safe slope**: 6° (10,5%)
- **Maximum obstacle height that can be climbed and descended**: 50 mm (2 in)
- **Range**: 24 km (15 Miles) approximately
- **Seating capacity**: 1
- **Max load**: 100 kg (220 lb)

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