Instructions for use Quick Clean™ Microwave bags

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Important safety information

The Quick Clean microwave bag is to be used for breast pump accessories, bottles, teats, pacifier and darts, by means of steam treatment. You can use the bag up to 20 times. The Quick Clean microwave bag is intended for use only with microwave safe plastics. Do not use the bag to clean plastic or grinder parts of the...

Cleaning and hygiene

Clean parts immediately after use to avoid the drying up of milk. Use a cleaning quality soap for any of the following cleaning procedures. Follow these cleaning instructions. Note: Steam treatment may cause a scaldation of the cleaned parts. This will not affect performance.

How to use

1. Do not use less than 60 ml of water. Using less water may scald the parts. 2. Do not overfill the bag. 3. Do not detangle the tubing in the microwave bag as this may lead to it becoming damaged.

One bag should contain approximately 150 ml of breast milk or 100 ml of milk powder.

Heating times

1. 100W+ = 1.5 minutes 1.400-1.600W = 3 minutes 1.500-750W = 5 minutes

Heating times

1. 100W+ = 1.5 minutes 1.400-1.600W = 3 minutes 1.500-750W = 5 minutes

Check box after each use

1. Carefully remove all parts from the bag. Dry each part with a clean cloth. Do not use the bag for a maximum of 20 times. To keep track of the number of uses, check the box after each use.

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