Instructions for use SpecialNeeds™ Feeder

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Mode d'emploi SpecialNeeds™ Tétine

Read all instructions before use

- Use the product only for its intended use as described in this instruction manual.
- Use by more than one infant without adequate reprocessing may present a health risk and could cause cross-contamination.
- Use Medela original accessories only.
- Inspect before each use. Throw away at the first signs of damage, mould or weakness.
- Never leave packaging and components unattended. Keep them out of reach of children.

SAVES THIS INSTRUCTION MANUAL.

Important safety information

- Plastic bottles and component parts become brittle when frozen and may break when dropped.
- Also, bottles and component parts may become damaged if mishandled, e.g. dropped, over-tightened, or knocked over.
- Take appropriate care in handling bottles and components.
- Do not use the liquid in bottles or components become damaged.

Cleaning

- Only use drinking-quality water for cleaning.
- Reprocess all parts prior to first use (see below).
- Clean all parts immediately after use.
- The cleaning instructions below are general instructions. For country-specific regulations and guidelines, ask your Medela agent or visit our website www.medela.com.

After each use

- Discard the feeder into its individual parts. Make sure to separate the yellow valve plate from the test. Also separate the white membrane from the valve plate.
- Note:** You can re-use the milk that is left in the bottle. However, you must throw away the milk left in the test. Therefore, hold the feeder above the sink. Carefully uncrow the lid and remove it together with the test and valve plate, so as not to spill milk. Discard the leftover milk from the test.
- Rinse all parts with warm water (approx. 20 °C). Clean all parts with plenty of water, soapy water (approx. 30 °C). Use a commercially available washing or liquid, preferably without artificial fragrances and colouring (for hospital use: 5–15% anionic tenside, nonionic tenside, pH neutral). Rinse all parts with cold water (approx. 20 °C).
- Dry with a clean cloth or leave to dry on a clean cloth.
- Alternatively, clean the product in the dishwasher. Place the parts on the top rack or in the cutlery section. Use a commercially available dishwasher detergent.

- Rinse all parts with warm water (approx. 20 °C). Clean all parts with plenty of water, soapy water (approx. 30 °C). Use a commercially available washing or liquid, preferably without artificial fragrances and colouring (for hospital use: 5–15% anionic tenside, nonionic tenside, pH neutral). Rinse all parts with cold water (approx. 20 °C).
- Dry with a clean cloth or leave to dry on a clean cloth.
- Alternatively, clean the product in the dishwasher. Place the parts on the top rack or in the cutlery section. Use a commercially available dishwasher detergent.

Home-use:

- Before first use and once a day**
- Clean the feeder as described under "After each use".
- Disinfect the feeder as described under "Disinfection".

Hospital use:

- Before first use – once a day – before passing on to another infant**
- Clean and disinfect the feeder as described under "Disinfection".
- If required by national, regional or in-house regulations, sterilise the feeder as described under "Sterilisation".

Disinfection

- Home-use:**
- Cover all parts with water and boil for five minutes.
- Or use a Quick Clean microwave bag in accordance with the instructions.
- Dry with a clean cloth or leave to dry on a clean cloth.

Hospital use:

- Clean and disinfect all parts in a mechanical washer-disinfector.
- Use appropriate inserts for small parts. Use a common cleaning detergent for washer-disinfectors (NTA-, phosphate-, chlorine- and tenside-free, alkaline with pH level 11.7–12.7).
- Disinfect for 5 minutes at a temperature of 62 °C.
- Select an appropriate program according to the above instructions that includes drying. If there is no program with drying, dry the clean parts with a clean cloth or leave to dry on a clean cloth.

Sterilisation

- Hospital use:**
- Check that the product parts are clean, dry and intact.
- Wrap the parts in an appropriate sterile packaging.
- Sterilise the parts in the autoclave at 134 °C for 5 minutes. To avoid damage, do not stack up against external pressure.

Storage

- Put the clean parts in a clean storage bag or a clean environment. It is important that all residual moisture dries. Do not store parts in an airtight container/bag.

How to use

- It is recommended that the mother performs her first feedings with the SpecialNeeds Feeder under the supervision of a professional/ trained staff.
- Always check the temperature of the liquid before feeding.

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© Medela AG