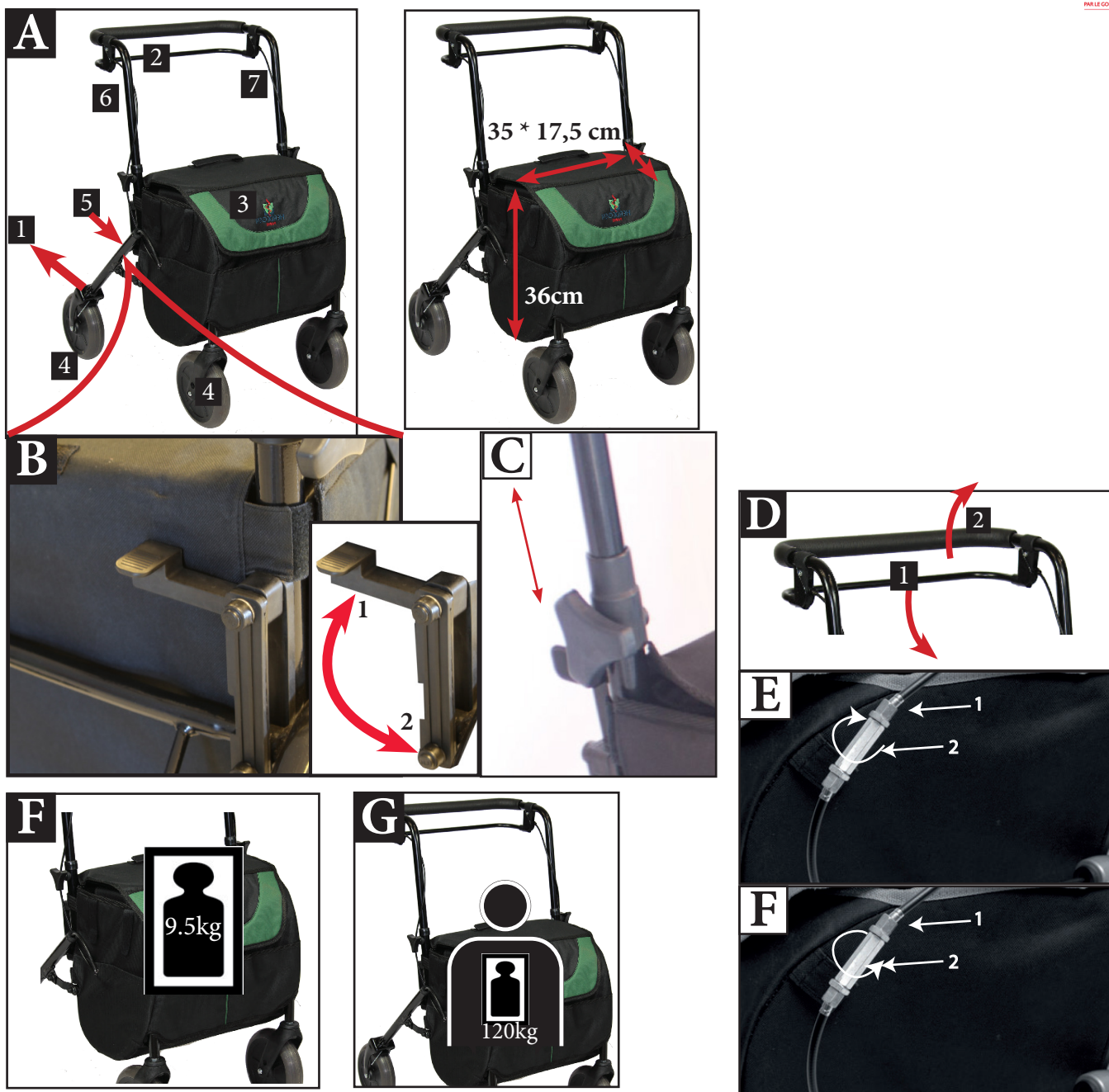
Designations									
243 880	90,5/94,5/98,5 cm	58 cm	23*34/38 cm	35 cm	57,5 cm	66 cm	20 cm	120 kg	9.5 kg

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Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indication

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0° or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips..)
- ♥ Check regularly if all screws are tightened.
- ♥ Hold firmly and completely the handle when using.
- ♥ In case of falling : let the product fall down on the side, so that you don't fall on it.
- ♥ Always move forward quietly and with short steps to ensure a good balance of your weight and to avoid losing your balance.
- ♥ Don't use on wet or icy surfaces. (snow, ice, water..) !!
- ♥ Don't use in stairs, severe risk of injuries!!
- ♥ Use only as a walking aid and in proper way.
- ♥ ALWAYS observe the weight limit on the labelling of your walker.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the walking aid, make sure that it is securely locked in OPEN position and level to the ground before using.
- ♥ DO NOT hang anything on the Walking aid.
- ♥ ALWAYS test to see that the walking aid and attachments are properly and securely locked in place before using.
- ♥ In case you didn't use your walking aid for a long time, let it checked by the technical staff , before using.
- ♥ Be careful when using: beware holes and deformations in the floor.

2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient.

The device must be adjusted individually by the technical staff to suit your needs.

Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

1-Place the rollator on a flat floor and pull back the rear legs to unfold the rollator [A] (1).

2-Lock the frame in open position by pushing down the plastic lever [B] from position (1) to position (2)

3-Adjust the height of the handle by using the screw [C]

4-Brakes : to brake, pull the top portion of the brake handle up towards the push handle [D] (2).

5-Parking brake : To lock, push down on the bottom portion of the brake handle until you hear a click [D] (1). To release the parking brake see [D] (2).

6-Adjusting the brake: to increase the resistance [E] to reduce the resistance [F]

Bag : the maximum weight capacity is 30kg. This bag isn't suitable to the transport of animals or human being. The bag can be used as a seat. Usage as a seat :respect the following informations :

- use only the chair on a flat and solid ground.
- put the brakes in the «parkingbrake» position.
- remain seating in a upright position
- always seat in the direction of travel of the rollator
- the backrest provides an additional safety and avoid falling backwards
- do not lean out of the chair to avoid losing balance.
- Beware : this seat must be used for a short period of time and it doesn't replace in any case a chair.

3. Cleaning

The rollator has to be cleaned regularly with a mild cleaning agent and a soft cloth.

NEVER use OILY substances!! Slip hazard!!

Ask your retailer or the technical staff to check the rollator regularly.

4. Materials

The product is made up of a steel frame with epoxy painting, one pushing bar with a brake bar, a shopping bag that can be used as a seat and 4 castors.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Test: EN-ISO 111 99-1

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