ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS

- Hallux Valgus
- Foot & toe post-surgery or trauma

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

APPLICATION INSTRUCTIONS

The Podalux II post-op shoe allows for adaptable rigidity. Should the post-op protocol require less rigidity, remove the rigid foot plate.

- 1) Open the contact closure straps.
- 2) To remove the foot-plate, take the inner sole out (fig 1), remove the foot plate (fig 2) then replace the inner sole (fig 3).
- 3) Place the foot in the shoe.
- 4) Close the contact closure straps starting with the distal strap, followed by the proximal strap (fig 4).
- 5) Test the shoe by walking a short distance and readjust the tightness of the closure straps if needed.

- If the contact closure straps are too short or not tight enough, use the removable fastening strap to secure shoe closure. Place the removable fastening strap on the upper part of the shoe and attach using the contact closure side strips.
- The shoe also includes a removable half-sock to protect the forefoot.
- Do not hook the closure straps on the half-sock.

CARE

Hand wash in cold water with mild soap, rinse thoroughly. Air dry. Do not machine dry or bleach.

COMPOSITION

- Outer shell: Polyester / Polyamide
- Podalux II: Polyurethane and Polyamide
- Half-sock: Acrylic and Elasthane

LATEX **LATEX FREE**
Not made with natural rubber latex

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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