AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear Customer,

Thank you for placing your trust in a Bauerfeind product. The OmoTrain you have purchased is a high-quality medical product.

Please read through all of these **instructions for use** carefully, because they contain important information about the use, indications, contraindications, application, care and disposal of your OmoTrain. Please keep these instructions safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

Purpose

OmoTrain is a shoulder support. It improves the function of the shoulder joint and counteracts painful movements. The massage pad, which can be individually positioned, produces a constant massage effect to improve blood circulation. This helps swelling and effusions reduce more quickly. If used consistently, OmoTrain can speed up the mobilization of the affected shoulder and noticeably relieve pain.

Indications

- Shoulder pain (due to injury / surgery)
- Irritation of the soft tissue in the shoulder area (chronic, due to injury / surgery)
- Shoulder joint wear (osteoarthritis of the shoulder)

⚠ Risks of using this product

OmoTrain must only be worn in accordance with the specifications contained in these instructions and used for the areas of application (purpose, places of use) listed.

OmoTrain is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her / him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product. To ensure the optimum fit of the OmoTrain, the size of OmoTrain required must be determined by a trained specialist¹.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.

No product liability is accepted in the event of improper use.

Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.

Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects that affect the entire body have not been reported to date. Correct fitting is assumed. Do not put the OmoTrain on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the strap of the OmoTrain and, if necessary, have the size of your OmoTrain checked.

 Contains natural rubber latex (elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present.

Likewise, any raised scars with swelling, redness and excessive heat build-up

- Impaired sensation and movement disorders of the arms (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support
- Vein occlusion in the arm (arm vein thrombosis)

Application instructions

Putting on the OmoTrain

A – Knitted section of the support, B – Velcro straps, C – Eyelet, D – Massage pad (Delta pad)

- 1 Fasten one the OmoTrain straps and put it out through the eyelet. To avoid damaging your clothes or the OmoTrain, fasten the ends of the strap loosely onto the strap underneath it.
- 2 Secure the knitted section onto the fastened straps. The opening should be pointing toward you.
- 3 Then guide the affected arm through the knitted section of the support until the shoulder is well covered. The massage pad (Delta pad) should be resting on the bony protuberance at front and back of the shoulder. If this is not the case, adjust the position of the support or the massage pad.
- 4 Arrange both straps in a flat (without twisting), diagonal position across the stomach.
- 5 Take hold of them by inserting your hand through the loop from the inside.
- 6 Pull the loop behind your head and slip your arm through it.
- 7 Guide the rear strap across your back toward the affected arm and secure it in place.
- 8 Then guide the front strap around your abdomen toward the affected side. Pull the strap that you unfastened to begin with back through the eyelet and fasten it.
- 9 Then tighten the abdominal strap until the required degree of stabilization is achieved. Fasten the end of the strap.

Taking off the OmoTrain

Undo both straps. To avoid damaging your clothes or the OmoTrain, fasten the ends of the straps loosely. Slip your arm out of the sling first, and then slip it out of the knitted section of the support.

Cleaning instructions

Note: Never expose the OmoTrain to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the OmoTrain.

The sewn-in label of the OmoTrain, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the inside of the knitted fabric.

⚠ All Train products can be machine washed separately at 30 °C with liquid detergent.² Please note the following when washing your Train products:

Close the Velcro fastenings to avoid any damage. Remove the massage pad before washing. Wash the support separately in a lingerie bag with a mild laundry detergent at 30 °C. Do not run the spin cycle on your washing machine at speeds higher than 500 rpm. Wash the OmoTrain regularly so that the knitted fabric retains its compressive properties. Wash the massage pad by hand if required.

Refasten the massage pad after washing. To do this, lay the OmoTrain with the straps facing upward on the long side of the knitted fabric. Fasten the massage pad onto the knitted section with the long tapered end pointing toward you.

Notes on reuse

The OmoTrain is intended for your treatment only and has been specifically selected according to your measurements. For this reason, do not pass the OmoTrain on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship

between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Version: 2017-12

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports which are relevant to you.
² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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