AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

MalleoTrain is a medical device. It is a support for the relief and stabilization of the ankle.

Indications

- Chronic, post-operative and post-traumatic irritation (e.g. after sprains)
- Osteoarthritis and arthritis, joint effusions and swelling
- Tendomyopathy
- Ligament insufficiency
- Early functional / non-surgical treatment
- Relapse prevention

Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the MalleoTrain. They will also inform you whether the MalleoTrain can be used simultaneously with other products, and if so, which ones.
- Please note all the information you received with the MalleoTrain as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and the warranty will not be valid.

- The effects of this support are felt primarily during physical activity. Take the support off during extended rest phases – e.g. sleeping and long periods of sitting down.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience any unusual changes connected to the use of the MalleoTrain – e.g. an increase in discomfort – stop using it and contact your physician without delay.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contra-indications

No side effects that affect the entire body have been reported to date.

If you have any of the following conditions, the product should only be used after consultation with your physician:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
2. Impaired sensation and circulatory disorders of the legs / feet (e.g. diabetes mellitus)
3. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

Fitting MalleoTrain

1. Fold the upper part of the support outwards until you can grip the heel. **1**
2. Pull the support over the foot up to the heel. **2**
3. Fold back the folded-over part of the support.
4. Then pull it up until the bones of the ankle rest in the rounded hollows of the pads. **3**
5. Adjust the fit of the support as necessary. **4**

Cleaning instructions

Please use mild detergent and a lingerie bag. Never expose the product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential

claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoTrain have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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