AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear customer,

Thank you for choosing a Bauerfeind product. Please read the **instructions for use** carefully and contact your physician or medical retailer if you have any questions.

Intended use

The MalleoLoc is an orthotic device which is anatomically contoured to fit the ankle joint. The MalleoLoc prevents sideways tilting, and provides effective support for muscular stabilization. MalleoLoc can be adjusted individually using the adjustable velcro straps. MalleoLoc can be worn comfortably in regular shoes and sports shoes.

Indications

- Early functional treatment for injuries of the lateral malleolar ligaments
- Postoperative protection after ligamental suturing / reconstruction
- Conservative treatment for severe ankle joint distortion and ruptured ligaments
- Chronic ligament insufficiency

Application risks

- The use and duration of use must be discussed with the physician who prescribed the MalleoLoc. He will also inform you whether the MalleoLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the

MalleoLoc as well as the instructions for use.

- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the MalleoLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.

Contraindications

Side effects involving the body as a whole have not been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up
- Impaired sensation and circulatory disorder of the legs or feet, e.g. in diabetes mellitus
- Impaired lymph drainage, including soft tissue swelling of uncertain origin located some distance away from the orthosis.

Application instructions

Putting on / taking off the MalleoLoc

Sit down to put on the MalleoLoc.

-  Open the two velcro straps.
 -  Push the loop of the lower strap down and out of the holder, to loosen the strap.
 -  Turn the MalleoLoc so that the straps point inwards. The projection on the lower end of the shell points forwards.
 -  Now place the foot centrally in the shell of the MalleoLoc. The inner part of the stay will be located behind the centre of the ankle and the outer part will be in front of the centre of the ankle.
 -  Close the upper strap by placing it around the ankle joint from the back. When applying the strap, make sure that it sits without folds.
- The lower strap is now placed around the foot and lower leg in the shape of a figure of eight. To do this, proceed as follows:
-  Place the strap diagonally over the back of the foot, towards the back, around the lower leg.
 -  Now guide the strap diagonally forwards once more, to the outer side of the foot, and
 -  push the loop from the bottom to the top, into the holder.
 -  Fasten the end of the strap upwards onto the part of the strap lying underneath.

Cleaning instructions

Note: Never expose the MalleoLoc to direct heat (e.g. heaters, sunlight, in your car etc.) This may damage the material.

This can impair the effectiveness of the MalleoLoc.

For cleaning, remove the straps from the MalleoLoc.

To avoid damage to the product, wipe the shell of the MalleoLoc with a mild detergent only.

Fasten the Velcro fastening. Wash the straps by hand.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Permissible conditions and places of use

In accordance with the indications (ankle joint).

Assembly and fitting instructions

The MalleoLoc is supplied ready for use.

Assembling the MalleoLoc after cleaning:

Fitting the loop:

 Hold the loop with the tip pointing downwards and the Bauerfeind logo "B" forwards between thumb and forefinger. Now guide the strap with the light gray broad side through the loop from the front.

Fitting the straps:

Place the long strap that you have just fitted with the loop in front of you, with the dark gray side upwards.  Guide the strap with the narrow side from the outside inwards, through the lower (closed) part of the MalleoLoc shell.

 Now guide the short strap with the dark gray side pointing upwards from the inside outwards, through the open part of the MalleoLoc shell.

Technical specifications / parameters

The MalleoLoc is an orthotic support for the ankle with two straps attached to it. The straps are removable. In addition, a loop for fastening is fitted to the lower (longer) strap. The two straps and the loop are available as spare parts.

Notes on reuse

The MalleoLoc is intended for your treatment only. For this reason, do not pass the MalleoLoc on to other people.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the product in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make

any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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