



 **BAUERFEIND AG**
Triebeser Straße 16
07937 Zeulenroda-Triebes
GERMANY
P +49 (0) 36628-66-4000
F +49 (0) 36628-66-4499
E info@bauerfeind.com

ValguLoc®

Hallux-Valgus-Orthese
Hallux Valgus orthosis
Orthèse Hallux Valgus
Ortesis Hallux Valgus

 **MADE
IN
GERMANY**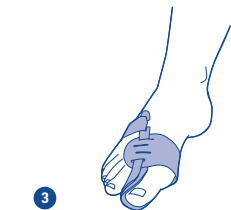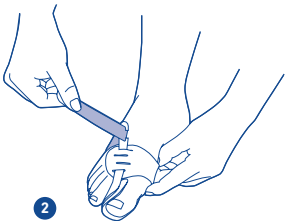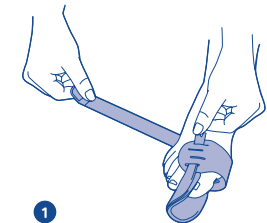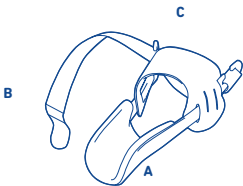

| | | | | | | | |
|------|------------|-------|----|------|------------|-------|----|
| (de) | deutsch | | 3 | (sr) | srpski | | 43 |
| (en) | english | | 6 | (sl) | slovenski | | 45 |
| (fr) | français | | 8 | (ro) | romanian | | 48 |
| (nl) | nederlands | | 11 | (el) | ελληνικά | | 50 |
| (it) | italiano | | 13 | (tr) | türkçe | | 53 |
| (es) | español | | 16 | (ru) | русский | | 56 |
| (pt) | português | | 18 | (ei) | eesti | | 59 |
| (sv) | svenska | | 21 | (lv) | latviešu | | 61 |
| (no) | norsk | | 23 | (li) | lietuvių | | 63 |
| (fi) | suomi | | 26 | (uk) | українська | | 65 |
| (da) | dansk | | 28 | (he) | עברית | | 69 |
| (pl) | polski | | 30 | (ja) | 日本語 | | 71 |
| (cs) | česky | | 33 | (ko) | 한국어 | | 73 |
| (sk) | slovensky | | 36 | (ar) | عربي | | 76 |
| (hu) | magyar | | 38 | (zh) | 简体中文 | | 78 |
| (hr) | hrvatski | | 40 | | | | |

(de) deutsch

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Dear customer,

Thank you for placing your trust in a Bauerfeind product. By choosing the ValguLoc, you have purchased a high-quality medical product.

Please read through all of these instructions for use carefully, as they contain important information about the use, indications, contraindications, application, care, and disposal of the ValguLoc. Please keep these instructions in case you need to refer to them again in future. If you have any questions, please contact your physician or medical retailer.

Intended use

The ValguLoc is an ergonomically preformed orthosis¹ for treating deformities of the big toe (hallux valgus). It is worn without shoes, usually at night when the foot is not under strain. The ValguLoc helps correct big toe misalignments. Long-term use stretches the soft tissue and joint capsule and the toe is straightened up by applying small degrees of force.

⚠ The ValguLoc must be individually adapted for the patient by a trained orthotist. This is the only way to guarantee its full effect and optimal wearing comfort.

Indications

- Hallux valgus (bending of the big toe toward the little toe) – conservative and post-operative treatment.

Risks of using this product

⚠ Caution²

The ValguLoc must be worn in accordance with these instructions and in the specified areas (where it fits the orthosis).

To guarantee the best possible fit of the ValguLoc, it is essential to check whether the product is adequately

shaped, or whether it needs to be reshaped by a specialist³ to fit the body. Only a trained specialist may carry out the (initial) fitting of the orthosis and provide instruction.

The ValguLoc is a night splint. It is not suitable for use while walking.

Consult your physician before combining with other products e.g. as part of compression therapy (compression stockings).

If you notice any unusual changes or your symptoms get worse, please contact your physician without delay. No product liability is accepted in the event of improper use.

Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.

No side effects that affect the entire body have been reported to date. This assumes the orthosis is used/fitted correctly.

Do not fit the ValguLoc too tightly, as this could lead to local pressure points. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the strap on the ValguLoc and have the size of your ValguLoc checked if necessary.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders/injuries in the part of the body being treated, particularly if inflammation occurs. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
- Impaired sensation and circulatory disorders in the leg/foot, e.g. diabetes mellitus

- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn.
- Gout (arthritis urica)

Application instructions

Putting on/applying the ValguLoc

A - Plastic brace, B - Strap with Velcro fastening, C - Ball tab

① Open the Velcro fastening on the ValguLoc strap. ② Push the plastic brace between the big toe and the second toe until the big toe is lying in the hollow of the ValguLoc. ③ Now feed the strap through the opening in the brace and secure it in place tightly with the Velcro fastening. In doing so, make sure that the position of the big toes looks correct without causing pain.

Taking off/removing the ValguLoc[®]

Open the Velcro on the strap and move the foot downward out of the ValguLoc.

Cleaning instructions

Note: Never expose the ValguLoc to direct heat (e. g. heaters, sunlight, leaving in your car, etc.) This may damage the material, which can impair the effectiveness of the ValguLoc.

Close all Velcro straps before cleaning to avoid damage. Clean the ValguLoc separately by hand with a mild detergent.

If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Where to apply the orthosis

In accordance with the indications (big toe). Please see intended use.

Maintenance instructions

If handled and cared for correctly, the product requires virtually no maintenance.

Assembly and fitting instructions

The ValguLoc is delivered in an anatomically (ergonomically) preshaped state. If necessary, the ValguLoc can be shaped further. The orthoses come in two versions for the right and left big toe.

Technical details / parameters, accessories

The ValguLoc is an orthosis for the big toe.

Replacement straps are available.

Notes on reuse

The ValguLoc is intended for your treatment only. It has been adjusted specifically for you. Never pass on the ValguLoc to other people.

Warranty

The statutory regulations of the country of purchase apply. Where relevant, the country-specific warranty regulations between the buyer and retailer apply. Please contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty.

Please do not make any alterations to the product yourself. This particularly applies to products that have been individually adjusted by a trained specialist. Please follow our instructions for use and care. They are based on years of experience and ensure that our medically effective products continue to work for a long time. After all, only aids that function perfectly will help you. Failure to follow the instructions can also limit the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-02

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

² Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

³ A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

Specialists

General instructions

The first time it is used, check that the ValguLoc fits the individual patient optimally. Make any necessary corrections.

Practice the correct procedure for putting on the orthosis with the patient.

No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.

The ValguLoc is intended for treating one patient only.

The ValguLoc is anatomically (ergonomically) reshaped, but may still need to be individually adjusted to fit the patient's body. To do this, apply a little force to form the orthosis into the anatomically correct shape.

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Bauerfeind Middle East FZ-LLC
Bauerfeind USA, Inc.