**BAUERFEIND AG**  
 Triebeser Straße 16  
 07937 Zeulenroda-Triebes  
 Germany  
 P +49 (0) 36628 66-40 00  
 F +49 (0) 36628 66-44 99  
 E info@bauerfeind.com  
**BAUERFEIND.COM**

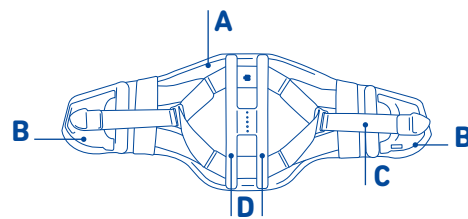
## Spinova®-Wirbelsäulenorthesen

- Spinova® Stabi Classic
- Spinova® Unload Classic
- Spinova® Support Plus Classic
- Spinova® Immo Plus Classic


Material	
<b>Grundbandage, Bauchverschluss</b>	<b>Überbrückungsrahmen Spinova Unload Classic</b>
Aluminium (AlMgSi)	Polyethylenterephthalat (PET)
Polyamid (PA)	AlMgSi (/)
Polyurethan (PUR)	Ethylen-Vinylacetat (Kunststoff) (EVA)
Elastan (EL)	Polyamid (PA)
Polyester (PES)	Polyurethan (PUR)
Thermoplastisches Polyurethan (TPU)	
Polypropylen (PP)	<b>Überbrückungspelotte Spinova Support Plus Classic</b>
<b>Schale Spinova Immo Plus Classic</b>	Polyamid (PA)
Polyethylen, niedrige Dichte (PE- LD)	Glasfaser (GF)
Edelstahl (/)	Polyester (PES)
Polyamid (PA)	Polyurethan (PUR)
Polyurethan (PUR)	Polypropylen (PP)
<b>Schalenbezug Spinova Support Plus Classic</b>	<b>Abdominalpelotte Spinova Immo Plus Classic</b>
Polyamid (PA)	Polyamid (PA)
Elastan (EL)	Elastane (EL)
Viskose (CV)	Polyethylen (PE)
Elastodien (ED)	Ethylen-Propylen-Dien-Kautschuk (EPDM)
Polypropylen (PP)	Polyurethan (PUR)
Polyurethan (PUR)	Polyamid (PA)
	Baumwolle (CO)
	Polyester (PES)
	Elastan (EL)

Material	
<b>Base corset, Front closure</b>	<b>Bridging frame – Spinova Unload Classic</b>
Aluminium (AlMgSi)	Polyethylene
Polyamide (PA)	Terephthalate (PET)
Polyurethane (PUR)	AlMgSi (/)
Elastane (EL)	Ethylene Vinyl Acetate (EVA)
Polyester (PES)	Polyamide (PA)
Thermoplastic Polyurethane (TPU)	Polyurethane (PUR)
Polypropylene (PP)	<b>Back pad Spinova Support Plus Classic</b>
<b>Shell Spinova Immo Plus Classic</b>	Polyamid (PA)
Polyethylene, low density (PE- LD)	Glass fiber (GF)
High-Grade Steel	Polyester (PES)
Polyamide (PA)	Polyurethane (PUR)
Polyurethane (PUR)	Polypropylene (PP)
<b>Shell cover Spinova Support Plus Classic</b>	<b>Abdominal pad Spinova Immo Plus Classic</b>
Polyamid (PA)	Polyethylene (PE)
Elastane (EL)	Elastodiene (ED)
Viscose (CV)	Polypropylene (PP)
Elastodiene (ED)	Polyurethane (PUR)
Polypropylene (PP)	Polyamide (PA)
Polyurethane (PUR)	Cotton (CO)
	Polyester (PES)
	Elastane (EL)

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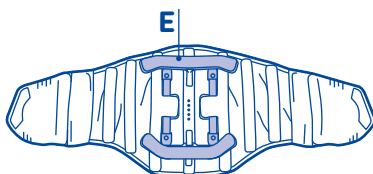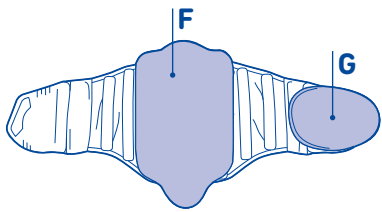
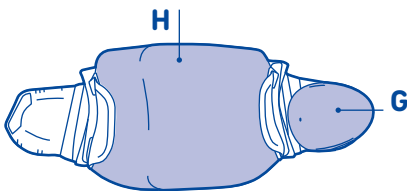
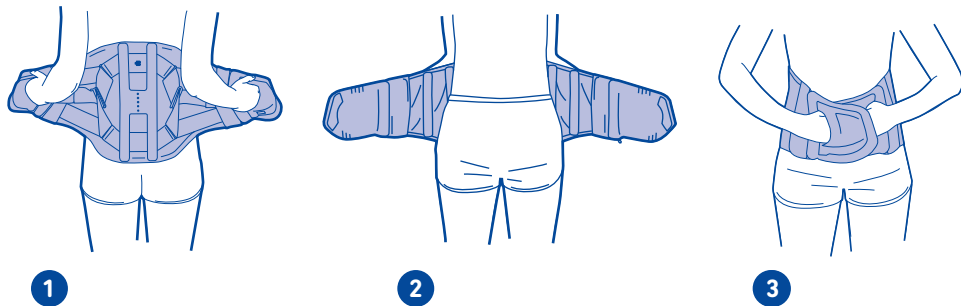
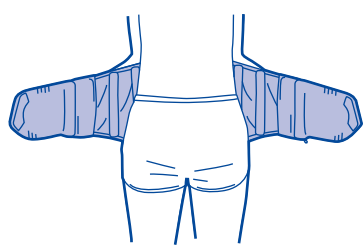
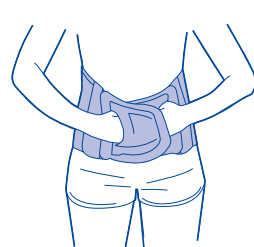
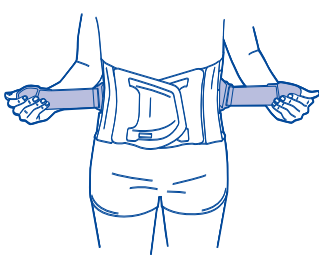
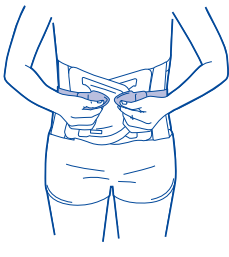
- Spinova Stabi Classic
- Spinova Unload Classic
- Spinova Support Plus Classic
- Spinova Immo Plus Classic

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en english

**These instructions for use can also be downloaded from:**

**[www.bauerfeind.com/downloads](http://www.bauerfeind.com/downloads)**

## Dear Customer,

Thank you for choosing a product from Bauerfeind. The SpinoVA spine orthosis you have purchased is a high-quality medical product.

Please read through all of these **instructions for use** carefully, because they contain important information about the use, indications, contraindications, application, care, and disposal of the product. Please keep these instructions for use safe, as you may want to read them again later. If you have any questions, please contact your physician or medical retailer.

The following information relates to these products:

### SpinoVA Stabi Classic

### SpinoVA Unload Classic

### SpinoVA Support Plus Classic

### SpinoVA Immo Plus Classic

Please also read the specific instructions relating to your product on the following pages.

## Risks of using this product

### **⚠ Caution\***

Your SpinoVA spine orthosis must only be worn in accordance with the specifications contained in these instructions for use and used for the areas of application listed (parts of the body this product is used for). It is a prescribed product that should be applied under a physician's guidance. To ensure the best possible fit, the orthosis must be individually adjusted by a trained specialist<sup>1</sup>. Please refer to the care instructions on the sewn-in label. The sewn-in label contains information on the size, manufacturer, care instructions, and CE marking and is located on the inside of the base support.

Use in combination with other products, e.g. as part of compression treatment (compression body support, lymph compression stockings), must be discussed beforehand with your physician. Should you notice any unusual changes or increased symptoms, please contact your physician without delay. No product liability is accepted in the event of improper use. Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids. No side effects that affect the entire body have been reported to date. Correct usage / fitting is assumed. Do not put the orthosis on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. After surgery, make sure that the wound is regularly checked by a physician.

**Note: Never expose the product to direct heat (e.g. heaters, sunlight, in your car, etc.)! This may cause damage to the material, which can impair the effectiveness of the orthosis.**

**⚠ SpinoVA Immo Plus Classic:** Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

## Application instructions

### Putting on the orthosis

The SpinoVA spine orthosis must be adapted by a trained specialist<sup>1</sup> before the first fitting.

**1** Position the orthosis centrally, at waist height, so that the lower back is covered. The stamped arrows on the inside of the abdominal fastening should be pointing upward. **2** Slide your fingers from the side into the finger pockets provided on the fastening and pull the pockets evenly to the side. Now pull the fastenings forward. **3** Lift the abdomen slightly from below with the left half of the fastening and position the right half of the fastening on top of the left half so that it can be fastened to it. Now take hold of the tensioning straps with both hands. **4** Pull these simultaneously and evenly, to the side at first, until the desired degree of stabilization is achieved. **5** Fasten the tensioning straps to the abdominal fastening.

### Taking off the orthosis

To take off the orthosis, start by undoing the tensioning straps. Then reach into the pockets of the abdominal fastening and unfasten the closure. After removing the orthosis, refasten the abdominal fastening to avoid damage.

## Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician: Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up, severe restriction of the performance of the heart and lungs (risk of a rise in blood pressure if the aid is worn during intense physical exertion), mental disabilities that render it impossible to use the orthosis safely.

## Cleaning instructions

If necessary, you can wash the orthosis by hand with a mild cleaning agent. Leave the orthosis to air dry.

## Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

## Notes on reuse

The SpinoVA spine orthosis is intended for your treatment only. It has been adjusted specifically for you. For this reason, do not pass the product on to other people.

## Warranty

The statutory regulations of the country of purchase apply. Where relevant, the country-specific warranty regulations between the buyer and retailer apply. Please contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty.

Please do not make any alterations to the product yourself. This particularly applies to products that have been individually adjusted by a trained specialist. Please follow our instructions for use and care. Failure to follow these instructions can limit the warranty.

## Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-10

## SpinoVA Stabi Classic

### Technical specifications / parameters

The SpinoVA Stabi Classic is an orthosis for the lower back. It consists of a base support with aluminum and corset stays and two tensioning straps.

### Intended use / part of the body this product is used for

The SpinoVA Stabi Classic is an orthosis that provides partial relief of the lumbar spine by reducing lordosis. By stabilizing the back, the SpinoVA Stabi Classic helps relieve pain.

## Indications

- Pain in the lumbar spine area with slight deficit symptoms in the case of bulging / herniation of the intervertebral disks (moderate lumbosciatica with slight deficiencies in the case of intervertebral disk protrusions / prolapse)
- Radiating pain in the lumbar spine area (moderate [pseudo] radicular lumbar syndrome)
- Vertebral displacement, formation of gaps in the vertebral joints (spondylolisthesis, grade I with lumbago)
- Following intervertebral disk surgery (condition after intervertebral disk surgery with small to medium prolapse)

## Components

A – Base support

B – Abdominal fastening with finger pockets

C – Tensioning straps

D – Aluminum stays

### Assembly and fitting instructions

The SpinoVA Stabi Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays and the tensioning straps must be individually adjusted by a trained specialist.

## SpinoVA Unload Classic

### Technical specifications / parameters

The SpinoVA Unload Classic is a orthosis for the lower back. It is made up of a base support with aluminum and corset stays, an abdominal fastening, two tensioning straps, and a bridging frame.

### Intended use / part of the body this product is used for

The SpinoVA Unload Classic is a orthosis that relieves the lumbar spine by straightening it. The bridging frame distributes (bridges) forces out toward the thoracic spine, pelvis, and sacral bone. Lordosis ("hollow back") is reduced. The bridging frame can be inserted or removed, depending on the progress of treatment, in consultation with your physician.

## Indications

- Vertebral displacement, formation of gaps in the vertebral joints (spondylolysis / spondylolisthesis, grades I and II)
- Wearing of the vertebral joints with unusual flexibility of the joints (facet syndrome with hypermobility)

- Radiating pain in the lumbar spine area (very severe [pseudo] radicular lumbar syndrome / very severe lumbar sciatica)
- Narrowing of the spinal canal with symptoms of numbness (lumbar spinal canal stenosis with paresis [conservative, post-operative])
- Narrowing of the side nerve root canal (foraminal stenosis, lateral)
- Vertebral fractures (fractures of the lumbar spine, relieving the rear edges of vertebrae)

#### Components

A – Base support
B – Abdominal fastening with finger pockets
C – Tensioning straps
D – Aluminum stays
E – Bridging frame

#### Assembly and fitting

The Spinova Unload Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays, the bridging frame, and the tensioning straps must be individually adjusted by a trained specialist.

#### Cleaning instructions

If necessary, you can wash the Spinova Unload Classic by hand with a mild cleaning agent. Unfasten the bridging frame **E** before washing. Refasten the bridging frame after drying. To do this, place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the bridging frame firmly around the struts of the frame using the Velcro tabs (the arrows on the bars should be pointing upward). Secure both bars on the bridging frame. Push one Velcro strip through the eye of the bar from the front **III**.

## Spinova Support Plus Classic

#### Intended use / part of the body this product is used for

The Spinova Support Plus Classic is a orthosis (modified Hohmann brace) designed to relieve and straighten the lumbar spine. The large back pad distributes (bridges) forces out toward the thoracic spine, pelvis, and sacral bone. It also relieves the lumbar spine with moderate correction of lordosis. The abdominal pad exerts compression on the abdomen, helping to straighten and relieve the spine. Depending on the course of treatment, the spine can be mobilized by removing the back pad. See **“Cleaning instructions”** for information on how to remove and attach the pads.

#### Technical specifications / parameters

The Spinova Support Plus Classic is a orthosis for the lower back. It is made up of a base support with aluminum and corset stays, an abdominal fastening, two tensioning straps, a back pad, and an abdominal pad.

#### Indications

- Vertebral displacement, formation of gaps in the vertebral joints (spondylolysis / spondylolisthesis, grade II)
- Wearing of the vertebral joints (facet syndrome)
- Radiating pain in the lumbar spine area (moderate [pseudo] radicular lumbar syndrome / lumbar sciatica, moderate)
- Narrowing of the spinal canal (lumbar spinal canal stenosis [conservative, post-operative])
- Vertebral fractures of the lumbar spine

#### Components

A – Base support
B – Abdominal fastening with finger pockets
C – Tensioning straps
D – Aluminum stays
F – Large back pad
G – Abdominal pad

#### Assembly and fitting instructions

The Spinova Support Plus Classic is supplied as a pre-assembled orthosis in standard sizes. The aluminum and corset stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist. The bridging pad and abdominal pad must be fastened to the base support.

#### Cleaning instructions

If necessary, you can wash the Spinova Support Plus Classic by hand with a mild cleaning agent. Unfasten both pads **F/G** before washing. Refasten both pads after drying. To do this, place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Fasten the back pad firmly onto the center of the support with the tapered end pointing downward. Now attach the abdominal pad to the right abdominal fastening.

## Spinova Immo Plus Classic

#### Technical specifications / parameters

The Spinova Immo Plus Classic is a removable orthosis for the lower back. It is made up of a base support with stays, an abdominal fastening, two tensioning straps, a shell, and an abdominal pad.

#### Intended use / part of the body this product is used for

The Spinova Immo Plus Classic is a orthosis for stabilization of the lumbar spine from the thoracolumbar to the lumbosacral transition. Its special design permits controlled treatment according to an individually adaptable multi-stage regimen.

##### Stage 1 (Stabilization phase):

To secure the spine after surgery or at the start of non-surgical treatment, use the orthosis in combination with the shell.

##### Stage 2 (Mobilization phase):

During the initial stages of mobilization with a progressive recovery process, the shell is removed and the base support is reinforced with the aluminum and corset stays in the affected lumbar / sacral area.

**Please ask your physician when you can remove the shell.**

#### Indications

- Vertebral displacement, formation of gaps in the vertebral joints (spondylolisthesis, grades II and III / spondylolysis)
- Wearing of the vertebral joints with unusual flexibility of the joints (facet syndrome with hypermobility / spondylitis)
- Chronic pathological changes (osteoarthritis) to the vertebral joints (spondylarthrosis)
- Radiating pain in the lumbar spine area (very severe [pseudo] radicular lumbar syndrome / very severe lumbar sciatica)
- Narrowing of the spinal canal (lumbar spinal canal stenosis with paresis [conservative, post-operative])
- Narrowing of the nerve root canal (foraminal stenosis, lateral)

#### Components

A – Base support
B – Abdominal fastening with finger pockets
C – Tensioning straps
D – Aluminum stays
G – Abdominal pad
H – Shell with fabric top cover

#### Assembly and fitting instructions

The Spinova Immo Plus Classic is supplied as a pre-assembled orthosis in standard sizes. The shell, the stays, the abdominal fastening, and the tensioning straps must be individually adjusted by a trained specialist. The abdominal pad must be fastened to the base support.

**V** To detach the fabric top cover, slip both the rubber bands on the base support over the front edges of the shell. Then remove the fabric top cover. To detach the shell from the orthosis, undo the middle screw in the back area. Remove the shell from the base support, then you can use the base support without it.

#### Cleaning instructions

Wipe the shell with a soft cloth and a mild cleaning agent. Then leave the orthosis to air dry. When using the Spinova Immo Plus Classic without the shell (mobilization phase), the orthosis can be washed by hand if necessary.

## Specialists<sup>1</sup>

The following information relates to these products:

#### Spinova Stabi Classic

#### Spinova Unload Classic

#### Spinova Support Plus Classic

#### Spinova Immo Plus Classic

Please also read the specific instructions relating to each product.

#### General instructions

**1–5** Adapt the orthosis according to the areas of application (indications) and the anatomy of the patient. Check the optimal and individual fit of the orthosis after the first fitting. Practice the correct procedure for putting on the orthosis with the patient. No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby canceling any product liability. The Spinova spine orthosis is intended for the treatment of one patient only.

Position the orthosis centrally, at waist height, so that the lower back is covered. The stamped arrows on the inside of the abdominal fastening should be pointing upward. To ensure optimal effectiveness, the orthosis should be positioned as low as possible without pressing against the groin area while the patient is sitting upright.

#### Fitting / assembly

##### Fitting the stays (dorsal aluminum stays and corset stays)

To ensure that the orthosis fits optimally without pressure points, the stays can be adjusted to suit the patient’s anatomy **D**. Take the stays out of the stay pockets on the base support and shape them to the patient’s individual body shape. Reinsert the stays into the stay pockets.

##### Shortening the tensioning straps **II**

First loosen the Velcro ends of the tensioning straps and attach them onto the middle of the fastening. Either you or your patient must then take hold of the tensioning straps in both hands and pull them forward until the desired level of tension is achieved. Fasten the tensioning straps with the loosened Velcro ends at the optimum length. Shorten the strap so that the upper part of the Velcro fastening can easily be closed over the end of the strap. Then attach the Velcro ends onto the strap again.

##### Adapting the abdominal fastenings to the patient’s body shape

You can adjust the angle and width of the abdominal fastenings to suit individual requirements. Detach the two abdominal fastenings from the base support and reposition them on the base support. Any protruding velor fabric on the base support can be cut off with fabric scissors once the product has been fitted. Protruding Velcro must be covered with a velor strip to avoid damage. In principle, you can also cut off the outer Velcro strip completely if it is not required. **II**

## Spinova Stabi Classic

#### Fitting / assembly

**Adjust the stays, tensioning straps, and abdominal fastening as described above.**

## Spinova Unload Classic

#### Fitting / assembly

**Adjust the stays, tensioning straps, and abdominal fastening as described above.**

#### Adjusting the bridging frame

To adjust the bridging frame, unfasten it from the base support. Remove the two Velcro strips from the bridging frame and open the two Velcro tabs on the base support. The aluminum stays on the bridging frame can be individually shaped. Then reattach the bridging frame, positioning it according to the patient’s body shape so that its delordosing function works to optimum effect. The arrows on the bars of the frame and the inside of the abdominal fastenings should be pointing upward. Please note that the Spinova Unload Classic is available in two heights **III**.

## Spinova Support Plus Classic

#### Fitting / assembly

**Adjust the stays, tensioning straps, and abdominal fastening as described above.**

##### Positioning the pads

Place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward **IV**.

Fasten the large back pad firmly onto the center of the support with the tapered end pointing downward. Now attach the abdominal pad to the right abdominal fastening. The upper edge of the back pad should sit approximately 3 cm below the shoulder blades. Please note that the Spinova Support Plus Classic is available in two heights.

## Spinova Immo Plus Classic

#### Fitting / assembly

**Adjust the stays, tensioning straps, and abdominal fastening as described above.**

##### Tools required

2,5 mm Allen key (available as an accessory)

##### Removing the shell from the orthosis

**V** Undo the middle screw on the back. Slip both the rubber bands on the base support over the front edges of the shell. Remove the base support from the back shell. If necessary, remove the fabric top cover from the shell.

##### Setting the height, width, and angle of the shell

After removing the shell from the base support, loosen the nut in the center of the shell by hand and undo the upper and lower screws with an Allen key so that both halves of the shell can be moved easily toward or away from each other. Position the shell on the patient’s torso so that the recesses of the pelvic frame sit directly above the iliac crests. Shift and bend the two halves of the shell until it is as flat as possible against the patient’s torso. Then tighten the three screw connections up again, making sure that they are aligned as

symmetrically as possible **VI**. The upper edge should end approximately 3 cm below the shoulder blades. If necessary, the upper edge can be shortened to change the height of the shell. Please note that the Spinova Immo Plus Classic is available in two heights. To attach the shell to the base support, carry out the steps listed in the **Removing the shell** section in reverse.

**Note: The plastic can be cut with standard leather shears. Smooth the cut edge (e.g. with sandpaper) to avoid causing injuries.**

##### Positioning the abdominal pads

Place the orthosis in front of you with the inside facing upward. The arrows on the inside of the abdominal fastenings should be pointing upward. Attach the abdominal pad to the right-hand abdominal fastening.

\* Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product).

† A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

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