AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear client,

thank you for choosing a product from Bauerfeind. GenuTrain A3 is a high-quality medical product. Please read through these **instructions for use** carefully and if you have any questions contact your doctor, medical retailer or our Technical Service department.

Intended use

GenuTrain A3 is an active support for the treatment of complex knee symptoms which produces its effect during movement, particularly during exercise.

During movement, the shaped viscoelastic insert compresses and massages the soft tissues. The resulting enhanced circulation in the joint capsule and promotion of lymph drainage lead to reduced swelling, pain relief and improved function while, at the same time, taking the strain off the kneecap. The three-dimensional knit ensures an anatomically perfect fit for the GenuTrain A3. The compression-reduced support edges stop the support cutting into the leg.

⚠ GenuTrain A3 must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

Indications

- Irritation of the knee, particularly joint wear (including effusion in the elderly), ligament insertion disorders, tendomyopathy, posttraumatic and postoperative
- Functional instability with muscle imbalance
- Prevention and treatment of anterior knee pain
- Lateral shifting of the patella
- Patellar tendinopathy)

Risks of using this product

⚠ Important Information

- GenuTrain A3 is a prescribable product. In case you have received it by prescription from your doctor or physiotherapist, the use and duration of use must be agreed with her / him before utilizing the product. In any case, instruction is required by the specialist who provided you with the product.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- The product's unique action is produced primarily during physical activity. The product should be removed during prolonged rest breaks (e.g. sleeping, continuous sitting).
- Avoid the use of ointments, lotions or any substances including grease or acids.
- Side effects, that affect the entire organism, have not been reported to date. Correct fitting is

assumed. Any supports and orthoses¹ applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

LATEX Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
2. Varicose veins.
3. Impaired sensation and circulatory disorders of the legs (e.g. diabetes mellitus).
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

Fitting GenuTrain A3

1. Firmly grip the top of the support with both hands (one either side of the lateral spines) **1** and pull evenly over the knee joint until its knee cap is free from pressure and is surrounded by the silicone insert. **2 – 4**
2. Adjust as required until a snug fit is achieved.
3. The bandage must not be put on using the wrong donning aid, as this can destroy the silicon coating.

Removing GenuTrain A3

To remove the support, pull down at the bottom of the support near the stays. **5**

Cleaning instructions

All Train products² can be machine washed separately at 30 °C with liquid detergent. Please note the following when washing your Train products:

We recommend using a mesh bag (spin cycle: maximum of 500 revolutions per minute). Air dry your Train product. Do not dry it in a dryer because this can compromise the knitted fabric. Please wash your Train product regularly so that the compression of the knitted fabric can be retained.

This product has been tested by our in-house quality management system. However, should you have any complaint, please contact our local expert adviser or our customer service department.

Notes on reuse

GenuTrain A3 is intended for your treatment only and has been specifically selected according to your measurements. Do not pass your GenuTrain A3 on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective

products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the GenuTrain A3 in accordance with national legal requirements.

Version: 2019-05

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

² Standardized washing tests show that after 50 washing machine cycles full functionality of the Train active support is retained. (Liquid detergent was used.)

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