AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear patient,

thank you for choosing a product from Bauerfeind. MOS-Genu is a high-quality medical product.

Please read the **instructions** carefully and if you have any questions contact your medical provider, medical retailer or our Technical Service department.

Intended use

MOS-Genu is a therapeutic device designed to aid your treatment. It helps to protect your knee during therapy as it assists your muscle development during physical therapy exercises.

MOS-Genu must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wear comfort from this orthosis¹.

Indications

- Non-surgical
- Cruciate and lateral ligament injuries
- Complex instabilities including genu recurvatum
- Support for the affected compartment in bowleg / knock-knee arthritis

- Post-op. (short MOS-Genu)
- Operations on the ligaments, i.e. artificial ligaments
- Meniscal suturing and meniscal replacement (meniscal implantation)

- Post-op. (long MOS-Genu)
- After axial correction of the shinbone (tibial osteotomy)
- After complex reconstructions of the ligaments
- In individual cases it is also indicated for fractures located close to joints

Risks of using this product

Important information

- MOS-Genu is a prescribed product that should be applied under a physician's guidance. MOS-Genu should only be applied in accordance with the instructions contained in these usage directions and only for the listed areas of application.
- No liability is accepted for improper use of the product.
- Changes must not be made for the product. Non compliance can reduce the performance of the product and thus no liability will be accepted.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your medical provider without delay.
- A combination with other products (e.g. compression stockings) must be discussed beforehand with your medical provider.
- Avoid the use of ointments, lotions or any substances including grease or acids.
- MOS-Genu is not designed to prevent injuries, especially when engaging contact sports. Contact sports inherently pose a greater risk of injury.

- If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to local pressure signs or, rarely, constrict the underlying blood vessels or nerves.

LATEX Contains natural rubber latex (elastodiene), which can cause skin irritation in persons with an allergy. Take off the orthose if you notice any allergic reaction.

Contraindications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical provider:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Varicose veins.
- Impaired sensation and circulatory disorders of the legs, e.g. diabetes mellitus.
- Impaired lymph drainage – including soft tissue swellings of uncertain origin located remotely from the fitted aid.

Application instructions

To make MOS-Genu as comfortable as possible for you to wear, please observe the following points:

- Keep the leg as clean as possible, particularly after the operation.
- Please put on and take off MOS-Genu whilst seated.

Putting on the orthosis

- Place MOS-Genu around your knee.
- Rotate round the knee pads until they are lying flat on the sides of the knee.
- Fasten the lower part of the orthosis as follows: Centre the main fixing strap incorporating the pad, pull on both sides simultaneously and secure the Velcro fastening. In the same manner, attach the strap below, beginning with the strap closest to the joint.
- Secure the upper part of the orthotic support as follows: First centre the strap closest to the joint, pull on both sides simultaneously and secure the Velcro fastening. Follow a similar procedure for the uppermost straps.
- Always pull the straps tight on both sides simultaneously.
- To ensure that the orthosis is fitted properly, use a mirror for verification.

Taking off the orthosis

IMPORTANT!

- Unfasten the main fixing strap incorporating the pad last.**
- First release all thigh straps, working from one side of the orthosis only.
- Then lift the upper part from the thigh. During this process the round pads at the sides remain in contact with the knee.
- Now, working from one side of the orthosis only, release straps on the lower leg section of the orthosis.
- After releasing all straps, slip the orthosis aforward and off the knee joint.

Product care

Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

The aluminium parts of the support are coated and can be cleaned with a pH-neutral soap. We recommend washing the pads at 30 °C on a gentle wash cycle and hand-washing the straps at 30 °C.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (knee). See purpose.

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Technical specifications / parameters

The MOS-Genu is a stabilizing orthosis for the knee joint. It consists of an upper and lower frame section, a hinge, cushions and straps.

Notes on reuse

The MOS-Genu is intended for your treatment only. It has been adjusted specifically for you. For this reason, do not pass the MOS-Genu on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

	
Version: 2018-04	

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

	
---------------	---------------

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