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- ### Indications
- Prevention of dislocation
 - Total replacement surgery
 - Total replacement revision surgery
 - Femoral head resection (Girdlestone arthroplasty)
 - Hip spacer for two-stage revision procedures
 - Osteoarthritis of the hip (moderate to severe)
 - Instabilities

Risks of using this product

⚠ Caution

• Side effects are not known to date. Proper use / application is assumed. All therapeutic aids applied externally on the body – supports and orthoses – can cause local pressure (irritations) if applied too tightly or in rare cases also constrict blood vessels or nerves. When fitted immediately after surgery, it is important to ensure that there are no unnecessary pressure points in the region of the surgical wound.
• Car devices are prescription medical devices. It is to be applied under medical supervision. SofTec Coxa must only be applied in compliance with these Instructions for Use and the indications listed.
• The SofTec Coxa is not to be opened or modified by a specific user.
• Manipulation of the joint must only be performed by appropriately trained specialist personnel. Failure to observe this may impair the performance of the product such that product liability is excluded.
• Improper modification of the product, including retrospective changes to the knitted fabric component must not be undertaken. Failure to observe this may impair the efficacy of the product such that product liability is no longer provided.
• If you experience unusual changes (e.g., your symptoms get worse), consult your physician immediately.
• Combination with other products must be agreed to in advance by the attending physician.
• Please do not allow the product to come into contact with fatty or acidic substances, ointments and lotions.
• Do not use the product in conjunction with household waste without problems.
• For package recycling see local regulations.

⚠ Contains natural rubber latex (Elastodien) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications
Pathological hypersensitivities are not known to date. Given the following symptoms, the application and use of such an aid is only indicated following consultation with your physician.

1. Skin diseases / injuries in the region of the body under treatment, especially those with inflammatory phenomena, as well as open scars with swelling, reddening and overheating

2. Sensory deficits

3. Increased impairment of performance of the heart and lungs (risk of raised blood pressure with the application of aids and increased physical activity).

Application and removal of the SofTec Coxa:
• Place SofTec Coxa around your pelvis. Position the orthosis snugly on your iliac crests. Take care that the side bar runs laterally, centered on the thigh **1**.
• For basic positioning, insert your fingers into the finger pockets on both parts of the side bar and pull them evenly towards the front **2**. Take care here that the straps run along the lower part of the pelvic shells and across both ends of the straps with Velcro® tape to the lower part of the front fastener **3**. The ends of the straps can also be attached to one another with Velcro® tape, as required.
• For easier handling of the SofTec Coxa, always refasten the opened tension straps loosely right away with the Velcro® tape. We need to help you practice putting on the orthosis correctly. There must be basic tension of the straps on the pelvic shells so that the SofTec Coxa cannot slip. Straps that are too loose lead to a loss of medical efficacy.

Leg harness:
• Position the outside leg harness centered on the thigh and place the leg support around the thigh from the back. Insert your fingers into the finger pockets on both halves of the fastener provided for this purpose **4**.

• Pull the Velcro® tape from the side of the fastener coming from the inside with the fastener coming from the outside **5**.

• Now fasten the two front straps on the leg harness **6**.

• To fix the SofTec Coxa, take the tension straps, which are passed around the back, in both hands and pull them evenly towards the front **7**. Take care here that the straps run along the lower part of the pelvic shells and across both ends of the straps with Velcro® tape to the lower part of the front fastener **8**. The ends of the straps can also be attached to one another with Velcro® tape, as required.

• For easier handling of the SofTec Coxa, always refasten the opened tension straps loosely right away with the Velcro® tape. We need to help you practice putting on the orthosis correctly. There must be basic tension of the straps on the pelvic shells so that the SofTec Coxa cannot slip. Straps that are too loose lead to a loss of medical efficacy.

Leg harness:
• Position the outside leg harness centered on the thigh and place the leg support around the thigh from the back. Insert your fingers into the finger pockets on both halves of the fastener provided for this purpose **4**.

• Pull the Velcro® tape from the side of the fastener coming from the inside with the fastener coming from the outside **5**.

• Now fasten the two front straps on the leg harness **6**.

• To fix the SofTec Coxa, take the tension straps, which are passed around the back, in both hands and pull them evenly towards the front **7**. Take care here that the straps run along the lower part of the pelvic shells and across both ends of the straps with Velcro® tape to the lower part of the front fastener **8**. The ends of the straps can also be attached to one another with Velcro® tape, as required.

• For easier handling of the SofTec Coxa, always refasten the opened tension straps loosely right away with the Velcro® tape. We need to help you practice putting on the orthosis correctly. There must be basic tension of the straps on the pelvic shells so that the SofTec Coxa cannot slip. Straps that are too loose lead to a loss of medical efficacy.

Leg harness:
• Position the outside leg harness centered on the thigh and place the leg support around the thigh from the back. Insert your fingers into the finger pockets on both halves of the fastener provided for this purpose **4**.

• Pull the Velcro® tape from the side of the fastener coming from the inside with the fastener coming from the outside **5**.

• Now fasten the two front straps on the leg harness **6**.

• To fix the SofTec Coxa, take the tension straps, which are passed around the back, in both hands and pull them evenly towards the front **7**. Take care here that the straps run along the lower part of the pelvic shells and across both ends of the straps with Velcro® tape to the lower part of the front fastener **8**. The ends of the straps can also be attached to one another with Velcro® tape, as required.

• For easier handling of the SofTec Coxa, always refasten the opened tension straps loosely right away with the Velcro® tape. We need to help you practice putting on the orthosis correctly. There must be basic tension of the straps on the pelvic shells so that the SofTec Coxa cannot slip. Straps that are too loose lead to a loss of medical efficacy.

Removing the SofTec Coxa:
• Open the front tension straps of the leg support first and attach these loosely to the tension straps with the Velcro® tape and then open the closure belt in the leg support. Then open the straps on the pelvic harness and attach them loosely to the straps with Velcro® tape. Now the closure belt on the pelvic harness can be opened. Keep the SofTec Coxa tight on the side on which the joint / leg shell is located.

• For easier handling of the SofTec Coxa, always refasten the opened straps loosely with the Velcro® tape.

Correct application has to be practiced with you.

Care instructions
• Do not expose the orthosis to direct heat (e.g., heaters, sunlight, etc.)!
• The SofTec Coxa knitted fabric may be cleaned by hand washing at 30 °C with mild detergent.
• The pelvic harness can be cleaned easily with a sponge and pH-neutral soap or shampoo.
• Please consult your orthotist about the necessary disassembly and subsequent assembly in this case.

We have tested the product as part of our integrated quality management system. Should you still have a complaint, please contact our technical advisor on-site or our customer service department.

Part of the body this product is used for
In accordance with the indications (Hip). See Purpose.

Maintenance instructions
With correct handling and care, the product is practically maintenance-free.

Technical specifications / parameters, accessories
The SofTec Coxa is an orthosis for the hip joint. It consists of a three-part pelvic frame – a hinge, a pelvic frame and a thigh harness.

Notes on reuse
The SofTec Coxa is intended for your treatment only.

Warranty
The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself! This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report
Due to regional legal regulations, you are obliged to immediately report any serious incident involving the use of this medical product to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal
You can dispose of the product in accordance with national legal requirements.

Information status: 2019-08

1 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
2 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
3 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
4 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
5 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
6 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
7 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
8 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
9 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
10 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
11 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
12 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
13 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
14 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
15 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
16 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
17 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
18 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
19 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
20 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
21 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
22 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
23 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
24 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
25 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
26 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
27 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
28 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
29 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
30 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
31 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
32 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
33 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
34 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
35 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
36 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
37 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
38 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
39 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
40 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
41 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
42 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
43 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
44 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
45 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
46 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
47 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
48 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
49 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
50 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
51 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
52 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
53 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
54 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
55 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
56 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
57 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
58 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
59 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
60 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
61 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
62 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
63 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
64 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
65 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
66 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
67 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
68 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
69 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
70 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
71 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
72 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
73 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
74 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
75 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
76 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
77 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
78 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
79 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
80 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
81 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
82 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
83 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
84 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
85 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
86 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
87 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
88 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
89 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
90 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
91 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
92 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
93 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
94 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
95 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
96 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
97 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
98 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
99 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
100 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
101 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
102 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
103 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
104 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
105 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
106 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
107 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
108 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
109 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
110 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
111 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
112 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
113 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
114 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
115 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
116 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
117 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
118 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
119 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
120 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
121 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
122 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
123 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
124 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
125 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
126 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
127 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
128 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
129 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
130 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
131 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
132 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
133 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
134 Specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.
135 Orthesis = orthopedic appliances used to stabilize, relieve, immobilize, control or correct the limbs or the torso.
136 Specialist is any person who is authorized, according to their