AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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www.bauerfeind.de/barriererfrei

en english

These instructions for use can also be downloaded from: www.bauerfeind.com/downloads

Dear patient,

Thank you for choosing a product from Bauerfeind Orthopedics.

LumboLoc Forte is a high-quality medical product.

Please read through these **instructions for use** and if you have any questions contact your doctor, medical retailer or our Customer Service department.

Purpose

LumboLoc Forte is an orthosis¹ for optimal spinal posture and relief of the lumbar spine. With the back pad (pressure pad) adapted to the correct anatomical position, the orthosis exerts continuous, extensive massage of the back muscles. The adjustable strap supports the effect of the pad, as well as permits individual adjustment of the force exerted in respect of height location and intensity.

LumboLoc Forte must be individually adapted to the patient by a trained orthopaedic technician in order to gain the maximum effect and optimal wearing comfort from this orthosis.

If the product is not used properly any claim for warranty will expire.

Indications

- Non-specific lower back pain (lumbago)
- Weak back/abdominal muscles (muscular insufficiency)
- Slight vertebral displacement (spondylolisthesis)
- Ligament/tendon/muscle irritation (ligamentotendomyopathies)

Risks of using this product

- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Any supports and orthoses applied externally to the body can, if tightened excessively, lead to localized pressure or, rarely, constrict the underlying blood vessels or nerves.
- LumboLoc Forte is a prescribed product that must be applied under a physician's guidance. LumboLoc Forte must only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- The product must be applied over a T-shirt or an undershirt.
- Incorrect use of the support will cancel any product liability.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby cancelling any product liability.
- Should you notice any unusual changes (e.g. increased symptoms), please contact your doctor without delay.
- Please consult your doctor before using LumboLoc Forte in combination with other products.
- Do not allow the product to come into contact with ointments, lotions or substances containing grease or acids.

Never expose the orthosis to direct heat (e.g. heater, sunlight, etc.)!

Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contra-indications

Hypersensitive reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and worn after consultation with your doctor:

- Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness and excessive heat build-up.
- Impaired sensation.
- Significant restriction of cardiopulmonary performance (risk of raised blood pressure if strenuous physical exertion is undertaken while wearing a therapeutic device).

Application instructions

Fitting and removing LumboLoc Forte

Fitting:

- Place the orthosis around the trunk so that the stays are positioned across the back, as shown in Fig. 1 (sewn-in labels at the top). The lumbar pad should be located mainly over the transition between the sacrum and the lumbar spine, and the sacral pad mainly below this point.
- Place the fingers in the specially provided finger pockets on the two broad, light-grey fastener halves and pull forward uniformly.
- Press the left half of the fastener against the abdomen and pull the right half over it to secure the velcro fastening (Fig. 2).
- When securing the velcro fastening, remove the fingers from the left finger pocket first, followed by the right (Fig. 3).
- Take the two lower LPT (Low Power Tension) straps in both hands and pull forward at the same time to produce the desired level of force (Fig. 4). Next, secure both LPT straps against the large velcro fastening (Fig. 5).
- **Correct fitting should be practiced with the patient.**

Removal:

- Unfasten the LPT tension straps and then refasten without any tension.
- Open the large, light-grey fastener.
- Remove the orthosis and refasten the light-grey fastener.

- For easier handling of LumboLoc Forte, always fasten any opened fasteners directly on the product.

Cleaning instructions

The sewn-in label of the LumboLocForte, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the inside of the knitted fabric.

- We recommend washing LumboLoc Forte separately by hand with a mild detergent at 30 °C (86 °F). To avoid damage and to preserve their long-term adhesive strength, we recommend closing the velcro fasteners before washing.
- The knitted fabric and strap should be separated from each other for cleaning purposes.
- Remove the pad for cleaning. The pad can be cleaned with a damp sponge.

This product has been examined by our own quality control system. However, should you have any complaint, please contact your retailer or our customer service department.

Part of the body this product is used for

In accordance with the indications (lumbar spine). See intended use.

Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

LumboLoc Forte is intended for single use by the patient.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly.

Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2018-06

Notes for the orthopedic technician

The height location of the fitted function strap can be adjusted by means of the velcro fastening. The corset stays should be adjusted as necessary.

Attaching the pad

- Two types of pads can be used depending on the indication: a lumbar or sacral pad.
- The pads must be velcro-fastened to the specially provided fleece belt on the inside of the support. For the lumbar pad (Fig. A), the wide end of the pad should always be at the top. The reverse applies to the sacral pad (Fig. B).

- The patient's anatomy should be taken into account when positioning the pad. Depending on the individual situation, the pad can be shifted up or down to achieve the correct fit.

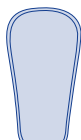
Fig. A
lumbar pad

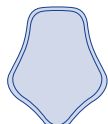
Fig. B
sacral pad

¹ Orthosis = orthopaedic appliance used to stabilize, relieve, immobilize, control or correct the limbs or the torso

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