ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE
The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:
• Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS
• Elastic lumbar support with 4 flexible lumbar stays and 2 abdominal stays.
• Indicated for the treatment of acute or chronic lower back pain.

CONTRAINDICATIONS
• None

WARNING & PRECAUTIONS
The product is intended for single patient use only. If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.

- APPLICATION INSTRUCTIONS**
- 1) Position the back support by placing the dorsal part against the lower back and taking care to center it along the axis of the spine.
 - 2) Adjust and tighten the back support around the waist according to the support required. Secure using the contact closure at the front of the support.
 - 3) Adjust and tighten the lateral straps, taking care to center their central part along the axis of the spine.
 - 4) Pull on the edges of both lateral straps, at the same time, and secure them on the front.

CARE
Wash in lukewarm water with a mild detergent. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry-clean. Do not iron.

COMPOSITION
Polyamide, Polyester, Plastic (Acetal and PP), Steel, Cotton, Rubber

LATEX CAUTION: THIS PRODUCT CONTAINS NATURAL RUBBER LATEX WHICH MAY CAUSE ALLERGIC REACTIONS.

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale to the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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©2018 DJO, LLC
MDS5 GmbH