**DONJOY**

ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to:

- Read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

INTENDED USE/INDICATIONS

- Reducible unilateral inguinal hernias
- Pre-operative use

CONTRAINDICATIONS

- None

WARNING & PRECAUTIONS

- We recommend that the first application be made under supervision of a health care professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- If worn directly on the skin, the skin should be free from injuries, moisture, oils, creams, gels or similar residues.

APPLICATION INSTRUCTIONS

- 1) Apply foam pad to right/left of rigid section according to location of inguinal hernia.
- 2) Put belt on hips and fasten waist strap in front
- 3) Tighten belt by pulling on grey velvet. Insert velvet into loop to keep tightening.
- 4) Rotate belt to position foam pad on the hernia
- 5a) Insert the under-thigh strap into white buckle on the side of the inguinal hernia. Pass strap under the thigh and attach to the central section.
- 5b) Adjust the fit using the white velcro tab.
- 5b) Réglage le serrage au niveau de la boucle blanche.

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