**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
F +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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(en) english

## Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

### Intended use

The ViscoPed S viscoelastic foot orthoses are medical products used for gentle correction of static or dynamic misalignment of the skeleton of the foot, providing shock absorption when the heel hits the ground, pressure relief, and / or redistribution of pressure in the entire heel-to-toe area of the foot.

⚠ ViscoPed S must be fitted for the first use by a trained specialist, otherwise we cannot accept any liability for the product.

### Indications

- Pressure pain around the sole of the foot
- Gentle correction of static or dynamic misalignment of the skeleton of the foot
- Pressure redistribution to prevent local pressure peaks
- Cornification of the feet
- Atrophy of the plantar fat pad
- Ulcer prevention for diabetes mellitus

### Risks of using this product

#### ⚠ Important Information

- ViscoPed S is a prescribed product that should be applied under a physician's guidance. ViscoPed S should only be applied in accordance with the instructions contained in these usage directions and for the listed areas of application.
- No liability is accepted for improper use of the product.
- The product should not undergo any improper modification. Failure to comply with the requirement can impair product performance and thus rule out any product liability.
- If you notice any unusual changes please contact your medical provider or our customer service department immediately.
- Never expose the ViscoPed S to direct heat (e.g. heater)! This may cause damage to the material, which can impair the effectiveness of the ViscoPed S.
- Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed.

### Contra-indications

If you have any of the following conditions, the ViscoPed S foot orthoses should only be fitted and worn after consultation with your physician:

Skin disorders / injuries to the foot, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up. The foot orthoses are not suitable for diabetics who have diabetic foot syndrome.

### Fitting ViscoPed S

1. Place the ViscoPed S insoles into the shoes as marked (R = right, L = left) so that they are flush with the heel area of the shoes.
2. If the ViscoPed S insoles are too large, the front part (not the heel) can be trimmed with a pair of scissors. The insoles are marked accordingly.
3. Place the fitting aid on the insole with the fluffy side downwards and slip foot into the shoe. Remove fitting aid.
4. Unless otherwise prescribed by the physician treating you, insert both the left and right version of the foot orthosis into your shoes.

### Product care

We recommend washing ViscoPed S handwarm with a mild detergent.

If necessary, the ViscoPed S foot orthoses can be washed with a mild detergent in lukewarm water and left to air dry. Avoid direct heat (e.g. heating, sunlight) as it may cause damage to the material. This can impair the effectiveness of the foot orthoses.

### Part of the body this product is used for

The foot orthoses must be worn in conjunction with hosiery in suitable shoes with a sufficient internal volume for both the foot and the selected orthosis.

### Notes on reuse

This product is intended for your treatment only.

### Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoPed S have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

### Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

### Disposal

The product can be disposed of without any restrictions.

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