Montageanleitung
Assembly instructions
Instructions d'assemblage
Indicaciones de montaje

AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

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en english

These instructions for use can also be downloaded from:
www.bauerfeind.com/downloads

Dear customer,

Thank you for placing your trust in a Bauerfeind product. The GenuTrain S Pro you have purchased is a high-quality medical product.

Read the **instructions** carefully and in full, as they contain important information on the use, indications, contraindications, application, care and disposal of the GenuTrain S Pro. Keep these instructions in case you need them again in the future. Consult your physician or specialist store with any questions you may have.

Purpose

The GenuTrain S Pro is an anatomically contoured active support for the knee. The lateral joint splints, which offer the possibility to restrict the movement of the knee, are held directly in place by the Velcro strap system and thus support the natural pattern of movement. During movement, the viscoelastic insert compresses and massages the soft tissues. The resulting enhanced circulation in the joint capsule and promotion of lymph drainage lead to reduced swelling, pain relief and improved function while, at the same time, taking the strain off the kneecap.

Indications

- Light to moderate instability of the knee
- Inflammatory changes in the knee joint (arthritis), e.g. chronic polyarthritis
- Deterioration of the knee joint (gonarthrosis)
- Meniscus injuries

Risks of using this product

⚠ Caution*

GenuTrain S Pro must only be worn in accordance with the specifications contained in these instructions and used for the areas of application listed. To ensure the optimum fit of the GenuTrain S Pro, it is essential to check whether the product is adequately shaped. The GenuTrain S Pro can be reshaped by a specialist¹ to suit the patient's anatomy if necessary.

Use in combination with other products, e.g. as part of compression treatment (involving wearing compression stockings), must be discussed beforehand with your physician.

Should you notice any unusual changes (e.g. increased symptoms), please contact your physician without delay.

No product liability is accepted in the event of improper use.

Do not allow the product to come into contact with greasy or acidic agents, ointments and lotions.

Do not make any changes to the product. Failure to comply with this requirement may adversely affect product performance, thereby voiding any product liability.

Side effects involving the body as a whole have not been reported to date. Correct fitting is assumed. Do not put the GenuTrain S Pro on too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, loosen the straps of the GenuTrain S Pro and, if necessary, have the size of your GenuTrain S Pro checked.

⚠ Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

Hypersensitivity reactions harmful to health have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your physician:

- Skin disorders /injuries in the relevant part of the body, particularly if inflammation is present; likewise, any raised scars with swelling, redness and excessive heat build-up
- Varicose veins
- Impaired sensation and movement disorders of the legs (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin located away from the fitted support

Application instructions

Putting on the GenuTrain S Pro

A – Support , B – Velcro straps, C – Guide channels with joint splints

➊ First loosen the four ends of the Velcro straps of the GenuTrain S Pro without pulling them out of the slits. ➋ Grip the tightening straps at the sides with your fingers and pull the support over the affected knee until the kneecap is enclosed by the massage ring (pad), without any pressure being exerted on it. The Velcro ends of the straps should now be on the outside of the leg. ➌ By pulling the straps you can stabilize the knee comfortably. ➍ Fasten all four of the strap ends.

Taking off the GenuTrain S Pro

➎ Undo the straps. Grip the support at the level of the calf. Pull the support off in a downward direction.

Cleaning instructions

Note: Never subject the GenuTrain S Pro to direct heat (e.g. heaters, sunlight, in your car, etc.)! This could damage the material, which can impair the effectiveness of the GenuTrain S Pro.

The sewn-in label of the GenuTrain S Pro, which contains information on the product name, size, manufacturer, washing instructions and CE mark, is located on the upper edge of the support.

Remove the Velcro straps. Fasten the ends back onto the straps to avoid any damage. Pull the joint splints upward and out of the guide channels. Wash the support separately in a lingerie bag with a mild laundry detergent at 30 °C (no woolite). Do not run the spin cycle on your washing machine at speeds higher than 500 rpm. Wash the GenuTrain S Pro regularly so that the knit fabric retains its compressive properties.

Once the support has dried, guide the joint splints back into the guide channels from above, in line with the markings. The joint thus point toward the hollow of the knee. Fasten the straps onto the eyelets of the joint splints. If you have any complaints, please contact your medical retailer only. Please note that only clean goods can be dealt with.

Part of the body this product is used for

In accordance with the indications (knee joint).

Maintenance instructions

With correct handling and care, the product is practically maintenance-free.

Assembly instructions

The GenuTrain S Pro is supplied in an anatomically contoured state. The active support is available for both the right and the left knee.

Technical data / parameters

The GenuTrain S Pro is an active support for the knee. The joint splints and the straps can be removed. There are 5 screws (1 replacement screw) included with the product. The screw set is available as a spare part.

Notes on reuse

The GenuTrain S Pro is intended for your treatment only. It has been adapted specifically for you. For this reason, do not pass the GenuTrain S Pro on to other people.

Warranty

The legal provisions of the country in which the product was purchased apply. Country-specific warranty regulations that govern the relationship between the retailer and the buyer also apply where relevant. If you believe that a claim may be made under the warranty, please first contact the retailer from which you purchased the product directly. Please do not make any changes to the product yourself. This particularly applies to individual adjustments made by specialists. Please follow our instructions for use and care. These are based on our experience gained over many years and ensure the functionality of our medically effective products for a long time. Only aids that function optimally can help you. Failure to follow the instructions may also restrict the warranty.

Disposal

You can dispose of the product in accordance with national legal requirements.

Version: 2017-07

Specialists

General notes

- Check the optimal and individual fit of the GenuTrain S Pro after the first fitting. Make corrections if necessary.
- Practice the correct procedure for putting on the orthosis with the patient.
- The product must not be changed in any improper way. Failure to comply with this requirement may adversely affect the medical and therapeutic effectiveness of the product, thereby voiding any product liability.
- The GenuTrain S Pro is intended for the treatment of one patient only.

Shaping the joint splints:

- ➊ Remove the Velcro straps.
- ➋ Take the joint splints out of the guide channels on the support. Check the fit of the joint splints on the patient. Pay attention here to any particular anatomical conditions (e.g. fibular knobs).
- ➌ If necessary, adjust the joint splints with a hot-air gun at 120–140 °C. ⚠ Wear protective gloves while you are doing this.
- ➍ Place the support on its side so that the guide channel is directly in front of you. The arrow marking on the joint should be pointing upward, while the curve of the joint splints points toward the hollow of the knee.
- ➎ Insert the joint splints into the guide channel through the large upper slits at the back.
- ➏ Pull the guide channels outward and upward for easier positioning at the tightening straps.

➐ Adjusting the hinge:

⚠ **Risk of injury: do not adjust the hinges on the patient!**
Flexion

Place a screw in the hole provided and screw it tight to limit the flexion angle of the joint splints. The flexion can be optionally restricted to 30° (lower hole), 60° (middle hole) or 90° (upper hole).

Extension

To limit the extension angle to 20°, insert a screw into the splint hinge from the outside and screw it tight.

Flexion and extension

It is possible to restrict both flexion and extension using two screws on each side of the hinge. Follow the procedure described above.

Fitting the straps:

- ➐ Sort the straps according to length and width. Arrangement: wide straps for the thigh area, narrow ones for the lower leg, longer straps for the back of the support, shorter ones for the front (side with Bauerfeind B). Place the support in front of you with the front facing downward.
- ➑ The straps are always attached at the upward-facing eyelets, starting from the inside of the support (take right / left version into account). Push the two longer straps with the straight ends and Velcro fastening upward through the eyelets. Fasten the ends together just after the eyelets.
- ➒ Then guide the rounded strap ends with Velcro fastening upward through the eyelet on the opposite side. Fasten the strap ends just after the eyelet.
- ➓ Turn the support over. Now follow the same procedure with the shorter straps.

¹ A specialist is any person who is authorized according to the state regulations for fitting and instruction in the use of supports and orthoses which are relevant to you.

* Note on the risk of personal injury (risk of injury, harm to health and accidents) or damage to property (damage to the product).

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