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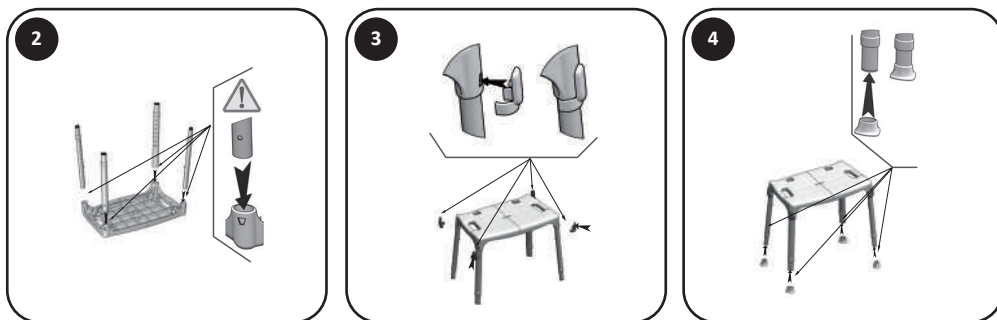
EN

CAPRI STANDARD - CAPRI ANATOMIQUE: SHOWER SEAT

INTENDED USAGE:
The products of the Shower Seats line allow the user an increase in independence during the shower stage of daily personal hygiene, reducing muscle fatigue due to standing position as well as slip and injury risks. The non-slip rubber feet ensure grip with the floor, and the height adjustable legs allow users to change the total height according to their needs easily. In addition, the shower seat with customised height can be used with different length settings between the front and rear legs. In particular, the front legs can be lower than the rear ones: this allows the user to keep a more correct posture while taking a shower, as well as to facilitate standing and sitting procedures. Due to its flexibility, the shower seat can be positioned even on a bumpy and irregular floor, like those typical of many shower trays. The versions provided with backrest and/or armrests give additional safety and comfort. The shower seat can be easily moved due to its lightweight. The seat can be completed with a soft cushion and a soft cushion for backrest to make the use more comfortable. The soft cushions, armrests, and backrest are available separately as accessories.

INDICATIONS AND CONTRAINDICATIONS:
The Shower Seats line products are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty maintaining the standing position during the shower stages of daily personal hygiene. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The shower seat must not be used if the user has problems related to balance or perception disorders.

- SAFETY WARNINGS AND INDICATIONS:**
- Do not exceed the maximum user weight indicated in the fitting instructions.
 - Before each use, ensure the shower seat is correctly assembled and positioned in a stable manner. The shower seat should only be used on surfaces free from obstacles.
 - Follow the assembly instructions strictly as shown in the User Manual.
 - After adjusting the height, make sure the legs are securely fastened.
 - Ensure the rubber caps are properly installed on all four legs, in the correct orientation.
 - If armrests and/or a backrest are installed, ensure they are securely attached before using the seat.
 - Make sure that any purchased accessories are attached in the correct orientation, as specified in the assembly instructions.
 - Be aware that the risk of finger injuries may increase during the assembly process.
 - Do not swing on the shower seat, as this may cause slipping, tipping over, or instability of the legs. Always ensure the seat remains on all four legs.
 - Do not stand on the shower seat or shower chair.
 - Do not alter or modify the shower seat or its components in any way.
 - Do not use the shower seat if it is damaged or has been altered in any way.
 - After use, remove the shower seat from the tray to prevent obstruction.
 - Never adjust the rear legs to a lower position than the front legs.
 - When lowering the front legs, ensure they are lowered by no more than one position.
 - Do not use the shower seat for purposes other than its intended use.
 - Do not store the shower seat outdoors.
 - Avoid direct contact with open wounds or broken skin to prevent infection.

MAINTENANCE INFO:
Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
 2. Remove the evident dirt before proceeding with the most accurate disinfection.
 3. Use water or non-aggressive detergent for the cleaning procedure.
 4. Wipe with a dry and clean cloth.
- Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

REUSE REQUIREMENTS:
The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

CORRECT DISPOSAL PROCEDURE:
Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

PERIOD OF USE:
The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.
If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

INCIDENT REPORTING:
If there is a serious incident involving this device, contact the manufacturer of this product at serviceclient@identites.tm.fr and the responsible authority in your country.

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