BARRE D'APPUI SOL-PLAFOND FIX

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DISPOSITIF MEDICAL

FIX FLOOR-TO-CEILING SUPPORT BAR

Instructions manual

MEDICAL DEVICE

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Manual de instrucciones

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REF. 815240

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.


Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Fix floor-to-ceiling support bar is a medical device. This product is intended for home use by individuals who have difficulty standing up. It allows users to safely pull themselves up and find support, even in areas that are not near walls where traditional supports are unavailable. The bar is securely fixed between the floor and the ceiling, provided that it is not a false ceiling. This device does not have any specific medical indications and is mainly intended for users seeking occasional or regular assistance to stand up independently.

PRODUCT DESCRIPTION

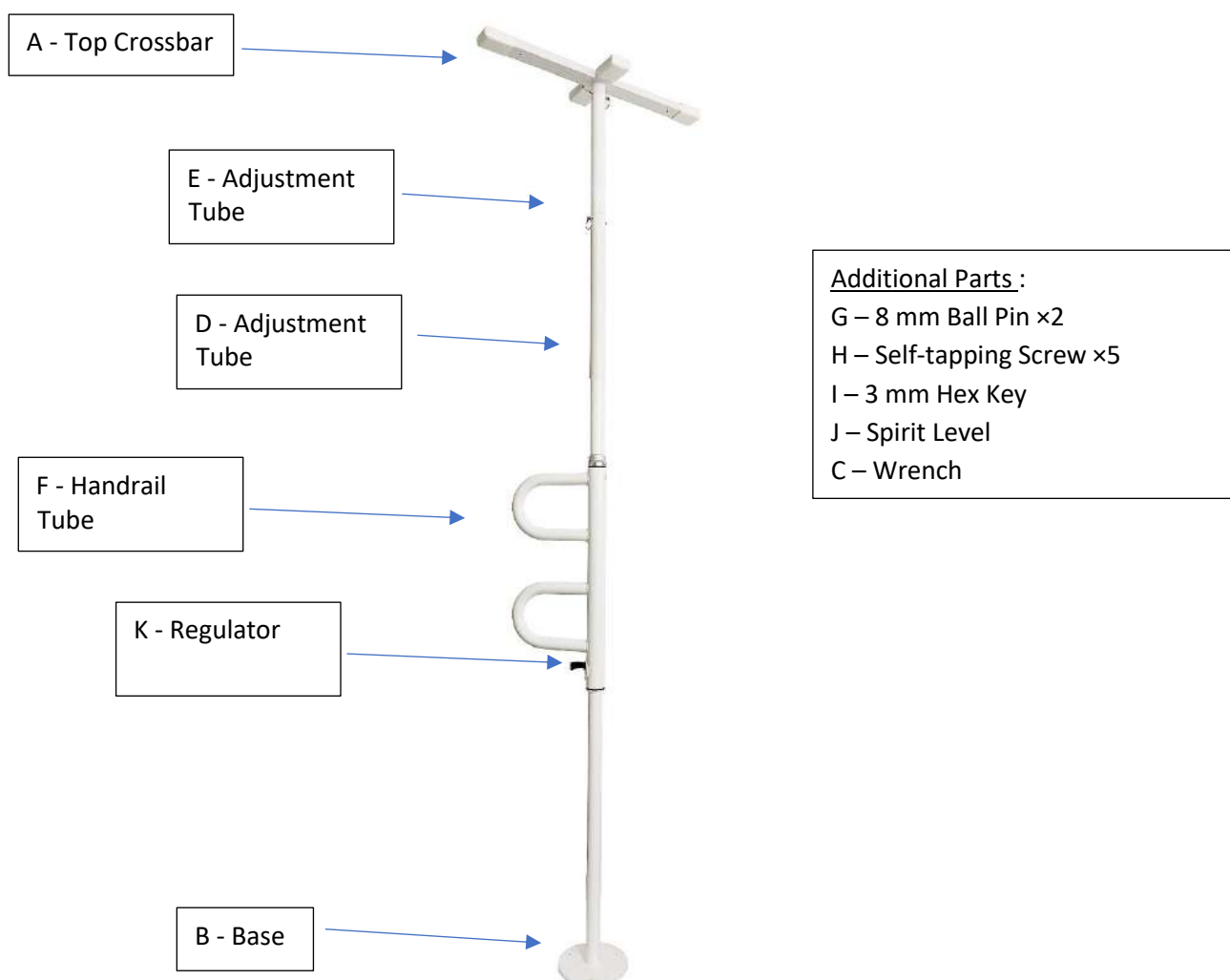SAFETY PRECAUTIONS


Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum authorized weight limit of 160 kg.
- This product must be properly assembled, installed, and maintained.
- This product must be installed by a qualified and competent person.
- Never drill into prestressed concrete to install the grab bar.
- Never install the product on a false ceiling.
- Always check for hidden electrical cables and water pipes before installation.
- Also verify legal requirements regarding grounding and the fastening of external metal parts.
- Inspect all parts before assembly to ensure they have not been damaged during shipping. If you find any damage, do not use the product and contact your distributor.
- Before use, make sure all screws are properly tightened and that the product is stable.
- Ensure all joints and connections are securely fastened and properly locked before use.
- If any parts are damaged, broken, worn, or missing, contact the distributor.
- Regularly check the general condition of the grab bar: look for signs of wear, deformation, or abnormal looseness.
- Do not use the grab bar if the floor or ceiling has defects, cracks, or lacks sufficient strength.
- Ensure that the floor and ceiling surfaces are flat, clean, and capable of supporting the load.
- Do not expose the grab bar to excessive humidity or corrosive environments.
- Do not modify or adapt the product in any way not authorized by the manufacturer.
- Keep out of reach of children when used in unsupervised environments.
- If the product is installed in a publicly accessible area, ensure compliance with local accessibility and safety standards.


ASSEMBLY AND USE

ASSEMBLY


1. Take out the assembly (D and F) and install it as shown in the diagram. Make sure to align the flat surface of the adjustment tube assembly (D) with the mark located at the interface of the handrail tube (F) before insertion.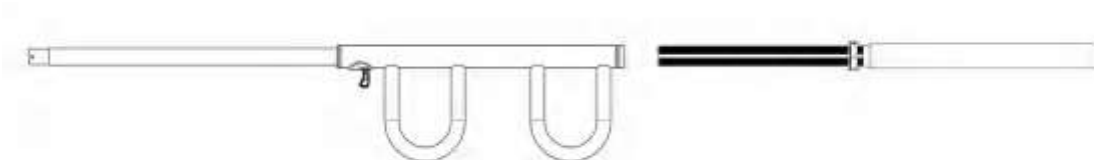


2. Take out the 8 mm ball pin (G) and the adjustment tube assembly (E), and install them in the locations indicated on the diagram. There are three adjustment levels (notches) inside the adjustment tube (E). Select the appropriate level for installation according to your needs.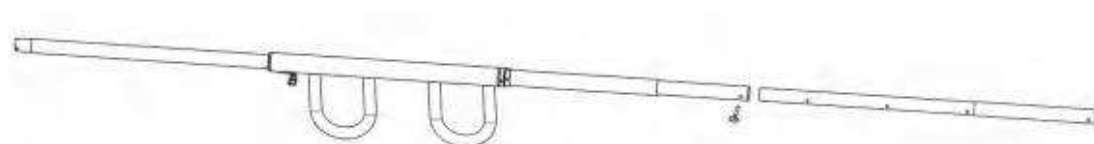


3. Take out the 8 mm ball pin (G) and the top crossbar (A), and install them in the locations indicated on the diagram.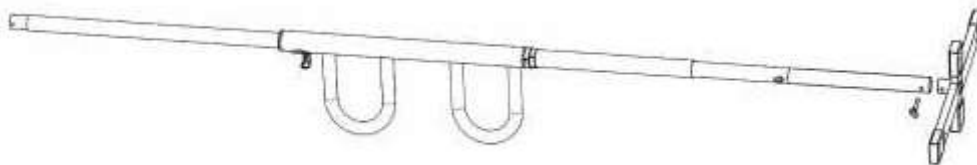

4. Lift the assembled part and insert it into the base (B).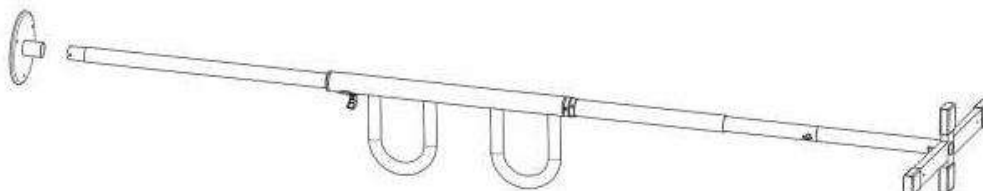

5. Use the wrench (C) to adjust the silver nut on the adjustment tube (D) until the product is firmly pressed against the ceiling, with no movement. Then use the 3 mm hex key (I) to tighten the small black screw on the silver nut (see the image below).
Note: The length of the threaded rod of the adjustment tube (D) used must not exceed 300 mm.

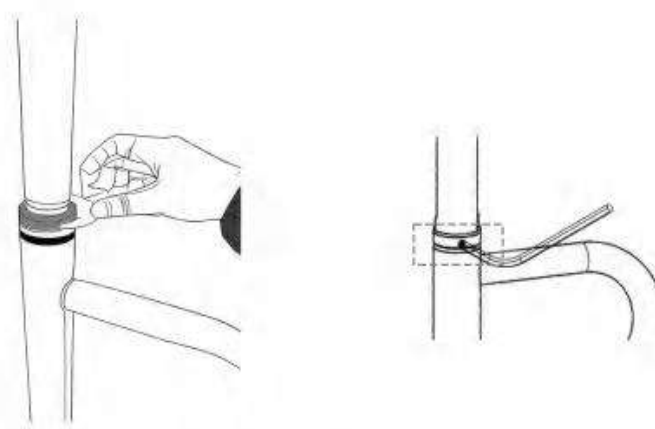6. Use the self-tapping screws (H) to fix the base (B) and the top crossbar (A).

USE

- Only install the product in an area where the mounting surface and its structure are sound and reliable.
- Also check the type of surface you plan to install it on, for example brick, wood, or concrete. Always ensure that the type of fixing you use is compatible with the surface.
- Before each use, it is essential to verify that the bar is properly installed, that all screws are tightly fastened, and that the bar is firmly pressed between the floor and the ceiling without any movement.
- The user must ensure that the bar correctly supports their weight and that the floor and ceiling surfaces are stable and solid.
- The grab bar is not designed to be used as a climbing device, nor to be subjected to excessive lateral forces. It must only be used as an aid for standing up or for slow and controlled movement. If you notice any looseness, abnormal movement, or visible wear, stop using the product immediately and contact the distributor for inspection or replacement of the affected parts.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean the product with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth to prevent oxidation.
- Do not use cleaning products containing phenol or chlorine.
- Periodically check the wear and stability of all screws, nuts, and other components.
- Before each use, check the fixings at the floor and ceiling.
- In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.

- Checklist :

To keep the product in good condition, check before each use, and regularly as follows :

Check the item	Anytime	Weekly	Monthly
General condition of the product and the handrail		X	
Cleanliness	X		
Screws and fixing elements		X	
Frame stability	X		
Tightening of screws and wear of parts			X

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place

- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL FEATURES

Adjustable for ceiling heights from 2.10 m to 3.00 m	
Base diameter (floor/ceiling): 19 cm	
Anti-corrosion metal structure	
PVC regulator (K) (contains DEHP)	
Maximum load capacity	160 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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