
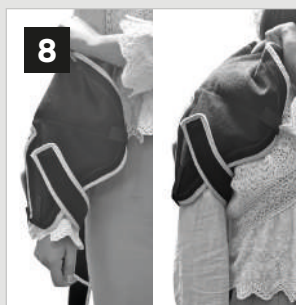
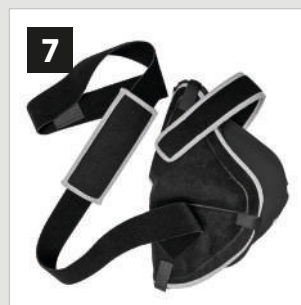
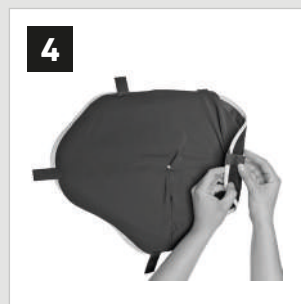
DONJOY®
SHOULDER ICE

enovis™

COLD PACK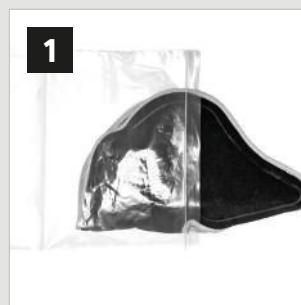

13


14

IMMOBILIZATION BRACE

15

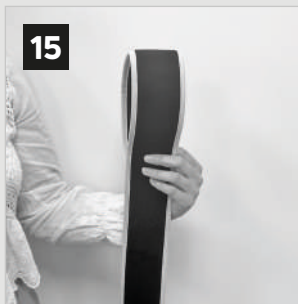16

17

18

ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER

The intended user should be a licensed healthcare professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

Postoperative care following shoulder surgery (prosthesis, rotator cuff repair). The cold therapy may help provide analgesic and anti-haemorrhagic effects and, reduce oedema.

PERFORMANCE CHARACTERISTICS:

- DonJoy Shoulder Ice devices achieve their intended use by providing support to the shoulder.. Providing cold therapy for a reduction oedema.

CONTRAINDICATIONS

Any known allergies to the medical device materials. For cold therapy: Sickie-cell Anemia, Raynaud's or other vasospastic diseases, cold allergy, cold hypersensitivity, previous cold injury, or any compromised local circulation, decreased skin sensitivity, hypertension, paroxysmal cold hemoglobinuria, and cryoglobulinemia.

WARNINGS AND PRECAUTIONS

- This medical device must be used according to your doctor's prescription or following the usage recommendations of a duly authorized healthcare professional (pharmacist, physiotherapist, orthotist, etc.).
- The manufacturer does not guarantee that the use of this product can prevent injuries. It is essential to check the completeness and integrity of the kit components before use.
- Do not use if damaged.
- Certain factors may increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or individuals with fragile skin); close medical supervision is recommended in such cases.
- The cold pack should never come into direct contact with the scar or the skin. Always use a cloth as a barrier.
- The cold pack must be flexible and free of frost before use.
- Do not heat or reheat the cold pack.
- Do not puncture the cold pack.
- In case of gel leakage: Handle the device with gloves. Do not ingest the gel. Do not pour the gel into a drainage system connected to the sewer or into a watercourse.
- Do not dispose of the device in nature or with household waste. It must be discarded as special industrial waste at a waste disposal site.
- If the gel comes into direct contact with the skin, rinse immediately and thoroughly with cold water.
- Contact your doctor if irritation persists.
- Keep out of reach of children and individuals unable to use this device properly.
- If you experience pain, swelling, unusual sensations, an inflammatory or allergic reaction, stop using the medical device immediately and consult a qualified healthcare professional (pharmacist, physiotherapist, orthotist, etc.) as soon as possible.
- Do not use this device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

BOX CONTENT

- 1 cold pack
- 1 skin protective textile
- 1 thoracic strap
- 1 freezer bag
- 1 arm strap
- 1 immobilization strap

APPLICATION INSTRUCTIONS

Cold Pack

1. Insert the cold pack into the dry freezer bag, then place the whole unit in the freezer.
2. After 8h of freezing (between -18 and -22°C), remove the cold pack and distribute the gel if necessary.
3. Position the cold pack on a flat surface, then place the lycra fabric over it.
4. Fold back each Velcro fastener and attach to the outside of the cold pack.
5. For a right shoulder (a), attach the chest strap to the right of the cold pack (r). For a left shoulder, proceed symmetrically (b).
6. Attach the arm strap to the left side of the red border (a), then attach it symmetrically to the right side (b) where the arrow is.
7. Fold the cold pack in on itself. The rolled-up arm strap should then be at the bottom.
8. Pass your arm through the arm strap (a), then bring the cold pack up to shoulder level (b).
9. Grab the chest strap on the back with the other hand.
10. Attach this strap to the cold pack (item r).
11. Adjust chest strap positioning, repositioning double-sided velcro if necessary.
12. Adjust the placement of the chest strap protection at armpit level.
13. Trim excess length of chest strap. Adjust arm strap if necessary.
14. After 20 to 30 minutes, remove the straps, replace the cold pack in the bag and return the unit to the freezer.

Immobilization Brace

15. Take the immobilization strap at the buckle. This will then be placed on the side of the shoulder to be immobilized at hip level.
16. Pass the immobilization strap over the opposite shoulder and wedge the buckle between the arm and hip.
17. Slide your arm inside the buckle, then grab the end of the strap and wrap it over your arm at wrist level.
18. Fix the Velcro so that the elbow is positioned at right angles.

CARE

Donjoy Shoulder Ice compression vest is hand-washable with mild soap. Do not machine wash or use bleach. Lay flat to air dry. Do not tumble dry or iron. The cold pack is hand-washable with mild soap. Do not machine wash or use bleach. Do not immerse in a disinfectant detergent bath. If necessary, spray with a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces). Lay flat to air dry.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

- Skin protective textile: Polyamide, Elastane
- Cold pack:
 - Gel: Water, Salt, Hydroxycellulose, Monopropylene glycol, Triethanolamine, Triarylmethane dye
 - Fabric: Polyamide, Polyurethane, Polyester, Cotton, Elastane
- Straps: Polyamide, Polyurethane, Polyester, Cotton, Elastane
- Plastic bag: Polyethylene
- Suitcase: Polyvinyl chloride, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

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