DONJOY®

KIT EXCELL ICE –
COLD THERAPY AND COMPRESSION
FOREFOOT

enovis™

7

8

9

ENGLISH

Before using the device, please read the following instructions completely and carefully. Correct application is vital to the proper functioning of the device.

INTENDED USER

The intended user should be a licensed healthcare professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

Postoperative care of forefoot surgery, sports traumatology such as sprains and tears, and rehabilitation. The cold therapy may help provide analgesic and anti-haemorrhagic effects and, reduce oedema.

PERFORMANCE CHARACTERISTICS:

- DonJoy Kit Excell Ice and Compression, Forefoot devices achieve their intended use by providing support and compression to the foot. Providing cold therapy for a reduction oedema.

CONTRAINDICATIONS

For cold therapy: Sickle-cell Anemia, Raynaud's or other vasospastic diseases, cold allergy, cold hypersensitivity, previous cold injury, or any compromised local circulation, decreased skin sensitivity, hypertension, paroxysmal cold hemoglobinuria, and cryoglobulinemia.

WARNINGS AND PRECAUTIONS

- This medical device must be used according to the medical prescription of your doctor or according to the recommendations for use of your duly authorized health professional (pharmacist, physiotherapist, orthotist, etc.).
- The manufacturer makes no warranty that use of this product will not prevent injury.
- It is imperative to check the completeness and integrity of the elements of the kit before use.
- Do not use if damaged.
- Factors can increase the risk of skin burns (malnourished patients, excessive smoking, insulin-dependent diabetics, or even people with fragile skin): close medical supervision is recommended.
- Cold pack should never be in direct contact with the scar or skin. Systematically use the hypoallergenic Lycra supplied. Cold pack should be soft and frost free before use.
- Do not heat or warm cold pack.
- Respect the number of pressures indicated to inflate the pack. If tingling occurs in the limb, remove and deflate the pack immediately.
- Do not pierce the cold pack, nor the manual pump and its connection pipe.
- In the event of gel leakage: handle the device with gloves, do not ingest the gel, do not pour the gel into a drainage system connected to the sewers, or into a watercourse.
- Do not throw the device in nature or with household waste. Disposal is done with special industrial waste, in the recycling center.
- In case of direct contact of the gel with the skin, rinse immediately and abundantly with cold water.
- Contact your doctor if irritation persists.
- Keep out of the reach of children and people unfit to use this device.
- In the event of pain, swelling, unusual sensations, inflammatory or allergic reaction, immediately stop using the medical device and consult your authorized healthcare professional as soon as possible (pharmacist, physiotherapist, orthotist, etc.), <https://www.orthonov.com/>
- Do not use this device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

BOX CONTENT

- 1 skin protective textile
- 1 manual pump
- 1 freezer bag
- 2 forefoot cold packs

APPLICATION INSTRUCTIONS

Cold Packs

1. Put on the lycra clothes. The cold pack should never be in direct contact with the skin.
2. After 8 hours of freezing (between -18 and -22 ° C), take out the cold pack and distribute the gel if necessary.
3. Place the cold pack on a flat surface. The rounded part should be at the heel.
4. Fold down the left part on the foot.
5. Fold down the right part of the cold pack on the left side and fix the Velcro.
6. Place the strap heel.
7. Go around the heel and fix the velcro.
8. Turn the tap to ON and screw on the pump. Inflate (do not exceed 10 pressures). Turn the tap to OFF and unscrew the pump.
9. After 20 min, turn the tap to the ON position and undo the straps. Wide and replace the cold pack in the bag and store in the freezer.

Do not walk with the pack installed.

CARE

Cold pack and Lycra are hand washable with mild soap. Do not machine wash and never use bleach. Do not immerse in a disinfectant detergent bath. If necessary, spray a disinfectant detergent solution (ANIOS disinfectant detergent for high surfaces). Always place the tap in position OFF. Leave to air dry and lay flat.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

COMPOSITION

- Skin protective textile: Polyamide, Elastane
- Cold pack:
 - Gel: Water, Salt, Hydroxycellulose, Monopropylene glycol, Triethanolamine, Triarylmethane dye
 - Fabric: Polyamide, Polyurethane, Polyester, Cotton, Elastane
- Pump kit: Silicone, Polyvinyl chloride, Polycarbonate
- Plastic bag: Polyethylene
- Suitcase: Polyvinyl chloride, Polypropylene

NOT MADE WITH NATURAL RUBBER LATEX

INTENDED FOR USE ON A SINGLE PATIENT

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: While every effort has been made in state-of-the-art techniques to obtain the maximum compatibility of function, strength, durability and comfort, there is no guarantee that injury will be prevented through the use of this product.

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