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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Neo Smart Carbon Rollator is a mobility aid designed for individuals with reduced mobility. Thanks to its ultra-lightweight carbon frame, it is easy to maneuver and perfectly suited for safe outdoor use. It is also equipped with a convenient seat for sitting and taking breaks.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 kg.
- The rollator should not be used by individuals with severe balance issues or insufficient muscle strength to stand and walk, even with support. A medical or physiotherapist's evaluation is essential.
- Ensure that all parts (wheels, cables, handles) are securely fastened and in good working condition.
- Do not use the rollator on stairs or escalators.
- Make sure the handles are neither loose nor damaged.
- Use only accessories and parts provided or approved by the manufacturer.
- The rollator is a walking aid. It is not designed to transport a seated person or to be self-propelled while seated.
- Use the rollator only as a walking aid and never while walking backward.
- Do not make any adjustments while using the rollator.
- Check the brakes for proper function before each use.

- The brakes must be locked before sitting down or during prolonged stops.
- All wheels must remain in contact with the ground during use.
- Hold the handles with both hands to ensure good stability.
- Use the rollator only on flat, stable, and dry surfaces, either indoors or outdoors.
- Avoid sloped, uneven, slippery, wet, icy, or snowy surfaces.
- Be especially careful when going up or down curbs.
- If any part is damaged, broken, or worn, contact your retailer.
- Be cautious of the risk of pinching when folding the rollator.
- The rollator bag has a maximum load limit of 5 kg.
- Use the rollator only for its intended purpose.
- Do not modify the rollator without written authorization from the manufacturer.
- When making adjustments, be careful not to pinch your fingers or any other body part between moving parts.

DESCRIPTION OF THE PRODUCT


MOUNTING & USE

MOUNTING


Step 1 :

Insert the handles into the frame tubes, then secure them using the provided screws and the Allen key, making sure to tighten them firmly.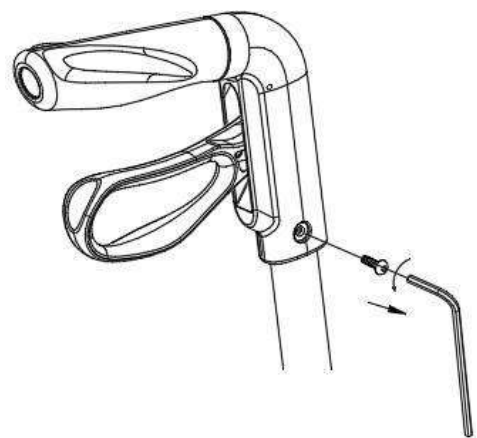

Step 2 :

Attach the backrest by inserting the ends into the brackets located at the back of the handles, then push firmly until it clicks into place.

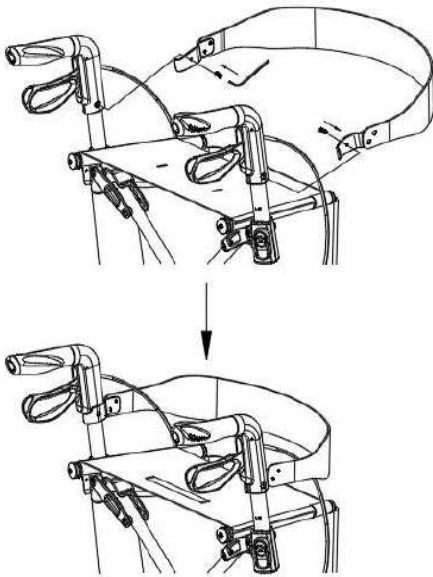Unfolding the Rollator

1. Unfasten the elastic strap of the locking system.
2. Grasp the handles with both hands and pull them apart to each side.
3. Press down on the seat cushion support tubes on both sides until you hear the locking mechanism click into place.

- After unfolding or assembling, always ensure that the seat support is securely locked and that the frame no longer moves.
- Always check that the handle tubes are firmly secured into the main support tube using the locking button to prevent them from slipping under pressure.
- Always ensure that all accessories are correctly and securely attached before use.

Folding the Rollator

1. Adjust the support rods on both sides to the lowest position.
2. Pull the locking strap located at the center of the seat cushion until the left and right frames are fully folded.
3. Secure the seat support tubes on each side using the elastic bands.

Adjusting the Height of the Push Handles

1. Unlock the handle locking button.
2. Adjust to the desired height.

- Both push handles must be set to the same height.
- The correct handle height should correspond to the user's hip level.
- Never leave the rollator without applying the brakes.

USE


Initial Inspection :

- First, take the rollator out of its packaging and check (using the image below) that no parts are missing.
- If you find any missing or damaged items at this stage, do not proceed further with assembling the product. Contact your supplier immediately.

Complete packaging includes:

- 1 seat, 1 shopping bag, 1 manual, 1 main support mechanism

Using the Hand Brakes :

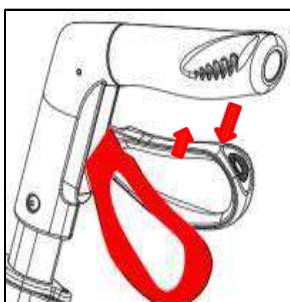
1. TO STOP THE ROLLATOR, simply pull the brake handles toward the push handles.
2. TO REMAIN STATIONARY, keep the brake handles in the "down" position.
3. TO START MOVING, release the brake handles. They will automatically lower, and the brakes will be disengaged.

- You should always engage the brakes when you do not plan to move the rollator.
- Never leave the rollator without the brakes engaged when someone is sitting on it!
- Never leave the rollator on a slope without the brakes engaged!
- Always sit up straight when seated on the rollator, with your forearms or hands resting lightly on the push handles.
- Never lean backward while sitting on the rollator.

Locking/Unlocking the Hand Brakes :

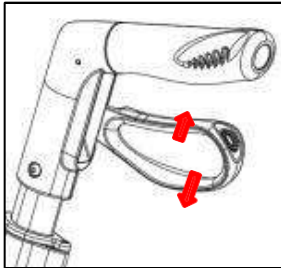1. To lock the brakes, push the brake handles down until you hear a "click."
2. To unlock the brakes, pull the brake handles up until you feel them release.

When using the rollator in a stationary position, the hand brakes MUST always be locked.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regularly clean the rollator with lukewarm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth.
- Make sure there is no dirt or debris buildup around the brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as these can damage the aluminum and polyamide materials.
- Lubricate the wheels with a small amount of oil if they start to squeak.
- Periodically check the wear and tightness of all screws and bolts. Replace any broken, damaged, or worn parts immediately.
- Ensure that every part of the rollator is securely fastened at ALL times.
- Check that all screws are properly tightened. If not, tighten them as necessary.
- In case of malfunction, do not attempt to repair the rollator yourself. Consult an authorized dealer.

Checklist :

To keep your walker in good condition, check before each use and regularly as follows.

Check the item	Anytime	Weekly	Monthly
Condition and operation of the brakes	X		
Condition and operation of the wheels		X	
Overall condition of the product		X	
Cleanliness	X		
Screws and fasteners		X	
Folding mechanism		X	
Frame stability			X

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for extended periods near a heat source or in direct sunlight (e.g., behind a window or near a radiator), or near a cold source.
- Keep away from flames and sparks.
- Follow the recommended conditions for storing your product.
- Store in a dry and temperate place.
- Protect your product from dust and corrosion (e.g., abrasive materials, sand, seawater, salty air) by using appropriate packaging.
- Store all removed parts together in the same place (or label them if necessary) to avoid mixing them with other products during reassembly.
- All components should be stored without any load (do not place heavy objects on product parts, do not pinch anything...).
- The rollator is an easily foldable product, convenient for transport. If transportation is necessary, we recommend folding it as described above. The rollator must be transported securely, without risk of sudden or dangerous movements.

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Length	65 cm
Width (open)	62 cm
Total height (folded, at lowest point)	82 cm
Width (folded)	22 cm
Handle height	L Size : 78 to 98 cm M Size : 70 to 90 cm
Seat dimensions	Width: 46 cm Height: 56 cm (M Size) or 62 cm (L Size)
Maximum user weight	136 kg
Weight	5.8 kg with bag and backrest

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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