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## *USER MANUAL*

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## **INTRODUCTION:**

This is an electrical Lift chair designed to help people to comfortably rise up and sit-down on the chair.

Please read the operation manual thoroughly for the proper chair's usage, especially the notes on Safety.

This manual should be kept available for future reference after you have read it through.

## **SPECIFICATIONS:**

SEAT WIDTH	50 cm	Power: 100-240V // 50Hz – 40W
SEAT DEPTH	50 cm	
SEAT HEIGHT	48 cm	
BACKREST HEIGHT	71 cm	
CHAIR WEIGHT	53 kg	
LOADING CAPACITY	135 kg	

## **CONTENT:**

### **1. SAFETY**

### **2. ASSEMBLY**

### **3. PLACING OF THE CHAIR**

### **4. PRODUCT CARE AND STORAGE**

## **1. SAFETY:**

### **a. Warning**

- The chair must be assembled correctly according to the assembly instruction (enclosed). The manufacturer of the chair, their agents, and distributors are not liable for any injuries or damages may be caused by improper assembly or usage of the chair.
- The chair must be placed on a flat floor level.
- Do not use your Chair in humid or dusty environment, as it may cause electrical shock or machine trouble. The chair must be in a dry room.
- Do not use your Chair in room temperature higher than 40 degrees Celsius/104 Fahrenheit. Do not expose your Chair to heaters, stoves, or direct sunlight.
- Do not use the Chair outdoor.
- When the Chair is not in use, disconnect the power cord from the main supply to avoid overheating.
- Children must be supervised by an adult when using the lift Chair. This Chair is not intended for children.
- Check that the voltage corresponds with the specifications indicated for the chair.
- Electrical connection must be made in accordance with local regulation.
- Do not connect or disconnect the plug from the electrical outlet with wet hands. It may result in electrical shock or machine troubles.
- Do not place the AC/DC adaptor cord under the Chair as it may damage the cord and result in fire or electrical shock. Do not let the AC/DC cord hang down from the chair.
- Do not use the Chair if the AC/DC cord or plug is damaged, or if the electrical outlet is excessively loose. Contact the professional for inspection.
- If necessary, the power cable may be replaced with an identical one, which must only be replaced by a qualified technician.
- Do not touch the surface of the plug immediately after use, as it can be very hot.
- Do not operate the remote control with wet hands, sharp or heavy objects.
- Do not drop the remote control.
- Do not step or place heavy objects on the remote control wire.
- While operating the Chair, keep your fingers and hands away from the seat and the footrest, i.e., when the chair is reclining or when the chair is returning back from the recline position to sit position.
- This Chair must be used in accordance with the instructions provided.
- The manufacturers of this chair, their agents and distributors are not liable for any damages to person(s) or property(s) resulting from improper and misuse of the Chair.

**Note: All repairing must be done by qualified technician or the After Sales Service.**

#### **Caution**

Should you use the Chair in a cold room, do not abruptly increase the room temperature. It is recommended that you increase temperature gradually to a normal level.

In the event that the Chair is stored in a cold place and is being brought into a warm place, it is advisable to use it about an hour thereafter.

**b. You should NOT use the Chair If:**

- The Chair is accidentally spilled with water. This may result in electrical shock.
- The upholstery is torn and the inside mechanism are exposed.
- You detect any main function or other abnormal condition during operation.
- There is a power failure; injury may be caused when power is restored unexpectedly.
- There is severe lightning

In the above events, please switch the power OFF immediately and disconnect the AC power cord from the electrical outlet. Contact a professional for service.

Note: Check for tears to the Upholstery Cloth regularly

**2. ASSEMBLY:**

The chair is packed in semi knock down form consisting of the back and the seat. Assemble the chair according to the instruction given in the Assembly Instruction.


**Assembly Instructions:**


Note: Do not assemble the chair if the Upholstery or any of the parts of the chairs are damaged. Check all parts carefully before assembling the chair. If any problem, return to the vendor. Once the chair is assembled as per Assembly instruction, place the chair on a flat surface in a dry room. The chair is now ready for using.

Note: Make sure all the wires are properly connected, if not the will not function properly.

- (1). Connector for the motor and wiring
- (2). Connector for wiring and adaptor
- (3). Connector for adaptor wiring and plug wiring
- (4). Connector for the motor and the hand control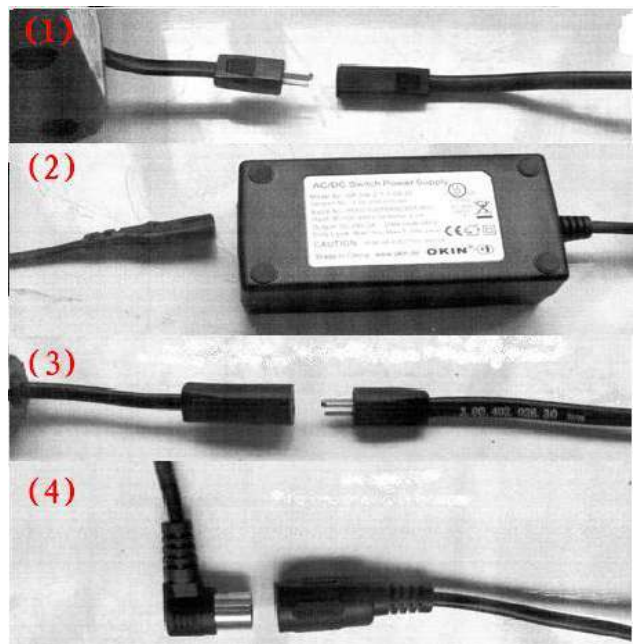

### **3. PLACING OF THE CHAIR :**

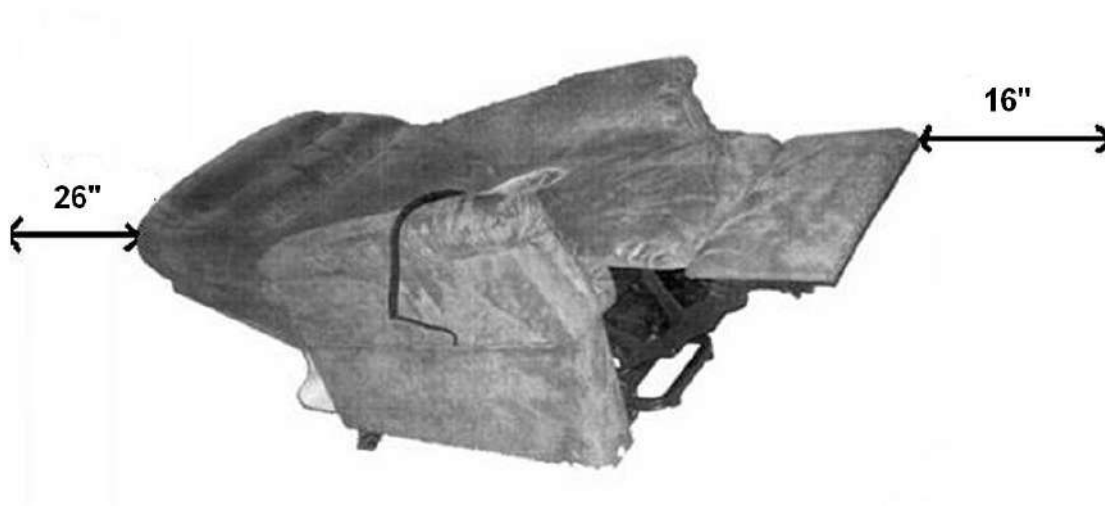- Allow sufficient space in front of and behind the chair for reclining purpose.
- Make sure there is an allowance of more than 26in behind the Chair and more than 16in in front of the Chair.
- Place the chair on a flat surface/floor in a dry room.

#### **Caution:**

- Make sure there is no obstruction behind or in front of the Chair for a safe recline and lift during operation.
- No objects should be placed between the Chair and the Footrest.

### **4. PRODUCT CARE AND STORAGE :**

#### **1. How to store**

- Wipe the dust and dirt off. Keep the Chair clean and away from places of high humidity
- If the Chair is not used for a long period of time, cover it with a piece of clean cloth to protect it from dust. Store in a dry room.
- Do not store the Chair under direct sunlight or in places of high temperatures.

#### **2. Self-maintenance**

- Before cleaning, ensure that the Main Switch is turned off and AC plug is removed for the electrical outlet.

#### **3. Main Body**

- Synthetic and very mild soap can be used to clean the plastic, pipe and upholstery of the Chair. Carefully wipe away all traces of the soap with dry cloth.
- To clean a dry spot, dip a piece of soft cloth into some warm soap water, wring the cloth dry, and apply it to the spot.
- Do not use cleaning agents like benzene or thinner, or any other cleaners which may result in fading of fabric color and scratching of upholstery.

- Do not remove the upholstery for washing. Ironing of upholstery is strictly prohibited

#### Remote Controller Unit

- Use only dry cloth for cleaning of the unit, as usage of wet cloth may result in machine trouble.

#### Remote control for lift and recline used only

1. Footrest Back & Lift Up
2. Footrest Open
3. Backrest Up
4. Backrest Down
5. Lift Up & Backrest Up
6. Recline & Footrest Open

#### Cautions:

- If the circuit (controller) doesn't work, please contact the supplier.
- Don't try to repair the Chair; please call for professional service