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EN

## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

## PRODUCT DESTINATION

The foldable two-wheel adjustable walker is a walking aid designed for people with reduced mobility. This walker is intended for indoor use and can be folded to facilitate storage or transport. It can also be used outdoors on flat surfaces.

## SAFETY PRECAUTIONS


Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 136 Kg.
- Make sure that all locking screws and parts are securely tightened, in good working condition, and that the height adjustment buttons are fully engaged.
- The walker must be fully opened on both sides before each use.
- Properly insert the locking pin into one of the eight adjustment holes and lock it securely.
- Never use the walker if it is partially opened or if you are unsure how to use it correctly.
- All legs, tips, or wheels must always remain in contact with the ground.
- Place your hands on both handles to ensure good stability.
- Do not use the walker to walk backwards, climb stairs, or on escalators.
- Avoid wet, icy, snowy, sloped, slippery, or uneven surfaces.
- Be especially careful when stepping up onto sidewalks or when walking on rough terrain: improper use may result in falls or injuries.
- The walker is a walking aid, not a wheelchair: do not sit on it, as this may damage the frame or the wheels.
- Do not make any adjustments while using the walker.
- Only use accessories provided by the manufacturer.
- In case of damaged, broken, or worn parts, contact your dealer immediately.

## ASSEMBLY

### Parts List :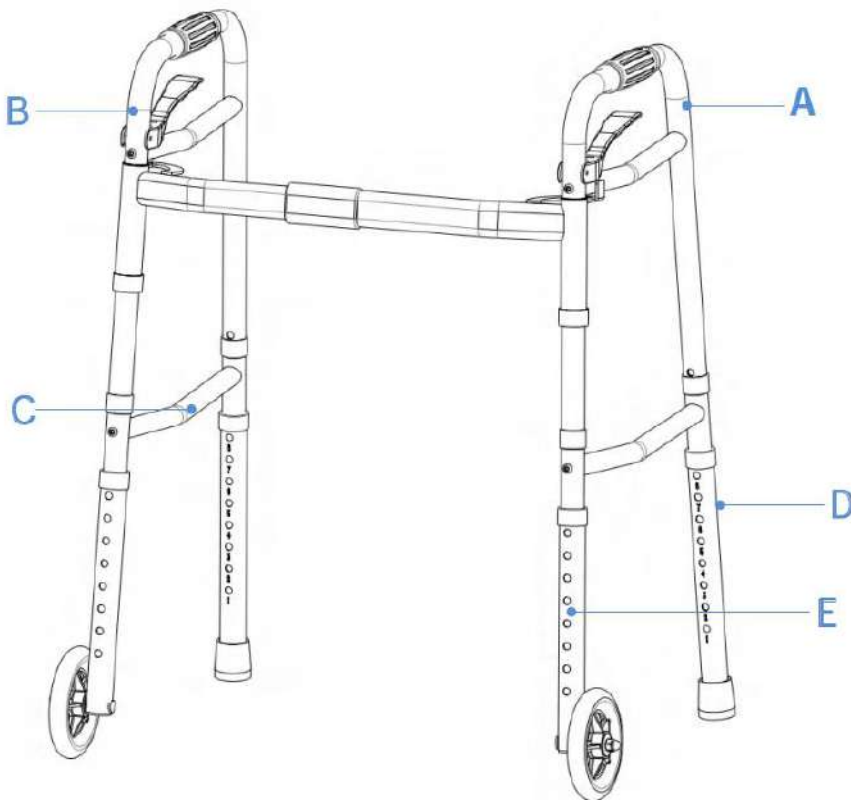


- A: Main frame (x1)
- B: Main frame (x1)
- C: Side bar with screw (x2)
- D: Rear leg with tip (x2)
- E: Front leg with wheel (x2)

Carefully remove all components from the carton. Before use, check that the product is free of any defects. A healthcare professional has assessed that the product provides the necessary support and that its settings are appropriate for the user.

### **Step 1 :**

- Assemble the main frames A and B together. You will hear a clicking sound if the assembly is done correctly.  
Note: Before assembly, make sure to activate the locking latch on frame B.

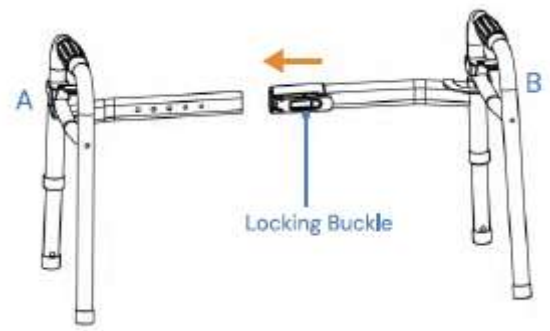
### **Step 2 :**

- Insert the front legs with wheels (E) as shown in the diagram. Press to engage the locking ball into its housing. A clicking sound will confirm that the operation has been successfully completed.



### **Step 3 :**

- Insert the rear legs (D) as shown in the diagram. Press the locking ball to secure it in place. A clicking sound will confirm that the operation has been successfully completed.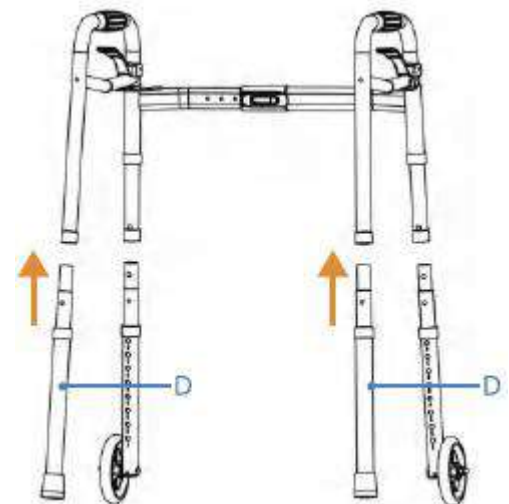

### **Step 4 :**


- Unscrew the screws on the side bar, then install bar C in the position indicated on the diagram. Tighten the screws on both sides using the L-shaped wrench.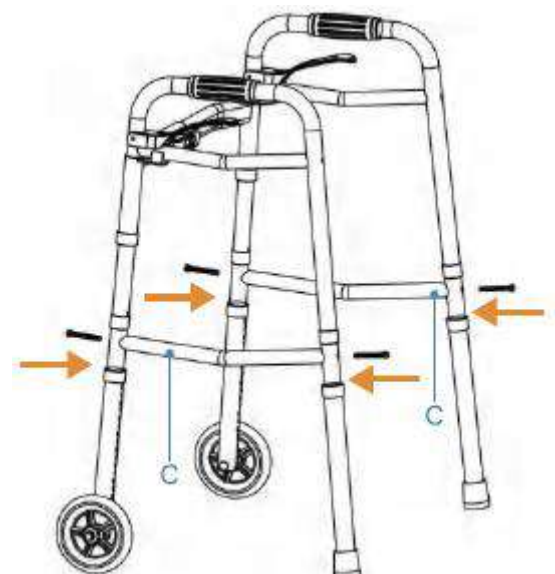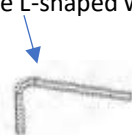

**FOLDING :**


**Step 1 :**

- Pull up on the release handle.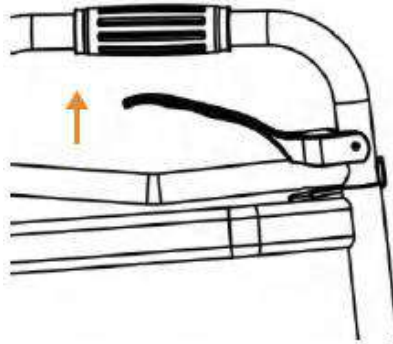

**Step 2 :**

- Fold the frame inward to close it.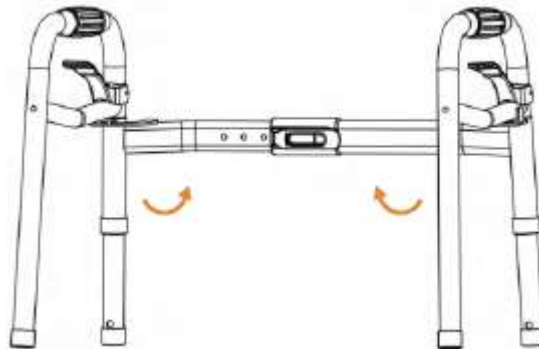

**HEIGHT ADJUSTMENT :**


**Step 1 :**

- Press the push button into the holes and rotate the leg. Adjust the leg height to your preference. A clicking sound will confirm that the operation has been successfully completed.

**Step 2 :**

- Adjust the width to the desired position.  
Note: Open the locking latch to 90° before adjusting the width. Once you hear the click, make sure the width is correctly set, then close the latch to secure the assembly.

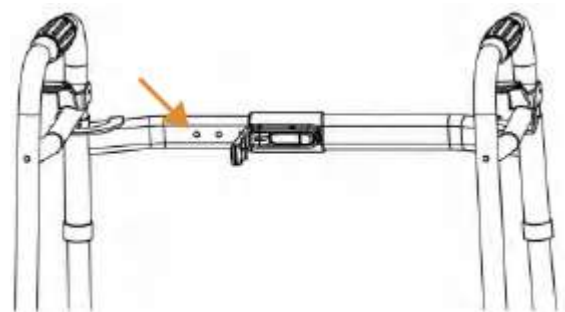## CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regularly clean the walking frame with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based products.
- Dry with a clean, soft, dry cloth.
- Ensure there is no accumulation of dirt or debris under the glides and wheels.
- Do not use cleaning products containing phenol or chlorine, as they may damage the aluminum and polyamide materials.
- Periodically check for wear and the tightness of all screws and bolts. Immediately replace any broken, damaged, or worn parts.
- In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.
- Checklist :

To keep the product in good condition, check before each use and inspect regularly as follows :

Check the product	Anytime	Weekly
Check the condition of the adjustment buttons	X	
Check the folding mechanism		X
Check the condition of the legs and handles		X
Check the wheels	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorized specialist.

## RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

## STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

## PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 57 to 69 cm (5 positions) x Depth 44.5 cm x Height 77.5 to 95 cm (8 positions)
Width between handles	39.5 to 49.5 cm
Frame	Aluminium
Wheels	PVC 5" tires
Maximum weight capacity	136 kg
Weight	2,6 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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