**REHAUSSE AVEC ACCOUDOIRS TAHITI***Notice d'utilisation***DISPOSITIF MEDICAL****TAHITI TOILET SEAT RAISER WITH ARMRESTS***Instructions manual***MEDICAL DEVICE****ELEVADOR CON APOYABRAZOS TAHITI***Manual de instrucciones***DISPOSITIVO MEDICO****REF. 811196***Scan this QR code to access the manual in multiple languages:*

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions, or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Tahiti toilet seat riser with armrests are toileting aids designed for people with reduced mobility. They attach to the toilet bowl and are intended for home use.

PRODUCT DESCRIPTION

The product is equipped with 2 mounting screws for the armrests, 2 legs, and 2 fastening screws for the base of the riser.

SAFETY PRECAUTIONS


Failure to follow the safety instructions may result in serious injury.

- Do not exceed the maximum weight limit of 100 kg.
- Use only accessories and parts supplied by the manufacturer.
- Visually check the overall condition of the product (armrests, toilet seat, riser).
- Make sure that all parts are properly secured and in good working order.
- Ensure that the product is correctly attached to the toilet bowl.
- Do not make any adjustments to the product while it is in use.
- Do not use the toilet armrest on stairs.
- Always place both hands on the handles to ensure the toilet armrest remains in a stable, parallel position.
- Do not use the toilet armrest as a step or climbing aid.

- Always lean on the toilet armrest when using it.
- In case of damaged, broken, or worn parts, contact your dealer

ASSEMBLY AND USE

ASSEMBLY :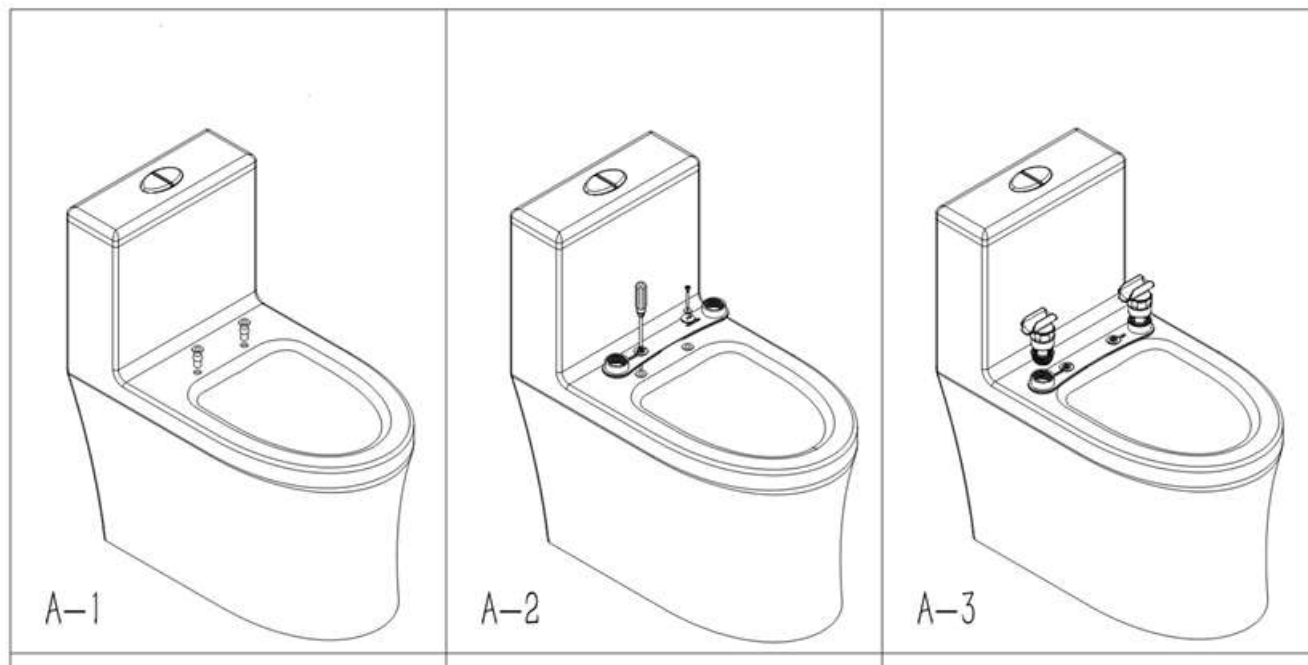

A-1 :


- Position the two fastening screws of the base into the holes provided at the back of the toilet bowl.
- Check that they are straight and properly aligned.

A-2 :

- Place the washers on the screws to ensure a proper seal.
- Tighten them with a screwdriver to securely hold the fastenings in place.

A-3 :

- Screw the two tightening nuts over the washers.
- Check that the fastenings are stable and firmly secured, without any play.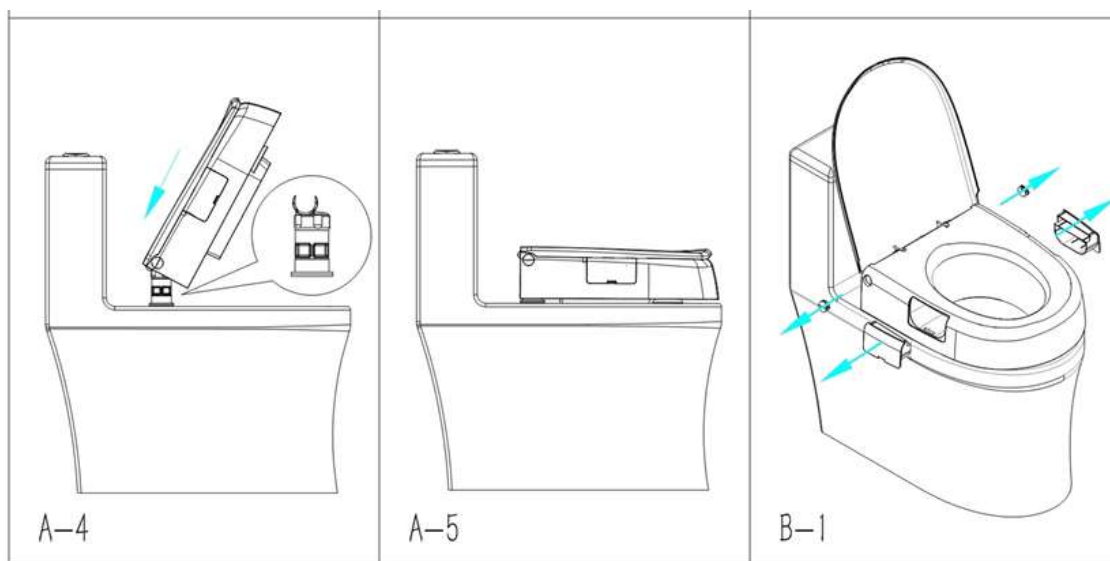

A-4 :


- Position the toilet seat (lid and seat) above the fittings.
- Align the hinges with the fixing rods and push them firmly into place until they “click”.

A-5 :

- Gently lower the toilet seat to check that it is securely attached and properly centered.
- The system must be stable and should not move sideways.

B-1 :

- Close the lid and the seat to check the alignment.
- If necessary, adjust the position by slightly loosening the screws, then retighten them.
- Finally, snap the protective caps onto the fittings.

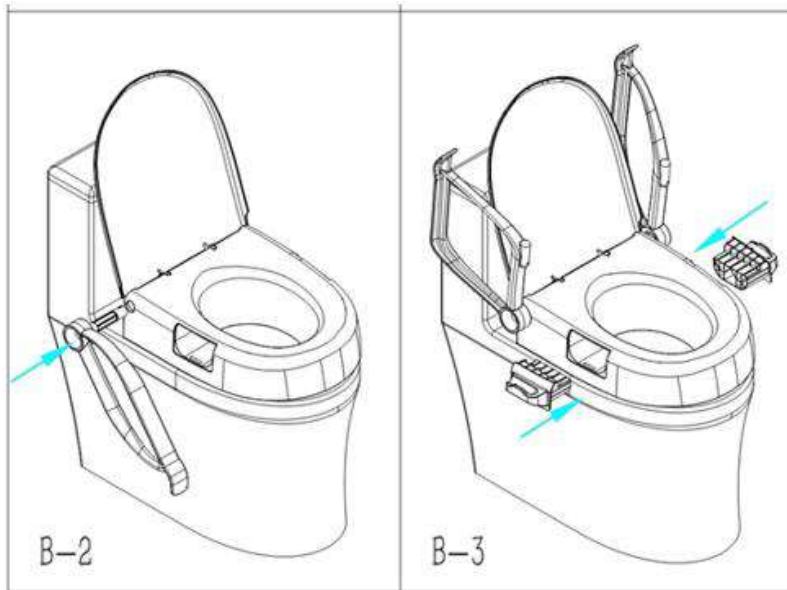B-2 :

- Attach the armrest on the designated side.
- Slide it into the slot until it locks into place.

B-3 :

- Repeat the operation for the second armrest.
- Check that both fittings are properly locked and aligned.

B-4 : The armrests can be attached to the protective covers using the screws provided if you wish to make them fixed.

- Attach the armrests to the protective covers using the screws provided.
- Check that the seat and lid can still open and close without obstruction.

USE :

To use the product, lean on the armrests for support and sit down gently.

ENTRETIEN - MAINTENANCE

Failure to maintain and service your product may result in a risk of serious injury.

- Regularly clean the product (all fittings) with a damp cloth and a disinfectant. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth.
- It is recommended to service the product at least once a year. If the product is subject to intensive use and/or used in a multi-user environment, we recommend having it serviced more frequently.
- In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.

- **Checklist:**

To keep the product in good condition, check it before each use, and carry out regular checks as follows :

Check the item	Anytime	Weekly
Condition and operation of the armrests	X	
Cleanliness		X
Screws, fastening elements		X
Stability of the product	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable, and you must throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place.
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Seat dimensions	Width 38 x depth 44 cm
Seat height relative to the toilet bowl	+7 cm or +11 cm
Width between the armrests	50 cm
Maximum authorized weight	100 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design, or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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