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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise you on the most appropriate use for you.

Distributor: This manual must be given to users of this product.

## PRODUCT DESTINATION

The Soft bath access chair is a medical device designed for people with reduced mobility. It allows you to sit comfortably outside your bathtub and slide into it without having to stand up. It is specially designed for individuals who have difficulty stepping over a bathtub, providing safety, comfort, and independence during bathing.

## PRODUCT DESCRIPTION

1. Lift-up foam armrest
2. Seat
3. Height-adjustable leg tube
4. Non-slip tip
5. Backrest
6. Rotating system

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in serious injury.

- Do not exceed the maximum permitted weight limit of 136 kg.
- Ensure that the product is in proper working condition.
- Check that all parts are securely locked in place.
- Make sure the user locks the latch that enables the seat transfer.
- In case of damaged, broken, worn, or missing parts, contact the retailer.
- Ensure that the bar connecting the four legs is perfectly horizontal.

- Make sure that the locking pins on all four legs are fully engaged.
- Before each use, check the grip on the bottom of the bathtub and ensure that all four legs are stable.
- All components must be inspected before use to ensure they are not damaged and are properly secured.
- Approved for indoor use only.
- Be cautious of the risk of pinching clothing or body parts.
- Prevent any unauthorized use, especially by children.

## ASSEMBLY AND USE

### Step 1 :

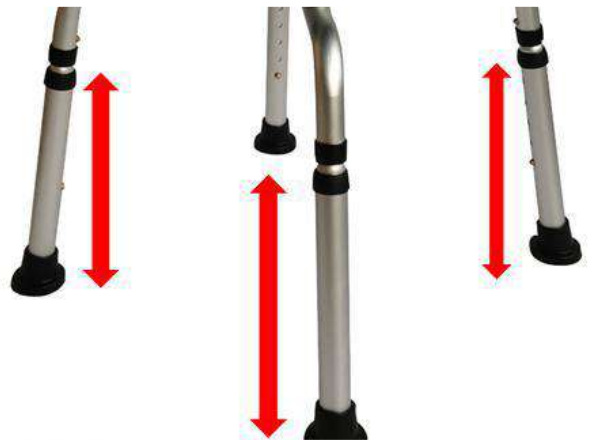
Attach the backrest to the frame by pressing the push buttons and inserting the backrest tubes into the supports located on each side and at the back of the seat.

### Step 2 :

Attach the legs to the frame by pressing the push buttons and inserting them into the tubes located on each side of the frame.

Make sure the push buttons lock properly into the holes, that the legs are stable, and that they are adjusted to the same height.

### Step 3 :

Using the armrests, press the button and lift the armrests backward.

**Step 4 :**

1. Pull the lever and rotate the seat to position it facing the user, then lock the seat by lowering the lever and continue rotating the seat until it clicks into the locked position.
2. Unlock the seat by pulling the lever.
3. When the user is seated, lock the seat and ensure it is securely fixed in position and no longer rotates freely.

**Step 5 :**

By pulling the red lever to activate the sliding function.

**MAINTENANCE – CARE INSTRUCTIONS**

Failure to properly maintain and service your product may result in serious injury.

- Clean the product regularly with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based cleaners.
- Dry with a clean, soft, dry cloth.
- In case of malfunction, do not attempt to repair the product yourself. Please contact an authorized dealer.
- Checklist :

To keep the product in good condition, check before each use and inspect regularly as follows:

Inspect the item	Anytime	Weekly
Check that the seat slides smoothly along the structure.	X	
Check the condition of the seat.		X
Check the proper functioning of the sliding lock system.	X	
Check the proper functioning of the four height adjustment clips.	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, make sure to provide all the necessary technical documents to the new user. Prior to this, the product must be inspected by an authorized specialist..

## RECYCLING

When the product becomes unusable and must be disposed of, please contact your specialized dealer. If you wish to recycle the product yourself, please consult your local waste management company for disposal guidelines.

## STORAGE

Failure to comply with storage conditions may lead to product deterioration and therefore poses a risk of serious injury.

Do not store your product for extended periods near a heat source or in direct sunlight (e.g., behind a window or near a radiator) or near a cold source.

Keep away from open flames and sources of sparks.

Follow the proper conditions for storing your product:

- Store in a dry, temperate place.
- Protect your product with appropriate packaging against dust, corrosion (e.g., abrasive elements, sand, seawater, salty air).
- Store all removed parts together, in the same place (or label them if necessary) to avoid mixing them with other products during reassembly.
- All components must be stored without any load (do not place heavy objects on product parts or wedge anything against them).

## USAGE DURATION

The expected service life of this product is 5 years under normal conditions of use, safety, and maintenance. Beyond this period, the product may continue to be used as long as it remains in good condition.

## TECHNICAL SPECIFICATIONS

Overall dimensions	Width 85/88 cm x Depth 50.5/56.5 cm x Height 89.4/108.1 cm
Seat dimensions	Width 40.5 cm x Depth 33.5 cm x Thickness 5.1 cm
Anti-corrosion aluminum frame	
Weight	8,1 kg
Maximum supported weight	136 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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