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## IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise you on the most appropriate use for you.

Distributor: This manual must be given to users of this product.

## PRODUCT DESTINATION

The Soft swivel shower chair is a medical device designed for people with reduced mobility. It allows the user to sit comfortably and swivel 360° in the bathroom to easily position themselves in front of the taps.

## PRODUCT DESCRIPTION

## SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in a risk of serious injury.

- Do not exceed the maximum authorized weight limit of 135 kg.
- Ensure that the product is in good working condition.
- Check that all parts are securely locked in place.
- Make sure the user locks the lever that allows the seat to swivel.
- In case of damaged, broken, worn, or missing parts, contact the retailer.
- Make sure that the locking pins on all four legs are properly engaged.
- Before each use, check the grip and stability of all four legs.

- All components must be inspected before use to ensure they are not damaged and are properly secured.
- Approved for indoor use only.
- Beware of the risk of pinching clothing or body parts.
- Prevent any unauthorized use, especially by children.

## ASSEMBLY AND USE

### **Step 1 :**

Attach the backrest to the frame by pressing the push buttons and sliding the backrest tubes into the supports located on each side and at the back of the seat.

### **Step 2 :**

Attach the legs to the frame by pressing the push buttons into the tubes located on each side of the legs. Make sure the push buttons lock properly into the holes, and ensure that the legs are stable and adjusted to the same height.

### **Step 3 :**

Check stability; the seat must not be unstable or wobbly.

1. Rotate the seat towards the user : Lift the lever and swivel the seat to position it facing the user. Then lock the seat by lowering the lever and continue to swivel the seat until it locks securely in place.
2. Unlock the seat: Lift the lever to unlock the seat.
3. Pre-use check: When the user is seated, lock the seat and ensure it is securely fixed and no longer rotates freely.
4. Safe position: Lock the chair when the user is facing the open side of the bathtub. The user can then stand up carefully and slowly.
5. Using the armrests, press the button and lift the armrests backward.

## MAINTENANCE – CARE INSTRUCTIONS

Failure to properly maintain and service your rollator may result in a risk of serious injury.

- Regularly clean the product with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based products.
- Dry with a clean, soft, dry cloth.
- In case of malfunction, do not attempt to repair the product yourself. Contact an authorized dealer.
- Checklist:

To keep the product in good condition, check before each use and inspect regularly as follows:

Check the item	Anytime	Weekly
Ensure that the chair rotates properly on the frame.	X	
Check the condition of the seat.		X
Check the proper functioning of the swivel locking system.	X	
Check the proper functioning of the four height adjustment clips.	X	

## CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene guidelines. When transferring the product, make sure to provide all the necessary technical documents to the new user. Prior to this, the product must be inspected by an authorized specialist

## RECYCLING

When the product becomes unusable and needs to be disposed of, please contact your specialized dealer. If you wish to handle the recycling yourself, please consult your local waste management company for disposal guidelines.

## STORAGE

Failure to comply with storage conditions may lead to product deterioration and serious injury risks.

Do not store your product for an extended period near a heat source or in direct sunlight (e.g., behind a window or near a radiator), or near a cold source.

Keep away from open flames and sources of sparks.

Follow the correct storage and handling conditions for your product.

- Store in a dry, temperate location.
- Protect your product with appropriate packaging from dust and corrosion (e.g., abrasive elements, sand, seawater, salty air).
- Store all removed parts together, in the same place (or label them if necessary) to avoid mixing them with parts from other products during reassembly.
- All components must be stored without any load (do not place heavy objects on the product parts, do not jam or squeeze them)

## DURATION OF USE

The expected service life of this product is 5 years under normal conditions of use, safety, and maintenance. Beyond this period, the product may continue to be used as long as it remains in good condition.

## TECHNICAL SPECIFICATIONS

Overall Dimensions	Width: 50.6 cm x Depth: 47.5/50 cm x Height: 81.2/91.4 cm
Seat Dimensions	Width: 40.7 cm x Depth: 33.7 cm x Thickness: 5.2 cm
Anti-Corrosion Aluminum Frame	
Weight	5,4 kg
Maximum Supported Weight	135 kg

## GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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