**CHAISE DE DOUCHE/WC ASTON
PRO TILT**

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DISPOSITIF MEDICAL

**ASTON PRO TILT SHOWER/WC
CHAIR**

User manual

MEDICAL DEVICE

**SILLA DE DUCHA/INODORO
ASTON PRO TILT**

Manual de instrucciones

DISPOSITIVO MÉDICO

REF. 822070

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USER MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

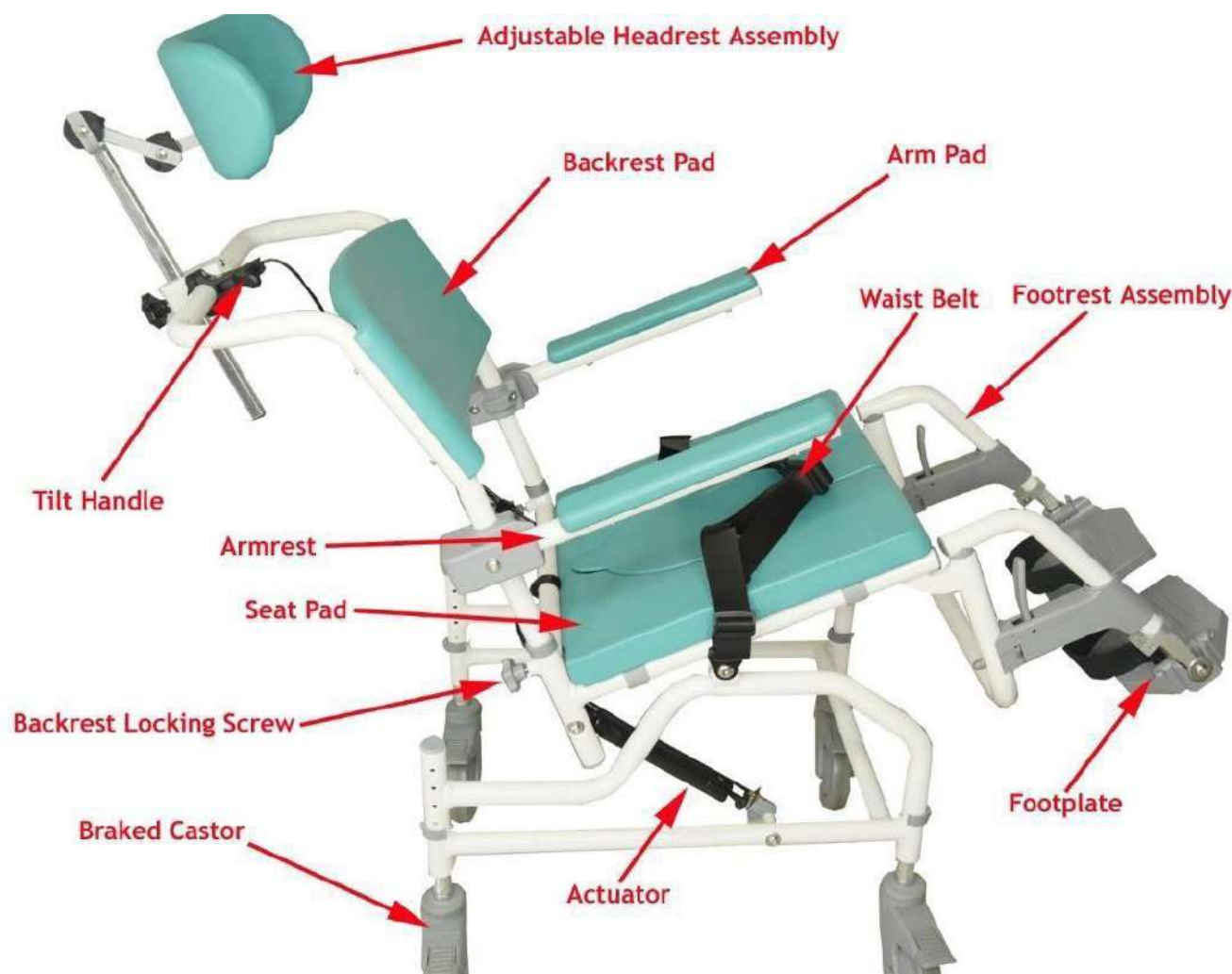
Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Aston Pro Tilt shower/toilet chair is a medical device. It is a technical aid for showering or toileting that is designed for people who are elderly, have reduced mobility, are disabled or injured, or who can easily lose their balance.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Never exceed the maximum supported weight of 160 kg for references
- Check all components are secured before use.
- Make sure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and wheels should be checked.
- Check the stability of the chair before use.
- Only use on level ground.
- When stationery, apply the brakes on the wheels.
- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary, and the brakes are applied when transferring to and from the device.
- Do not lean out of the device.
- Check the wheels bearings the brakes.
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

First attach the upper section of the frame to the lower section of the frame. To do this, remove the bolt, washers & nut from each side of the lower frame section, which are loosely fitted into tabs on the top of each side of the lower section.

Next, place the upper section of the frame onto the lower section, ensuring that the tabs align correctly on top & bottom halves. Replace the bolt, washers & nut into the tabs, locking the two halves together.

(See photo below with fixings shown by a red arrow.) **Ensure** that there is a nylon washer between the two tabs on each side. **DO NOT** overtighten or this will prevent the chair from tipping when required.

Please also ensure that the actuators are correctly fitted to both sections of the chair.

➤ Backrest Attachment

Push the backrest tubes into the sockets just behind the seat pad on the upper section of the frame & tighten the Backrest locking screws until the screws become tight. (The locking screws may need to be removed from the frame before the tubes can be inserted into the sockets.)

➤ **Footrest Attachment**

Attach the footrests by placing the top bracket into the receiver tube at the top of the lower frame section, ensuring that the footrest is swung away from the front of the frame. Swing the footrest towards the front of the chair until it locks into position.

Repeat for the other side of the chair.

➤ **Seat Pad Attachment**

Locate the seat pad by sliding the back of the seat pad under the tube at the rear of the frame. Press down on the seat pad until all four clips lock into position on the frame. Add the aperture cover if required.

➤ **Footrest Operation and Adjustment**

The footrests can swing in & out to allow the user access & egress from the chair. To swing the footrests out, firstly, flip the footplate up & then press the lever towards the chair & then rotate the footrest outwards.

Once the user is in the chair, then rotate the Footrests back to the front position, ensuring that they lock in the closed position securely. Once the footrest is locked in position, the Footplate can be flipped back down to allow the user to rest their foot.

➤ **Wheel Adjustment**

The castors can be locked so that the castor wheel cannot move forwards or reverse, nor will the castor wheel swivel. To lock the brakes, press down on the locking tab as shown on the left. To unlock the brake, press on the section of the tab that has been raised.

➤ **Wheel Adjustment**

The height of the chair can be adjusted by adjusting the castor height.

To adjust the castor height, remove the 'E' Clip, reposition the castor stem to the desired height & then replace the 'E' Clip back into the tube.

Perform the same process for each of the other three castors, ensuring that they are all at the same height.

➤ **Potty Installation and Removal**

The potty is installed and removed by sliding the potty in to the two rails mounted to the underneath of the seat. To install, slide the potty underneath the seat from the rear of the chair, ensuring the potty sits on both rails of the seat. Ensure nobody is sat on the chair when installing or removing the potty to prevent any trapping risk. Cover the potty with the lid to reduce the risk of spillages and carefully dispose of the potty contents in a sluice or WC. Do not dispose of in an open drain or sink as this can cause serious health risks. The potty can be cleaned and disinfected manually or in an autoclave / aqueous decontamination machine. It should be cleaned after each use. Replacement potties are available from your supplier.

➤ **Seat Operation and Removal**

The seat pad is a horseshoe shape with removable cover for the aperture. When used as a shower chair, it is recommended that the aperture cover is removed to allow better access to the occupant for cleaning. Ensure weight is not applied directly to the aperture cover as this can fall through the seat.

The seat pad clips on and off the frame. To remove the seat pad for cleaning, first remove the potty and aperture cover. Then lift up the front of the seat pad to release and slide the seat pad away from the chair. To attach the seat pad first locate the seat pad by sliding the back of the seat pad into the frame, then press down on the whole of the seat pad until secure.

➤ Arm Operation

The arms can flip up to aid side transfer. Simply lift the front of the arms to flip up, and then push down to lower. The brakes should always be applied before transferring.

➤ Waist Belt

The Aston TiS chair is supplied with a waist belt to provide additional security to the user. The waist belt should be used each time that the chair is occupied. Please ensure that the belt buckles are completely locked before moving the chair, as if left unsecured, then the user could be harmed. Please see the photos below to show the correct locking of the belt.

➤ Headrest

This product also has a headrest supplied as standard. The headrest is height adjustable as well as being able to be positioned in a way that will provide the most comfort to the user. The pictures on the next page will show the various adjustments that can be made with the headrest to provide the greatest level of comfort.

These are just a few of the different configurations that are possible with this adjustable headrest. If a greater range of movement is required, the clamp section of the headrest can be removed & turned around by 180° so that the star handle is directly behind the back pad. This will allow the headrest pad to be closer to the back of the users head.

➤ **Tilt**

This product has been specially designed to allow the seat section to tilt to allow the user to be tilted backwards, making it easier for a carer to wash the users hair & body when required. To tilt the chair press the handle that is connected to the backrest (as shown in picture below) & gently tilt the chair to the desired angle. The chair has a maximum tilt angle of 30°.

Once the desired tilt angle has been reached, release the handle & the chair will lock into the desired position. To return to the original angle, press the handle & gently pull the chair back up to the seated position.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

Type of check	Method	Frequency
Seat and backrest	Adjustment/visual check	Before each use
Fasteners	Adjustment	1 time/month
Wheels	Adjustment	1 time/month
Brakes	Adjustment	Before each use
Armrests + footrests	Adjustment	Adjustment 1 time/month

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- If the device is subject to heavy use and/or use in a multi user environment then we recommend the device is serviced more frequently.
- Worn tyres, wheel bearings, wheels, brakes, leg rest locking mechanism, seat and back cushions, armrest cushions, quick release pin, pot and lid, "E" clips) should be repaired, replaced, adjusted and/or lubricated if necessary.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- To ensure optimal product life, check before each use and verify as follows:

Tyre wear	Arm Pads
Wheel bearings	Quick Release Pin
Wheels	Potty and Lid
Brakes	E' clips
Leg rest locking mechanism	
Seat and Back Pads	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Aston Pro Tilt Shower/WC chair - 822070	
Overall dimensions	Width 61.5 cm x Depth 96 cm x Height 124.5 / 129.5 cm
Seat dimensions	Width 51 cm x Depth 42 cm x Height 54 / 59 cm
Backrest height	22 cm wide
Width between armrests	Width 51.5 cm
Weight	18.5 kg
Maximum authorised weight	160 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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