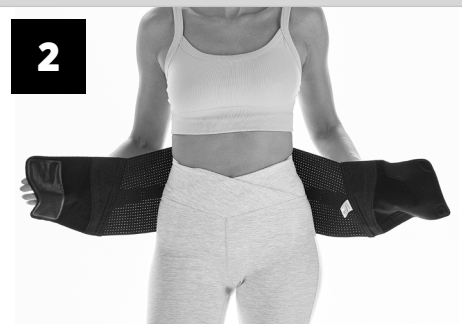
ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER

The intended user should be a licensed healthcare professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS

The Donlay Skinstrap is designed to provide comfort and support of the lumbar spine. It may be suitable for the symptomatic relief of acute and chronic low back pain. Provides compression without significant restriction of mobility.

- Acute lower back pain
- Chronic low back pain
- Muscular weakness
- Lumbosacralgia
- Lumbalgia
- Lumbar arthritis
- Spondylolisthesis

PERFORMANCE CHARACTERISTICS

- Providing support and stabilization to the limb or body segment.
- Providing partial immobilization or controlled movement to the limb or body segment.
- Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional
- Do not use if you are allergic to any of the materials contained within this product.

WARNING AND PRECAUTIONS

- We recommend that the first application be made under supervision of a healthcare professional.
- If you have skin conditions/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a healthcare professional.
- If you experience any discomfort, pain, swelling or sensation changes, or any unusual reactions while using this product, consult your healthcare professional immediately.
- This product is for single patient multiple use only.
- Do not use this device if it was damaged and/or packaging has been opened.
- The product should be removed during long periods of rest.
- Do not use during showering or bathing.
- Keep the product away from children.
- Do not use on open wounds and/or sensitive skin.
- Do not use this product if it is damaged or defective.
- Do not modify the device.
- Do not use this product in any way that may damage its integrity or use the device other than intended.
- The frequency and duration of use should be determined by a healthcare professional.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

APPLICATION INSTRUCTIONS

If possible, the back support should be put on when standing.

Open the belt.

1. Place the belt around the waist, placing your hands in the finger loops located on the front closures. Make sure that the lumbar belt is correctly centred over your spine (the straps should be positioned on either side of the spine). Position the bottom part of the belt at the top of the gluteal cleft.
2. Pull the finger loops to extend the main fabric and wrap it around your waist.
- 3a./b. Fasten the belt with the right side over the left.
- 3a./b. Fixate the cinch in the centre and pull it around your waist.
4. The 2 front straps can be removed through the slots.
5. The level of compression can be readjusted as needed during the day.

ACCESSORIES

- The Donlay Skinstrap has been designed to be used with the following optional accessories:
 - Massage back pad: The pads should face the back's curvature. Fasten the pad to the back of the belt using the hook-and-loop fasteners (arrow symbol pointed upwards).
 - Shoulder straps: These allow you to open the belt support on a need not without having to remove the entire belt. Attach the shoulder straps to the belt using the hook-and-loop fasteners.
 - Hot/cold pack with pouch: Fasten the pouch to the back belt using the hook-and-loop fasteners.

DISPOSAL

Dispose of the device and its packaging in accordance with local or national environmental regulations.

CLEANING INSTRUCTIONS

Machine washable at gentle cycle (max 40°C) with a mild detergent. Do not use fabric softener as the straps should be dried for washing to avoid any damage. Do not bleach. Rinse thoroughly. Air dry. Do not machine dry. Do not dry clean. Do not iron.

COMPOSITION

Front Closure: Polyamid, Polyester, Polypropylene, Polyurethane
Belt: Polyamide, Polyester, Polyethylene, Steel, Elastane.

NOT MADE WITH NATURAL RUBBER LATEX

FOR SINGLE PATIENT USE ONLY.

WARRANTY DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of 12 months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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