**APPAREIL DE TRANSFERT RS4***Notice d'utilisation* - DISPOSITIF MEDICAL**RS4 TRANSFER DEVICE***Instructions manual* - MEDICAL DEVICE**RS4 DISPOSITIVO DE TRANSFERENCIA***Manual de instrucciones* - DISPOSITIVO MEDICO**REF. 815238**

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The RS4 transfer device provides a safe and easy solution for a caregiver wishing to transfer a user. This transfer device is highly maneuverable and light to use.

The RS4 transfer device enables users to participate in transfers, thereby improving muscle tone. This product is ideal for transferring users from room to room, increasing their independence.

For users who can move from a seated to a standing position, the RS4 transfer device is perfect. The caregiver rotates the seat pads, and the user then stands on the provided footplate. The caregiver then lowers the seat pads, allowing the user to sit back down with their legs supported by the knee pad, and hold onto the horizontal bar in front of them, giving them a sense of security.

The four wheels, two of which are equipped with brakes, make it easy to maneuver this device from room to room. The narrow width of this device enables the caregiver to move easily through standard doorways.

The transfer device can be positioned directly in front of a toilet, with the wheels on either side, offering the user greater dignity. To sit down, the above process is simply reversed.

Note: The RS4 transfer device has adjustable width legs, allowing the device to be positioned around wider recliner chairs.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Only use the device in accordance with the above instructions. Ensure that all users are familiar with these instructions and with the operation of the device before use.
- Do not use the device if any part of it is damaged.
- Ensure that the product is in good working condition.
- Visually inspect the product.
- Only use the product on a flat surface.
- Make sure that both the healthcare professional and the patient are wearing appropriate footwear.
- Do not leave a patient unattended on the device.
- If any parts are damaged, broken, worn out, or missing, contact the distributor.
- Only use the device on flat surfaces.
- The user should not be left unattended when seated in the RS4.
- The user should not lean outside the footprint of the RS4.
- Do not exceed the maximum user weight of 185 kg.
- The brakes on the wheels must always be applied during handling.
- Be cautious when handling and locking the seat pads
- The RS4 may not be suitable for all users. A risk assessment should be conducted before use.
- Not all accessories are suitable for the RS4 and/or the user. A risk assessment should be conducted before use.
- A caregiver should always counterbalance the device while it is being used by a user.
- If the RS4 is used with a wheelchair, the wheelchair brakes should be applied and the leg rests removed before use.
- Do not tilt or lift the device during use. All four castor wheels should remain in contact with the floor at all times.
- The device is designed for indoor use and should only be used on a flat surface.

Before each use, check that:

- The locking levers are securely locked.
- The brakes are working properly.
- The seat is free from cuts, damage, or wear.
- The handrail and the shin guard are firmly attached.
- The knee pad is securely fixed and undamaged.
- The footplate is securely fixed and clean.


ASSEMBLY

Box Contents :


- 1 x Base unit
- 1 x Right side frame with M10 countersunk bolt and large washer
- 1 x Left side frame with M10 countersunk bolt and large washer
- 1 x Knee support with 4 x plastic spacers and M8 star handles
- 1 x Left seat pad
- 1 x Right seat pad
- 1 x Support bar with 2 x sling anchor bolts
- 6 x M8 hex head bolts with safety washers
- 2 x Seat pad retaining plates
- 2 x Protective caps
- 2 x M8 grub screws (not shown)

Tools Required :


- 1 x 5 mm hex key
- 1 x 17 mm open-end wrench
- 1 x 14 mm socket
- 1 x 13 mm socket
- 1 x 6 mm hex key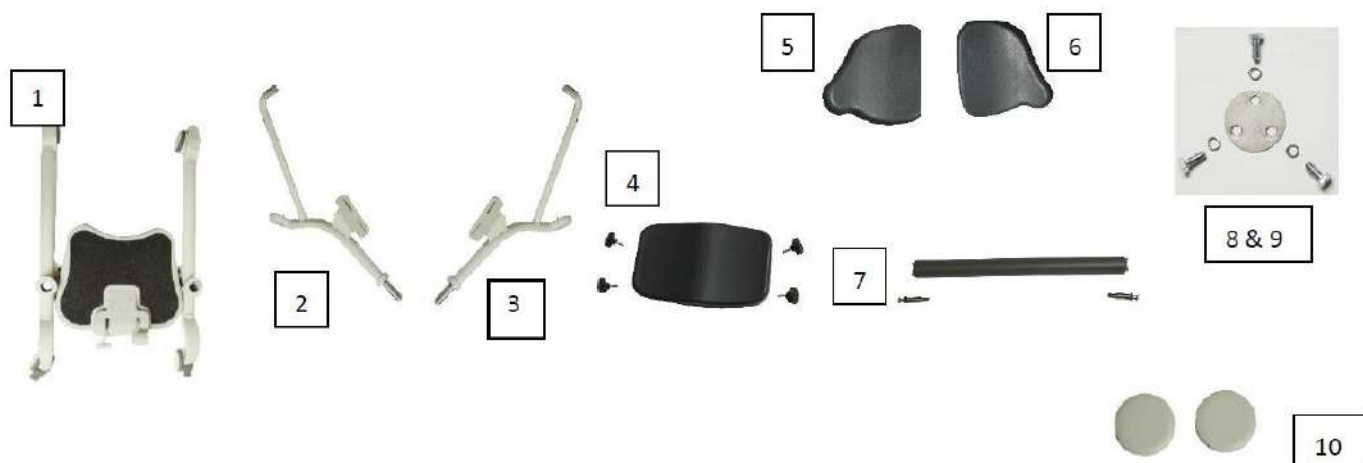

1. Select the appropriate seat cushion and insert it into the side frame.
Be especially careful to choose the correct cushion, as the cushions are specific to the left and right sides.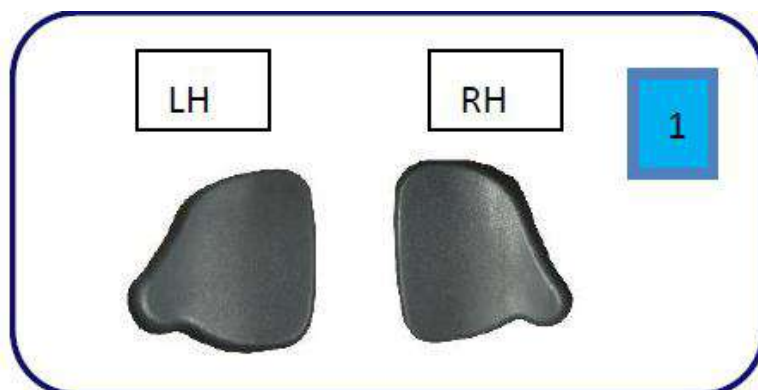

- Align pin (b) with slot (a).

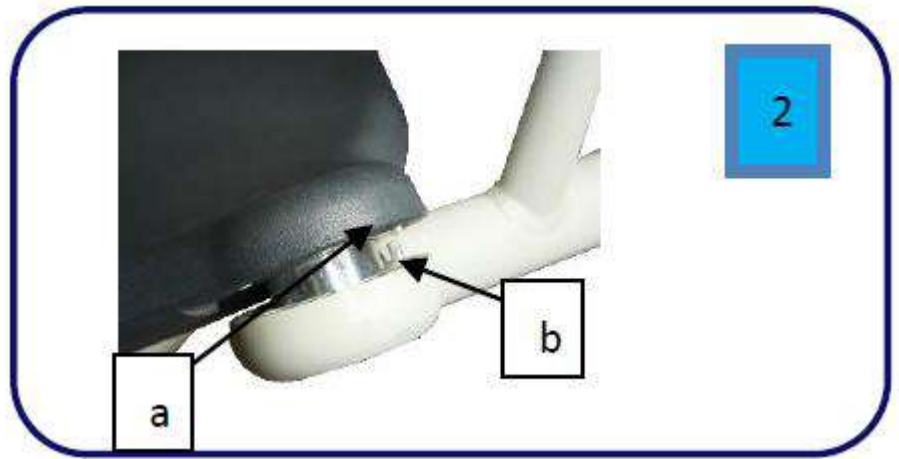- Place the safety washers on the bolts (x3).
- Position the seat cushion retaining plate and secure the bolts with the safety washers (x3).
- Attach the caps (x2).

- Repeat the process for the second side frame.
- Make sure the cushions rotate smoothly, with minimal effort and without excessive play.
- Insert the left side frame into the base unit. Repeat for the right side.

- Using the provided washers, attach the knee cushion to the side frame without fully tightening, making sure the cushion is correctly positioned.


10. Lock the knee cushion by tightening the 4 star knobs. The cushion should not slide up or down if properly positioned.
11. Locate the strap anchor bolt.

12. Insert the strap anchor bolt on the side of the push handle.
Tighten and secure the pin by locking it into the foam center of the handle.
This bolt also helps secure the foam handle in place.

13. Attach the side frame to the base unit by inserting the large washer, then lock it in place with the M10 countersunk bolt.
Repeat the process on both sides.

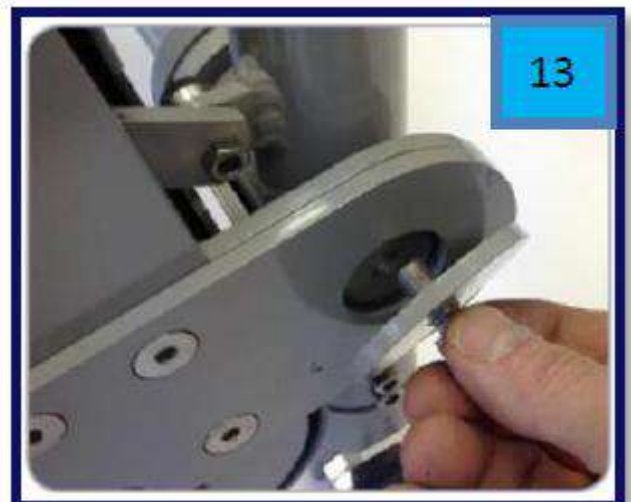14. Insert the M8 set screw on the side of the base unit.
Repeat on both sides.

15. The unit is now fully assembled.

USE

Transfer to the RS4

1. Position the RS4 close to the user requiring a transfer.
2. Spread the legs of the RS4 to provide greater stability. To do this, press the pedal as indicated below.

3. Activate the brakes of the rear wheels by pressing the locking tab.

4. Lift the seat cushions to allow the user to sit in the RS4.

5. The user's feet should be positioned on the footrest. The user should then grasp the support bar. Using the bar for assistance, the user should be able to stand up and press the front of their legs against the knee support cushion.
6. Once the user is fully standing, the seat cushions should be lowered so they can lean against them. Once the user is in a secure position, release the brakes of the wheels by lifting the locking tab. The caregiver can then move the RS4 using the push handles as indicated below.

Transfer from the RS4

1. Position the RS4 close to the chair or bed.
2. Spread the legs of the RS4 to provide greater stability. To do this, press the pedal as indicated below.

3. Activate the brakes of the rear wheels by pressing the locking tab.

4. Ask the user to stand up using the support bar. Lift the seat cushions.

5. The user can now lower themselves slowly into the seat using the support bar. The caregiver should hold the RS4 by the push handles during this operation.

6. The user must now remove their feet from the footrest, and the caregiver can release the brakes of the wheels. The RS4 can then be moved for storage.

Using the RS4 with a Chair

The RS4 can be used to assist a user in standing up from a chair, a toilet, or a wheelchair. If used with a wheelchair, the wheelchair brakes must be applied, and the leg rests should be removed before using the RS4.

1. Position the user as indicated, so they are facing the RS4, with their feet on the ground.

2. Spread the legs if necessary and ask the user to place their feet on the footrest. Lift the seat cushions and apply the brakes of the wheels.

3. The user must grasp the support bar with both hands and use it to help themselves stand up.

4. Once the user is fully standing, the caregiver must lower the seat cushions, and the user can then sit down.

5. The caregiver can then release the brakes and propel the RS4, including the user, to their destination.

Knee Cushion Adjustment

The knee cushion of the RS4 can be adjusted in height. This is done by loosening and then tightening the four adjustment buttons of the knee cushion, as shown in the image below.

To adjust the height of the knee cushion, loosen the four adjustment buttons of the cushion by turning them counterclockwise (as shown below). Reposition the knee cushion to the desired height, then tighten the four adjustment buttons by turning them clockwise until they are hand-tight. (It is important to check each button after tightening all of them, as they may have loosened slightly while adjusting the other buttons.)

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product daily with warm water and a soft cloth. Do not use abrasive products, detergents, or solvent-based products. For example, a mild disinfectant based on chloride (a solution of 10,000 parts per million) can be used. Excess water should then be removed from the device using a towel or compressed air tool. Place the device in an appropriate environment for complete drying.
- Dry with a clean, dry soft cloth.
- To prevent cross-contamination, the RS4 unit must be thoroughly cleaned between uses by different patients/users.
- Please note that the blue polyurethane seat cushions should not be cleaned with a chlorine solution.
- Caution:
- If a hot air dryer is used to dry the equipment, the temperature must not exceed 80°C (176°F). Do not use petroleum-based solvents or similar products, as they may damage plastic parts.
- This device is not suitable for autoclave cleaning.
- The seat is removable and can be washed in a washing machine at a temperature of 40 degrees Celsius using a biological detergent.
- After cleaning, please ensure that all parts of the device are completely dry before reusing it.
- In case of malfunction, do not attempt to repair the product yourself. Consult an authorized dealer.
- Weekly Checks:
 - The following weekly checks are recommended:

- Check the brakes as well as the secure attachment and maneuverability of the wheels.
- Ensure that the transfer aid is stable, meaning all four wheels are in contact with the ground.
- Ensure that the seat and knee cushions are securely attached, not loose or damaged.
- The wheels should be able to rotate and pivot freely.
- Check that all fasteners are tight and not corroded or damaged.
- Ensure that the foot adjustment mechanism operates smoothly and is not loose.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Respect the conditions for storing and storing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

TECHNICAL FEATURES

Total length	82 cm
Total height	106 cm
Total width	53 cm (closed), 73 cm (open)
Dimensions of the knee cushion	Length 39 cm x Height 26 cm
Height of the knee cushion from the base	25–49 cm
Height of the handle (from the base)	90 cm
Height of the seat	(From the ground): 62 cm (low), 67 cm (high)
Length of the chassis	87 cm
Maximum allowable weight	185 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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