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IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The walking frame with brachial supports is a walking aid for people with reduced mobility. This walking frame with brachial supports is designed for indoor and outdoor use and folds up for easy storage or transport.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

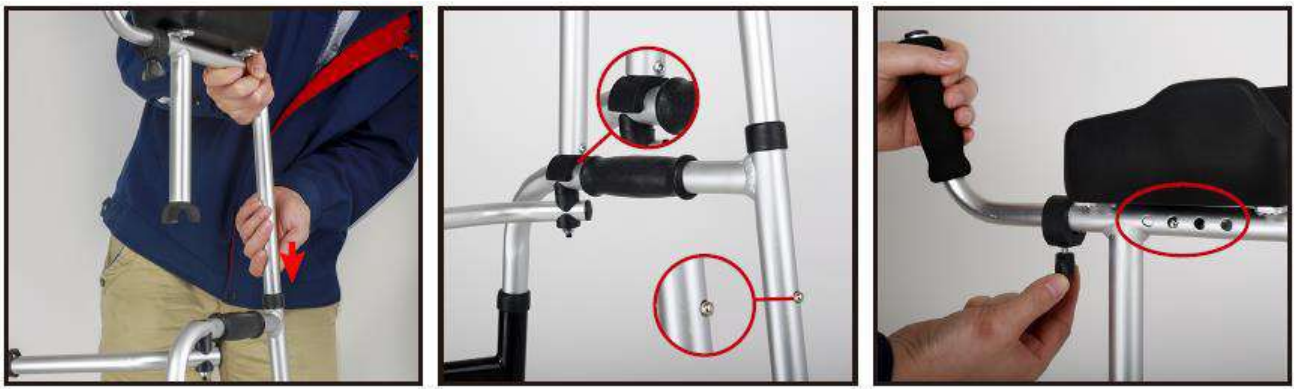
- Do not exceed the maximum weight limit of 125 kg.
- Do not use the step frame on stairs or escalators.
- Make sure that the walking frame is correctly unfolded and that all legs are adjusted to the same height.
- Only use the accessories and parts supplied by the manufacturer.
- Make sure that the knobs are tight and at the same height.
- Make sure that all parts (wheels, handles) are secure and in good working order.
- Only use the walking frame as a walking support.
- Do not use the walking frame to walk backwards.
- Always lean on the brace supports of the walking frame. Serious injury to the user and/or damage to the walking frame and/or wheels can result from improper use. Do not make any adjustments to the walking frame while it is in use.
- All wheels should always be in contact with the ground.
- Be sure to put your hands on each handle so that the walking frame can be in a parallel force position.
- Take care when climbing kerbs or when using un sloping, uneven or slippery surfaces. Failure to do so may result in a significant risk of falling or injury.
- Do not use the walker on wet, icy or snowy slopes.
- If any parts are damaged, broken or worn, contact your dealer.

MOUNTING & USE

MOUNTING:

1. UNFOLD THE WALKING FRAME

1. Remove the walking frame and accessories from the box
2. Open the two side frames until you hear a "click" (see second photo above)
3. Attach the wheels to the front of the step frame. Adjust the height by pressing the push button. A "click" will be heard showing that the height lock is correct.
4. Attach the fixed feet to the rear of the step frame in the same way as the wheels. Please check that the two front wheels and the two rear feet are all positioned at the same height (see **4- HEIGHT ADJUSTMENT OF FEET AND WHEELS**).
5. Position the brace supports on the top of the step frame and using the snaps. A "click" will be heard showing that the height lock is correct.
6. Adjust the length of the brace using the knob and snaps on the side of the brace (see **3 - ADJUSTING THE BRACEAU SUPPORTS**).

2. FOLDING THE WALKING FRAME

To fold the walking frame with brace supports, press the locking piece on the front of the frame. Then fold in the side frames.

3. ADJUSTMENT OF THE BRACHIAL SUPPORTS

1. Unscrew the armrest adjustment knob to release the handle
2. Adjust the handle horizontally to the length of your forearm
3. Screw the brachial support adjustment knob back on
4. Repeat for the other armrest. Make sure that both supports are positioned at the same distance.

4. ADJUSTING THE HEIGHT OF THE LEGS AND WHEELS

1. Press the push button to release the foot or wheel
2. Slide the tube to adjust to the desired height
3. The height of the foot or wheel is correctly locked when you hear a "click".
4. Make sure both legs and wheels are at the same height before use

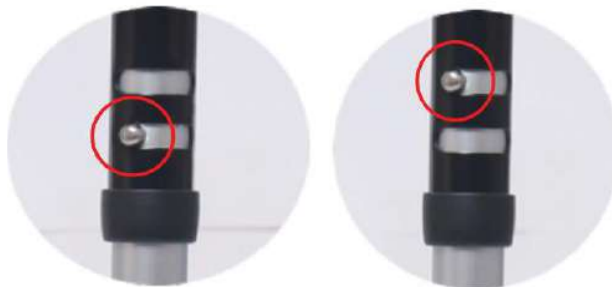
5. FIXED OR ARTICULATED FUNCTION

Fixed motion function: Allows both wheels of the rollator to move forward at the same time.

Press the push button to place it in the upper hole. The walking frame is correctly attached when you hear a "click".

Articulation function: Allows the two wheels of the rollator to move forward alternately.

Press the push button to place it in the lower hole. The walking frame is correctly attached when you hear a "click".

USE:

To use the walking frame place your forearms on the brachial supports and your hands on the handles. Adjust the height if necessary. Move forward slowly.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the rollator regularly with warm water and a soft cloth. Do not use abrasive products, detergents or solvents.
- Dry with a clean, dry soft cloth.
- Make sure there is no accumulation of dirt or debris around the axle, brakes or wheels.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Lubricate the wheels with a little oil if they squeak.
- Periodically check the wear and strength of all screws and bolts. Replace all broken, damaged or worn parts immediately.
- Check that the brakes are working before each use.
- Check the parking brakes by pushing the handles into the locked position, then trying to push the rollator.
- If there is a malfunction, do not repair the rollator yourself. Consult an authorized dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Dimensions	Width 66 x Depth 53 x Height. Brackets from 96 to 112 cm in 7 positions
Adjustable height	73 to 90 cm in 8 positions
Width between the 2 wheels	47 à 62 cm
Handle height	106 à 121 cm
Width between the handles and between the two wheels	52 cm
Folded width	12 cm
Weight of the walking frame	4,2 kg
Maximum load	125 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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