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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Flashing safety canes are walking aids for people with reduced mobility. The canes are foldable, made of aluminium and easy to transport. Fitted with a directional LED light on the front and an alarm system on the side. Comes with a wrist strap.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

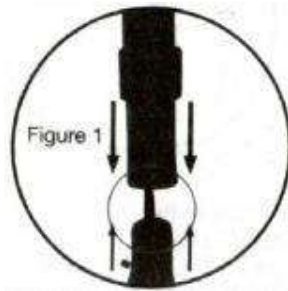
Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 110 kg.
- Only use the accessories and parts supplied by the manufacturer.

- Visually check the overall condition of the product (tube, handle, strap, stand, lamp).
- Make sure that the height adjustment clips are fully inserted before each use and that the anti-noise screw is tight.
- To prevent falls, always make sure that the different segments of the cane are locked in place and that the locking button is fully engaged.
- Always ensure that the swivel foot of the pole is securely screwed to the pole to prevent falls.
- Make sure all parts are secure and in good working order.
- Not suitable for people with latex allergies.
- Do not make any adjustments to the product while it is in use.
- Always use the handle for support.
- If any parts are damaged, broken or worn, contact your retailer.

ASSEMBLY AND USE

To unfold your cane, hold the handle and let the foldable rod segments unfold towards the ground. To prevent falls, you should always check that the individual rod segments (Fig. 1) are securely locked in place.

CAUTION

DO NOT dismantle the body of the pole. The cane contains an elastic cord inside. Uncontrolled release of the elastic cord inside can cause serious injury to unprotected parts of the body, particularly the eyes. If the elastic cord is cut, frayed or damaged, stop using the cane.

DO NOT stretch the cord more than necessary to dismantle or assemble the segments.

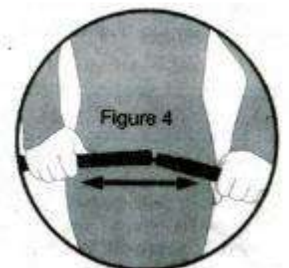
TO ADJUST THE HEIGHT OF YOUR CANE

Loosen the twist lock by turning it clockwise several turns. Press the lock button and pull the handle upwards to lengthen the cane, or push it downwards to shorten it. Don't forget to retighten the twist lock after adjusting the cane to the desired height.

TO FOLD THE CANE

Your cane is easily folded into a convenient travel size by separating the 4 individual rod segments from each other.

Hold each set of rod segments close to the joint and pull them apart. The individual rod segments fold away neatly.

TO USE THE LIGHTS ON YOUR CANE

The button at the top of the handle activates the lights. Pressing the button once switches on the light at the front of the cane, pressing it again switches on the red light at the base of the handle and pressing it a third time switches them off. The front light can be rotated to direct the light where you need it.

REPLACING THE BATTERIES

Open the battery compartment at the back of the handle.

Remove any dead batteries and dispose of them properly.

Insert 2 AAA batteries, observing the correct polarity (positive pole facing outwards, where the spring is). Close the neck of the battery compartment.

Inspect the cane regularly for signs of wear, such as fraying of the elastic cord or wear on the rubber feet. If you see any signs of wear, stop using it immediately.

WARNING

Do not mix new and used batteries. Do not mix alkaline batteries, standard (carbon-zinc) or rechargeable (nickel-cadmium) batteries.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth to prevent oxidation.
- Make sure there is no build-up of dirt or debris around the end caps.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminum and polyamide materials.
- Periodically check all rubber fixings, handle, lamp and feet for wear and strength.
- Before each use, check that the rubber end cap is in good condition.
- Before each use, check that the height adjustment clips are fully extended and the anti-noise screw tighten.
- In the event of a malfunction, do not repair the product yourself. Consult an authorized dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
 - In a dry and temperate place
 - Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL FEATURES

Overall dimensions	Length 83 to 95 cm in 5 positions
Weight	0,35 kg
Maximum supported weight	110 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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