**PROTECTION DE BARRIERE DE LIT EN
MOUSSE FORMEE**

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EN

IMPORTANT

Read this leaflet carefully before using this product and keep it for future reference.

User: If you are unable to read or understand any of the warnings, precautions or instructions, please contact your healthcare professional or distributor before using this product, as this may cause hazards and damage.

Consult a healthcare professional before using any medical device. He or she will advise and inform you on the most appropriate use of this product.

Distributor: This leaflet should be given to users of this product.

PRODUCT DESTINATION

The bed rail protection is a Class I medical device that complements the use of the healthcare bed. This product is attached to the bed rail with fasteners and protects the agitated patient from the kicks he might give to the bed rail. It creates a reassuring cocoon for the patient. The choice to use a bed rail protection is made by the multidisciplinary care team. It is intended for patients with senility pathology to complement their safety. Monitoring of the patient is important to ensure patient acceptance.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Check the condition of the seams each time you use it. Do not put a damaged protection on a bedrail. Do not use if straps are torn and buckles are broken.

Check that the bed rail is accepted by the patient and that he/she will not remove it. If the patient tries to remove it, do not use it and tell the referring doctor. Choose the right length for the bed rail and the type of bed.

Available in 3 lengths: 90 cm - 140 cm - 190 cm.

If you remove the cover for washing, make sure you put it back on correctly.

USE

The bed rail protection can only be fitted by the nursing staff. The bed rail protection is placed over the bed rail. It is attached to the underside of the bed rail with quick-release fasteners.

Each time it is used, check the condition of the seams and the condition of the straps and buckles. It should be checked that it fits properly and that it holds well to the gate. If the guard does not fit properly and is not suitable for the gate, do not use it.

- **Checklist :**

In order to maintain the product in good condition, check regularly as follows:

Type of control	Method	Frequency
Presence of tears, untied product	Visual checking	Each use
Checks on fasteners (straps, buckles)	Visual checking	Each use

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Regularly clean the bed rail protection with a non-chlorinated bactericide (Spray, wipe).
- Do not use abrasives, detergents or chlorine products.
- Dry with a clean, dry soft cloth.
- If the cover is worn or torn, do not attempt to repair it: discard it or contact an authorised dealer to find out if it can be repaired by IDENTITES.
- The cover can be washed at 95°C and dried at 60°C with standard non-chlorinated detergents.

CHANGE OF PATIENT OR BED

The product can be reused. It must first be cleaned and disinfected in accordance with the hygiene and decontamination instructions. When changing the bed, remember to give the instructions to the new user. Beforehand, the product should be inspected to check its good condition.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling by yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may result in deterioration of the product and therefore risk of serious injury

The product can be stored between 0 and 50 °C. Keep away from flames and sparks.

Observe the conditions for storing your products:

- In a dry and temperate place
- Protect your product with a dust cover
- Do not place heavy loads on it.

FEATURES TECHNICAL

Assembly : by stitching polyester - cotton thread

Filling: polyether foam 39 kg / M³ (± 10 %)

Cover :

823100 : 41% Polyester - 59 % Polyurethane

823101 - 823124 : 37% Polyester – 63 % Polyurethane

(This article contains biocides according to EU 528/2012 / protection against microbiological deterioration.

Active ingredient : zinc-pyrithione)

823118 : 44% Polyamide - 56 % Polyurethane

(This article contains biocides according to EU 528/2012 / protection against microbiological deterioration.

Active ingredient : 3-iodo - 2 propynyl butylcarbamate.)

Accessories: Polyacetal (POM) buckle - Polyoxymethylene - 100% polypropylene strap

Overall Dimensions	823118 - 90 x 37 x 3 cm 823101 - 140 x 37 x 3 cm 823124 - 190 x 37 x 3 cm 823100 - 90 x 37 x 3 cm
Weight	823118 - 0,950 kg 823101 - 1,3 kg

823124 - 1,9 kg 823100 - 0,840 kg

PERIOD OF USE

The expected lifetime of this product is 2 years under normal use and maintenance. After this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for 2 years from the date of purchase by the user. IDENTITÉS will provide replacement parts for any part that has broken due to a manufacturing, design or material defects. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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