CANNE TRIPODE CLASSIC
CANNE QUADRIPODE CLASSIC

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DISPOSITIF MEDICAL

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

Tripod and Quadripod Classic canes are walking aids for people with reduced mobility. They can be used indoors and outdoors.

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the maximum weight limit of 113 kg.
- Make sure all parts are securely fastened and in good working order.
- Visually check the general condition of the product (tube, handle, stand).
- Make sure that the height adjustment clips are fully inserted before each use and that the anti-noise screw is fully tightened.
- Make sure that all parts are securely fastened and in good working order.
- Do not make any adjustments to the product while it is in use.
- Always use the handle for support.
- Place the cane as vertically as possible on the ground.
- Do not place the end of the cane too far from the body.
- Do not use the cane on a slope greater than 9 degrees or on a wet, icy or snowy slope.
- If any parts are damaged, broken or worn, contact your dealer.

USE

The Tripod and Quadripod Classic canes are easily height-adjustable for maximum support and comfort. They are fitted with an ergonomic handle for right- and left-handed users, ensuring a good grip.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Clean the cane regularly with lukewarm water and a soft cloth. Do not use abrasive products or detergents.
- Dry with a soft, clean, dry cloth.
- Make sure there is no build-up of dirt or debris around the tips.
- Do not use cleaning products containing phenol or chlorine, or any solvent-based products, as this could damage aluminium and polyamide materials.
- Before each use, check the following parts for wear and cracks: end caps, handles, tubes and screws.
- To ensure safe and continuous use of the rods, worn or cracked parts (tips, handles) must be replaced immediately with those recommended by the manufacturer. Spare parts are available from your dealer.
- If you are not going to be using your rods for a long time, we recommend that you have them checked by a specialist before using them again.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product.
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Adjustable height	816011 - Length 72.5 to 95 cm. Base 22.5 to 24 cm 816038 - Length 67 to 90.5 cm. Base diameter 14.5 cm.
Weight	816011 – 0,50 kg 816038 – 0,46 kg
Maximum supported weight	113 kg

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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