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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Fidji support arm on column is a medical device. It is a high-quality steel rail fixed to the floor that provides stable support to help the user get up and down to go to the toilet. The support is fixed to the column at the desired height. It is not intended for use in a shower.

PRODUCT DESCRIPTION

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Do not exceed the weight limit of 110 kg.
- This product must be correctly assembled, installed and maintained.
- This product must be installed by a qualified and competent person.
- Never drill through pre-stressed concrete to install the support rail.
- Always check for concealed electrical cables and water pipes during installation; also check your legal requirements regarding the bonding of foreign metal parts.
- Inspect all parts before assembly to ensure that they are not damaged during shipping. If there is shipping damage - DO NOT USE. Contact your dealer for further instructions.
- Before use, check that all screws are tight and that the product is stable.

- Make sure all connections are tight and secure before use.
- If any parts are damaged, broken, worn or missing, contact your dealer.

ASSEMBLY AND USE

ASSEMBLY

1. Position the wall plate against the wall at the required height and mark the position of the holes.
 2. Drill the holes, depending on the fixings/screws used.
- Note:** As the substrate can vary considerably, we do not recommend any particular fixing; if in doubt, consult your distributor.
3. Place the wall plate against the wall and secure it using the chosen fixings. Once mounted, check that it is securely fixed to the floor and that the arm can be raised and lowered without interference. Also check that the friction lock is effective and that the support rail will hold at any angle (i.e. it won't fall off by itself).
 4. Important: After installing the product, carry out a final tightening and inspection of all fasteners.
 5. Test the installation, ensuring that the rail is rigid and stable.

USE

- Only install the product in an area where the mounting surface and support are structurally sound.
- Modifications to reinforce the mounting surface may be necessary.
- Also check the type of support you intend to fix it to, for example brick, wood or concrete. Always check that the type of fixing you are going to use is compatible with the substrate.
- Make sure that, when correctly positioned, the rail can be raised and lowered without interference from other fittings and furniture and that the transfer can be carried out safely.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Clean the product regularly with lukewarm water and a soft cloth. Do not use abrasive products, detergents or solvent-based products.
- Dry with a soft, clean, dry cloth to prevent oxidation.
- Make sure there is no build-up of dirt or debris around the end caps.
- Do not use cleaning products containing phenol or chlorine, as this could damage aluminium and polyamide materials.
- Periodically check all screws, nuts and other components for wear and strength.
- Before each use, check that the height adjustment knob is fully operational.
- In the event of a malfunction, do not repair the product yourself. Consult an authorised dealer.

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product :
- In a dry and temperate place

- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

FEATURES TECHNICAL

Overall dimensions	Height of column: 110 cm. Length of support 76 cm. Bracket dimensions 25 x 15.5 cm / Diameter 32 mm
Maximum supported weight	110 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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