 **CHAISE DE DOUCHE MOBILE PLIANTE
MINORQUE**

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DISPOSITIF MEDICAL

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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your physician or health care professional before using any assistive device. He/she will advise and inform you on the most appropriate use in your case.

Distributor: this instruction manual must be given to the users of this product.

PRODUCT DESTINATION

The Minorca mobile folding shower chair is a medical device. It is a technical toilet aid for elderly, people with reduced mobility, disabled, injured or who lose their balance easily.

PRODUCT DESCRIPTION

Material delivered in the box:

- ~ Frame
- ~ Backrest cushion

- ~ Seat cushion
- ~ Footrest
- ~ 12-inch rear wheels (x2)
- ~ 5-inch front wheels with brakes (x2)
- ~ Bucket with lid
- ~ Knob screws (x2)
- ~ Screws and nuts
- ~ Allen key
- ~ Wrench

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

- Don't exceed the maximum supported weight of 125 kg
- Check all components are secured before use
- Make sure that the product is in good working order.
- Visually check the product
- Before each use of the product, the brakes and wheels should be checked.
- Check the stability of the chair before use.
- Only use on level ground.
- When stationery, apply the brakes on the wheels.
- Ensure the brakes are released before the chair is moved again.
- Always ensure that the device is stationary and the brakes are applied when transferring to and from the device.
- Do not lean out of the device.
- Check the wheels bearings the brakes
- In case of damaged, broken, worn or missing parts, contact the dealer.

MOUNTING & USE

- Carefully unpack and examine the product to ensure that it is complete.
- Before use, check for any signs of damage, such as cracked welding, peeling paint, cracked or damaged seat or bent tube. If you notice any of these problems, please contact us and refrain from using the product.

INSTALLATION:

Open the frame, install the rear wheels like on the picture;

Rear wheels are installed.

Adjust the knob to fix rear wheel to the frame.

Open the main frame when the rear wheels are installed well, insert the clip on both sides of the chair, as show in the picture above. Place the footrest over the metal hole of the frame, as show in the picture.

Put on the nut and tighten it with the allen key.

The footrest bracket can be deployed, as shown, to provide anti-tip protection.

Press down on the rear wheel stand to lift the front wheel and easily cross thresholds or obstacles

Put the bucket on the bucket frame, and clamp the seat panel to the back level bar. The installation is finished as show in the picture.

CARE AND MAINTENANCE

Failure to care for and maintain your product may result in the risk of serious injury.

- Regularly clean by hand the product (all wheels, seats, and fixations) with a damp cloth and a mild detergent and disinfectant. Do not use abrasive products, detergents or solvent-based products.
- Dry with a clean, dry soft cloth.
- The chair must be completely reviewed every year, with compulsory lubrication.
- It is recommended that the device is serviced annually.
- In case of malfunction, do not repair the product yourself. Consult an authorised dealer.
- Checklist :
To ensure optimal product life, check before each use and verify as follows:

Type of control	Method	Frequency
Seat and backrest check	Adjustment / Visual check	Before each use
Checking the fastenings	Adjustment	Every mix
Control of the wheels	Adjustment	Every mez
Brake control	Adjustment	Before each use

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring your product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions can lead to deterioration of the product and thus to the risk of serious injury.

- Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.
- Keep away from all flames and sources of sparks.
- Respect the conditions for storing your product:
- In a dry and temperate place
- Protect your product from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).


PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multi user environment (such as a nursing home or hospital) then service life may be reduced.

FEATURES TECHNICAL

Overall dimensions	Width 56 x depth 70 (88.5 with footrest folded) x height 95 cm.
Seat dimensions	Width 43 x depth 43 x height 52 cm
Width between armrests	49 cm
Weight	9,9 kg
Maximum user weight	125 kg

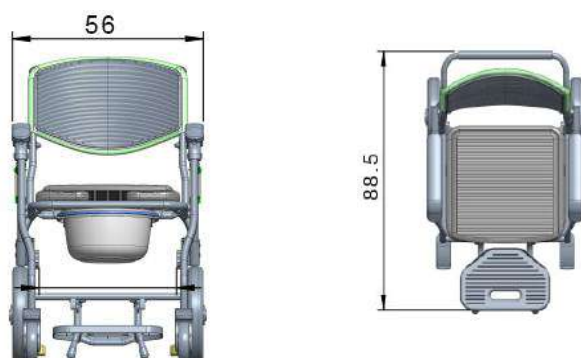GUARANTEE

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or manufacturing deficiencies. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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