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INSTRUCTIONS MANUAL EN

IMPORTANT

Read the instructions carefully before using this product and keep them if necessary.

User: If you are unable to read or understand any warnings, precautions or instructions, please contact your health care professional or distributor before using this product or risk and damage may result.

Consult your doctor or health care professional before using a assistive device. He or she will advise and inform you about the most appropriate use in your case.

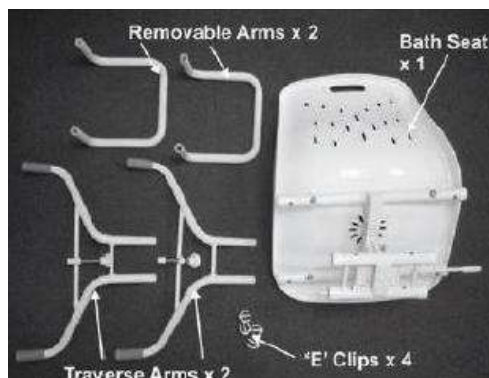
Distributor: this leaflet must be given to the users of this product.

PRODUCT DESTINATION

Swivel bath chairs are technical aids. They are intended to be used to facilitate access to the bathtub for people with reduced mobility. Without the use of any tools, the seat simply rests on the edges of your bathtub and is secured with two non-slip side stabilisers. Its all-encompassing seat is equipped with holes to facilitate water drainage.

PRODUCT DESCRIPTION

Note : Arms are liftable for the 812048

SAFETY PRECAUTIONS

Failure to follow the safety precautions may result in the risk of serious injury.

Do not exceed the maximum weight limit of 113 Kg.

Inspect all parts for shipping damage before assembly. IF there is shipping damage, DO NOT USE. Contact your dealer for further instructions.

Ensure that all four « E » clips are fully installed on the traverse arms.

Check all four corners of the swivel bath seat are resting on the top edge of the bath by at least 5 cm (2 »).

Check that the setscrews are lightly touching the inside of the bath. Do not over tighten the setscrews.

Always, sit in the centre of the swivel bath seat, and do not lean on the edges of the bath seat as this can cause instability. DO NOT stand on the bath seat.

If any parts are damaged, broken, worn or missing, contact the dealer.

ASSEMBLY

To assemble the seat:

1. Open the armrests so that they face away from the backrest.
2. Lift the seat frame up and insert it into the plastic connectors halfway up the arm frames - you will hear it "click" into place.
3. Press the seat down to connect it to the seat frame - you will hear a click into place. Make sure the seat is secure before using it.

USE

1. Lift up the lever located underneath the right hand side of the seat, and swivel the seat. Release the lever and keep swivelling the seat until it locks in position.

2. Position the seat so it is facing the user

3. The user should safely sit on the seat in the position above. The swivel lever can then be operated to allow the seat to be positioned in the bath (as shown). The user can now bathe.

4. After the user has bathed, the chair can be swivelled to the facing position for safe exit from the bath.

CARE AND MAINTENANCE

Failure to care for and monitor your product may result in the risk of serious injury.

- Wipe the swivel bath seat or shower and dry it after each use.
- Periodically, clean the bath seat with a damp cloth and mild detergent or disinfectant. For example, a mild chloride disinfectant (solution of 1000 parts per million) can be used.
- The bath seat can also be cleaned in an aqueous system (for example a Turbex machine) up to 80°C.
- Check that the bath seat swivels easily, smoothly and locks in to place. If the locking mechanism does not feel positive or there is excessive movement in the device, remove from use and contact your dealer.
- Check for cracks and damage to the bather, in particular the plastic seat and tubing.
- Do not use in environments over 60°C (saunas, for example).
- If there is a malfunction, do not repair the product yourself. Consult an authorized dealer.
- Checklist :

To keep the product in good condition, check before each use, and check regularly as follows:

Check the article	Anytime	Weekly
Check the operation of the spring loaded knob to allow 90° rotation of the product	X	
Checking the condition of the seat		X
Check the function and condition of the armrests	X	
Check that the brackets are fully attached to the edges of the bathtub	X	

CHANGE OF OWNERSHIP

The product can be reused. It must first be cleaned and disinfected in accordance with hygiene instructions. When transferring the product, remember to hand over all the necessary technical documents to the new user. Beforehand, the product must be inspected by an authorised specialist.

RECYCLING

When the product has become unusable and you have to throw it away: please contact your specialist dealer. If you wish to carry out recycling for yourself, ask your local waste management company for disposal guidelines.

STORAGE

Failure to observe the storage conditions may lead to deterioration of the product and thus to the risk of serious injury.

Do not store your product for an extended period of time near a heat source or in the sun (e.g. behind a window or near a radiator) or near a cold source.

Keep away from all flames and sources of sparks.

Observe the conditions for storing and stowing your product.

- In a dry and temperate place
- Protect your product by packaging it from dust, corrosion (e.g. abrasive elements, sand, sea water, salty air).
- Store all removed parts together in the same place (or locate them if necessary) to avoid mixing them with other products when reassembling.
- All components must be stored without any load (do not place any objects that are too heavy on the product elements, do not jam anything...).

TECHNICAL SPECIFICATIONS

812048 :

Fits standards baths with	Inside width between 54 cm and 68 cm
Overall dimensions	Width.73 x Depth 54 cm x Height 50 cm
Seat Dimensions	Larg.41 cm x prof.41 x hauteur 36 cm
Width between armrests	46,5 cm
Armrest Height from the seat	26 cm to 16,5 cm
structure	Steel
Weight	5 kg
Maximum weight	113kg

812174 :

Fits standards baths with	Inside width between 54 cm and 68 cm
Overall dimensions	Width.73 x Depth 54 cm x Height 50 cm
Seat Dimensions	Width 42 cm x depth 41 cm x height 36 cm
Width between armrests	49 cm
Armrest Height from the seat	18 cm
structure	Steel
Weight	6,4 kg
Maximum weight	113 kg

PERIOD OF USE

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions. Beyond this period, the product can be used for as long as it is still in good condition.

WARRANTY

This product is guaranteed for two years from the date of purchase by the user. IDENTITIES will provide replacement parts for any part that has broken due to a manufacturing, design or material defect. Contact the distributor from whom you purchased the product in case you need to make a claim. Failures resulting from accidental damage, misuse, alteration, or wear and tear are not covered by this warranty. Contact the dealer from whom you purchased the product to apply the warranty.

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