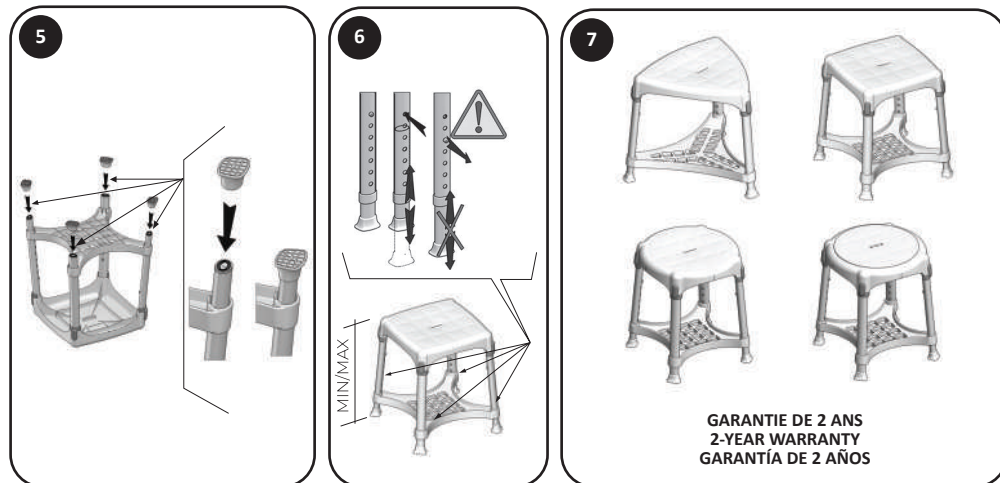
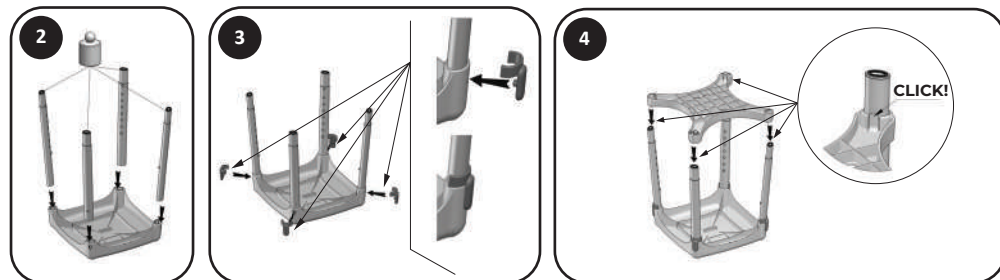
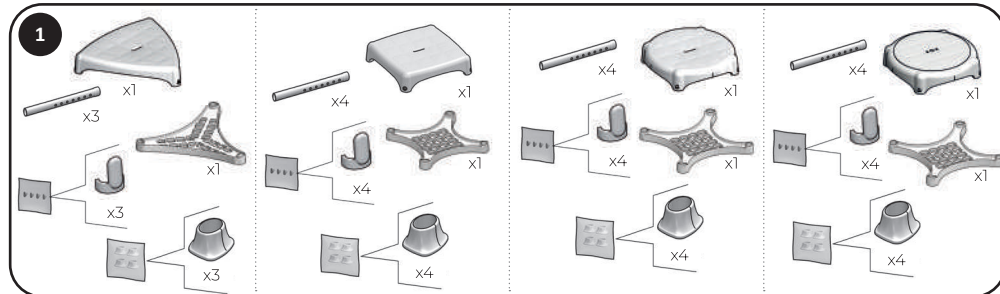
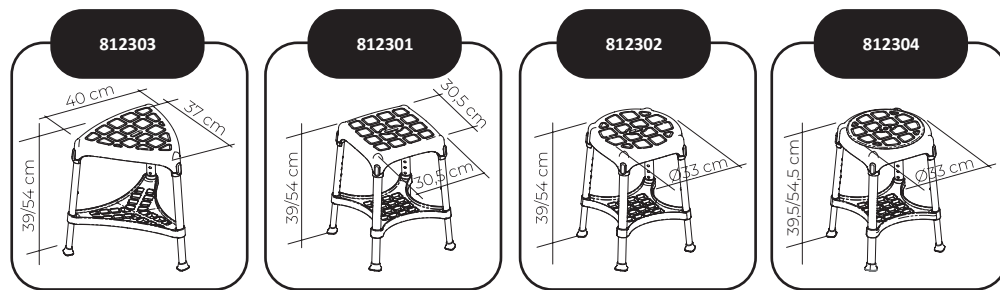
MADE IN ITALY

IDENTITÉS

**TABOURET CAPRI
TABOURET CAPRI COMPACT
Notice d'utilisation - DISPOSITIF MEDICAL**

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REF. 812001 - 812142 - 812292 - 812066
812303 - 812301 - 812302 - 812304

Scan this QR code to access
the manual in multiple languages:

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INTENDED USAGE:

The products of the Capri Stools family allow increased comfort and independence during shower or bathroom activities, reducing muscle fatigue due to standing as well as slip and injury risks. The versions provided with non-slip rubber feet ensure better grip on the floor, whereas the models with height adjustable legs allow users to change the total height according to their needs easily. The versions provided with a rotating disc increase the freedom of movement without the need to stand up from the stool. Due to its flexibility, the stools can be positioned even on a bumpy and irregular floor, like those typical of many shower trays. The stools are available in three different shapes. The stools can be easily moved due to their lightweight. The stools can be completed with a soft cushion to make the use more comfortable. The soft cushions and rotating discs are available separately as accessories.

The products of the Capri Stools family are aimed at elderly people, wheelchair users, people with reduced mobility of the lumbar, knee, or hip, and all those who have difficulty maintaining the standing position during the different activities performed in the bathroom as part of daily personal hygiene. The user is required to have autonomy of balance during sitting and standing procedures as well as to have adequate ability to keep sitting. The stool must not be used if the user has problems related to balance or perception disorders.

- The maximum user weight indicated in the fitting instructions must not be exceeded.
- Before every use, be sure the device is correctly assembled in a stable position. The device can be used only on surfaces free from obstacles.
- During assembly, strictly adhere to what is shown in the User Manual.
- Please ensure that the legs are firmly secured after adjusting the height.
- Please ensure the rubber caps are firmly installed on all four legs in the correct orientation.
- If rotating disk was installed, please make sure it is firmly attached onto the bathroom stool before using it.
- Please do not put your fingers between the rotating disc and the stool.
- Please make sure that the cushions (if purchased) are attached in the desired orientation as specified in the assembly instructions.
- The risk of finger injuries may increase during the assembly process.
- Do not swing on the device, this may lead to slipping, overturning, or subsidence of the legs. Always keep the device on its 4 legs.
- Please do not stand on the product.
- Do not modify the device and its components in any way.
- Do not use the device if damaged or altered in any way.
- After use, please move the device to a convenient spot to keep the area clear.
- Do not use the device for different purposes from the intended one.
- The device must not be stored outdoors.
- Avoid direct contact with open wounds or broken skin to prevent infection

Product maintenance consists of cleaning and sanitation procedures. In order to guarantee safe use and an adequate standard of hygiene, the user should perform these procedures before every use. The user has to make certain of the structural integrity of the device and its components, and the tightness of the screws must be checked. For cleaning and sanitation procedures, follow the steps below:

1. Gloves should be worn.
2. Remove the evident dirt before proceeding with the most accurate disinfection.
3. Use water or non-aggressive detergent for the cleaning procedure.
4. Wipe with a dry and clean cloth.

Alternatively, the product can be cleaned in domestic dishwashers at the maximum temperature. The soft cushion must be cleaned strictly by hand.

The product is suitable for reuse by other users. Before reusing by others, the product must be correctly cleaned and sanitized. Before the transfer of the product to others, the product must be checked by qualified dealers.

Do not dispose of the product in the regular household waste. Ask your local authority beforehand for the correct and environmentally friendly way to dispose of it.

The life expectancy of this product is 5 years under normal use, safety and maintenance conditions for a user. Beyond this period, the product can be used for as long as it is still in good condition.

If the device is used in a multiuser environment (such as a nursing home or hospital) then service life may be reduced.

If there is a serious incident involving this device, contact the manufacturer of this product at servicedient@identites.tm.fr and the responsible authority in your country.

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